



Welch Allyn
Staalweg 50
2612 KK Delft
The Netherlands

Welch Allyn, Inc
4341 State Street Road
Skaneateles Falls, NY
13153-0220 USA
www.welchallyn.com

CE 0297

Copyright

© Copyright 2008, Welch Allyn . All rights are reserved. No one is permitted to translate, reproduce or duplicate this manual or any part thereof, in any form, without prior written permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Unauthorized copying of this publication may not only infringe copyright but also reduce the ability of Welch Allyn to provide accurate and up-to-date information to users and operators alike.

Welch Allyn®, CardioPerfect® Workstation and SpiroPerfect® are registered trademarks of Welch Allyn.

Software in this product is Copyright 2008, Welch Allyn . All rights are reserved. The software is protected by United States of America copyright laws and international treaty provisions applicable worldwide. Under such laws, the licensee is entitled to use the copy of the software provided on the original distribution medium. The software may not be copied, decompiled, reverse-engineered, disassembled or otherwise reduced to human-perceivable form. This is not a sale of the software or any copy of the software; all right, title and ownership of the software remains with Welch Allyn.

The information contained in this manual is subject to change without notice. All changes will be in compliance with regulations governing manufacture of medical equipment.

User responsibility

This product is designed to perform in conformity with the description thereof contained in this manual and accompanying labels and inserts, when assembled, operated, maintained and repaired in accordance with the instructions provided. A defective product should not be used. Parts that are broken, plainly worn, missing or incomplete, distorted or contaminated should be replaced immediately. Should any repair or replacement become necessary, we recommend that service be performed at the nearest approved service center. The user of the product shall have the sole responsibility for any malfunction, which results from improper use, faulty maintenance, improper repair, damage or alteration by anyone other than Welch Allyn or their authorized service personnel.

Accessories

The Welch Allyn warranty can only be honored if you use Welch Allyn approved accessories and replacement parts.

**Caution**

Use of accessories other than those recommended by Welch Allyn may compromise product performance.

Warranty, Service, and Spare Parts

Warranty

All repairs on products under warranty must be performed or approved by Welch Allyn. Unauthorized repairs will void the warranty. In addition, whether or not covered under warranty, any product repair shall exclusively be performed by Welch Allyn certified service personnel.

Assistance and Parts

If the product fails to function properly or if assistance, service, or spare parts are required, contact the nearest Welch Allyn Technical Support Center.

USA	1-800-535-6663	Canada	1-800-561-8797
Latin America	(+1) 305-669-9591	South Africa	(+27) 11-777-7509
European Call Center	(+353) 469-067-790	Australia	(+61) 2-9638-3000
United Kingdom	(+44) 207-365-6780	Singapore	(+65) 6291-0882
France	(+33) 1-60-09-33-66	Japan	(+81) 3-5212-7391
Germany	(+49) 7477-927-173	China	(+86) 21-6327-9631

Before contacting Welch Allyn it is helpful to attempt to duplicate the problem and to check all accessories to ensure that they are not the cause of the problem.

When calling, please be prepared to provide:

- Product name and model number and complete description of the problem
- The serial number of your product (if applicable)
- The complete name, address and phone number of your facility
- For out-of-warranty repairs or spare parts orders, a purchase order (or credit card) number
- For parts order, the required spare or replacement part number(s)

Repairs

If your product requires warranty, extended warranty, or non-warranty repair service, please call first the nearest Welch Allyn Technical Support Center. A representative will assist you troubleshooting the problem and will make every effort to solve it over the phone, avoiding potential unnecessary return.

In case the return cannot be avoided, the representative will record all necessary information and will provide a Return Material Authorization (RMA) number, as well as the appropriate return address. A Return Material Authorization (RMA) number must be obtained prior to any return.

Note Welch Allyn does not accept returned products without an RMA.

Packing Instructions

If you have to return goods for service, follow these recommended packing instructions:

- Remove all hoses, cables, sensors, power cords, and ancillary products (as appropriate) before packing, unless you suspect they are associated with the problem.
- Wherever possible use the original shipping carton and packing materials.
- Include a packing list and the Welch Allyn Return Material Authorization (RMA) number.

It is recommended that all returned goods be insured. Claims for loss or damage to the product must be initiated by the sender.

Limited Warranty Statement

Welch Allyn, Inc. warrants that the Welch Allyn CardioPerfect Workstation computer based product you have purchased meets the labeled specifications of the Product and will be free from defects in materials and workmanship that occur within 1 year after the date of purchase. Accessories used with the Product are warranted for 90 days after the date of purchase.

The date of purchase is: 1) the date specified in our records, if you purchased the Product directly from us, 2) the date specified in the warranty registration card that we ask you to send to us, or 3) if you don't return the warranty registration card, 120 days after the date on which the Product was sold to the dealer from whom you bought the Product, as documented in our records.

This warranty does not cover damage caused by: 1) handling during shipping, 2) use or maintenance contrary to labeled instructions, 3) alteration or repair by anyone not authorized by Welch Allyn, and 4) accidents.

If a Product or accessory covered by this warranty is determined to be defective because of defective materials, components, or workmanship, and the warranty claim is made within the warranty period described above, Welch Allyn will, at its discretion, repair or replace the defective Product or accessory free of charge.

You must obtain a return authorization from Welch Allyn to return your Product before you send it to Welch Allyn's designated service center for repair.

THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. WELCH ALLYN'S OBLIGATION UNDER THIS WARRANTY IS LIMITED TO REPAIR OR REPLACEMENT OF PRODUCTS CONTAINING A DEFECT. WELCH ALLYN IS NOT RESPONSIBLE FOR ANY INDIRECT OR CONSEQUENTIAL DAMAGES RESULTING FROM A PRODUCT DEFECT COVERED BY THE WARRANTY.

Table of contents

1. GETTING STARTED.....	9
1.1 The ABP window.....	9
2. RECORDING AN ABP TEST.....	11
2.1 About recording ABP tests.....	11
2.2 Record an ABP test.....	11
2.3 Read back measurements.....	14
3. VIEWING AN ABP TEST.....	16
3.1 View an ABP test.....	16
3.2 View and add information.....	16
3.3 Views.....	16
3.4 Measurements view.....	17
3.5 Hourly intervals view.....	19
3.6 Trends view.....	20
3.7 Summary view.....	22
3.8 Scatter points view.....	23
3.9 Rise and fall view.....	24
3.10 Zoom in on a graph.....	25
3.11 Exceeding norms view.....	25
3.12 Frequency distribution.....	26
4. INTERPRETING AN ABP TEST.....	27
4.1 Edit and confirm an interpretation.....	27
4.2 View interpretation history.....	28
5. PRINTING AN ABP TEST.....	29
5.1 About printing ABP tests.....	29
6. CUSTOMIZE THE ABP MODULE.....	30
6.1 About customizing.....	30
6.2 ABP settings- General tab.....	30
6.3 ABP settings- Printing tab.....	31
6.4 ABP settings- Advanced tab.....	31
6.5 Protocols.....	32

6.6	Add or edit a protocol	33
6.7	Customize abp.txt	34
6.8	Device overview	34
6.9	ABP error list	34
7.	TROUBLESHOOTING	36
8.	INSTALLING THE ABP MODULE	37
8.1	Connect the recorder to the computer	38
8.2	Configure Welch Allyn CardioPerfect Workstation	38
9.	FUNCTION KEYS	39

General information

Welcome

Welcome to the ABP module, the ambulatory blood pressure module of the Welch Allyn CardioPerfect Workstation. The ABP module allows the user to easily record, view and print 24 hour ambulatory blood pressure tests.

This manual contains specific information about the ABP module. These instructions must be studied before putting the unit into operation. For all general information about the workstation software please refer to the Workstation manual, this describes a.o.:

- Creating and editing Patient cards
- General information about printing

For further information on installation and configuration please refer to the Workstation Installation and Configuration manual.

Intended use

The CardioPerfect Workstation (CPWS) software with associated accessories is indicated for the acquisition, analysis, formatting, display, printing and storage of certain physiologic signals, for the purpose of assisting the clinician in the diagnosis and monitoring of various diseases and/or treatment regimens. Using the optional ambulatory blood pressure (ABP) module of the Welch Allyn CardioPerfect Workstation (and associated accessories), the user can easily acquire, record retrieve, view and print 24 hour ambulatory blood pressure tests using the ABP recorder.

The ABP recorder is an automated, microprocessor controlled blood pressure monitor, which monitors, accumulates and stores heart beat (rate), systolic and diastolic data of an individual adult or pediatric patient (in the patient's environment) for a session which may last for up to 250 measurements.

Indications

Possible clinical indications for ambulatory blood pressure measurement:

- Suspected white coat hypertension
- Suspected nocturnal hypertension
- To establish dipper status
- Resistant hypertension
- Elderly patient
- As a guide to antihypertensive drug treatment
- Type 1 diabetes
- Evaluations of hypotension
- Autonomic failure
- Masked hypertension

Because of the widespread availability and application of the generic kind of device, the customer is assumed knowledgeable enough to determine if use of the device is indicated.

Contraindications and considerations


Some patients find the cuff pressure intolerable, particularly those with very high blood pressure and who have frequent repeat readings. ABPM is uncommonly associated with any complications. Petechiae of the upper arm and sometimes bruising under the inflating cuff may occur, and sleep disturbance is fairly common. In general, ABP may not be accurate during exercise or when driving, or when the cardiac rate is irregular, as in atrial fibrillation. There may be technical reasons why ambulatory readings fail in some patients (e.g. problems with cuff fitting in patients with conical-shaped arms, movement artefact, tremor, weak or irregular pulse, auscultatory gap). [The Medical Journal of Australia 2002, 176(12), p. 588-592, Barry P McGrath]

Note: for recorder specific information please refer to the appropriate hardware manual.

1. Getting started

1.1 The ABP window

The ABP window is the central part of the ABP module.

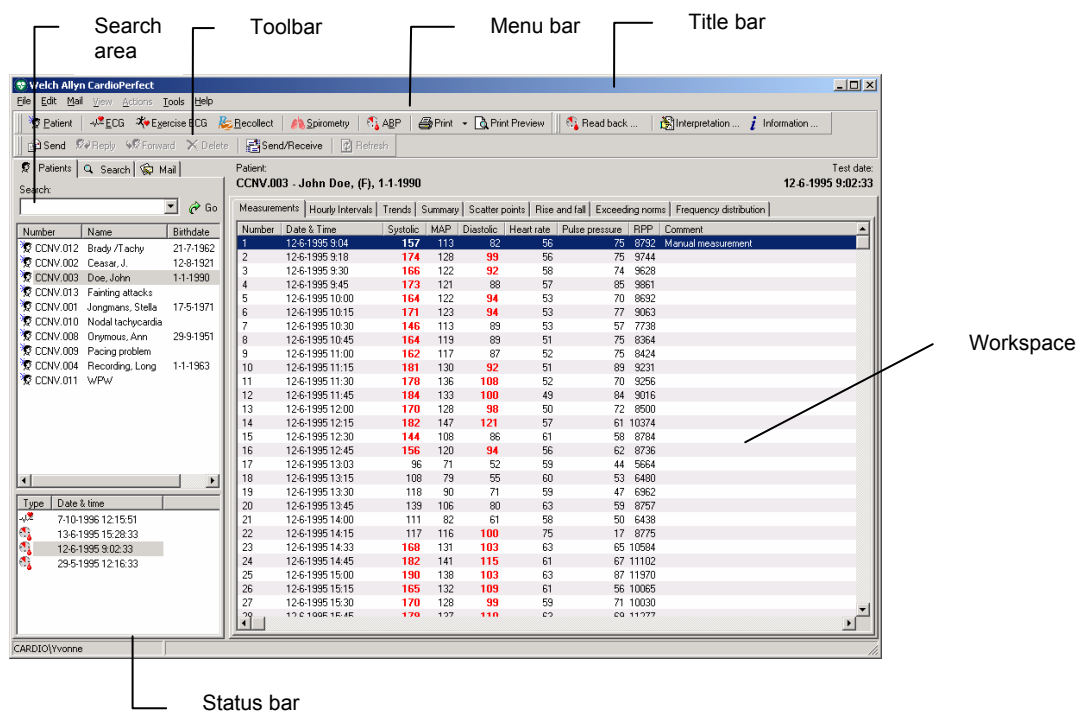

Title bar The title bar displays the name of the program. The title bar contains three buttons that you can use to maximize, minimize and close the ABP window.

Menu bar Each menu in the menu bar contains a number of tasks that you can perform with the ABP module. Sometimes, menus are gray. This means that you cannot access that functionality at that particular point, either because it is not available, or because you are not authorized to use it.

Tool bar The tool bar contains buttons that give you easy access to the most common tasks in ABP, such as recording and printing a test, reading back the measurements, interpretation and test information.

Search area The search area on the left hand side contains search and display functionality that lets you easily retrieve patients and tests. In the search area, you can find a patient, see which tests were recorded for that patient and see the kind of tests that were recorded. You can also create search patterns, an easy way of finding information that you need frequently.

- Workspace** The workspace displays tests and test related data, such as interpretations, measurements and comments. This is where you will do the actual recording, viewing and interpretation. The ABP workspace contains a viewing area with a number of tabs that let you see different aspects of the test. The tabs, also called views, that are available are: Measurements, Hourly intervals, Trends, Summary, Scatter points, Rise and fall and Exceeding norms.
- Shortcut menu** In the workspace, you can use a shortcut menu to access the most common tasks. You can access these tasks by clicking on the workspace with your right mouse button.
- Status bar** The status bar at the bottom of the window shows the current user.

2. Recording an ABP test

2.1 About recording ABP tests

An ambulatory blood pressure test consists of two parts:

- Starting a test and preparing the ABP recorder for use.
- Reading back the recorded measurements to the computer.


For both parts, the ABP recorder needs to be connected to the computer. Please refer to the ABPM 6100 hardware manual for instructions on how to handle and maintain the ABPM 6100 recorder.

2.2 Record an ABP test


Follow these global steps. Each step is explained in more detail on the following pages.

To record an ABP test:

1. Connect the blood pressure recorder to your computer.
2. Select a patient, or create a new one.
3. In the **File** menu, point to **New**, and click **Ambulatory blood pressure**. The ABP module starts to communicate with the ABP recorder.
4. If two devices are connected select which device should be used for the recording in the window that pops up.

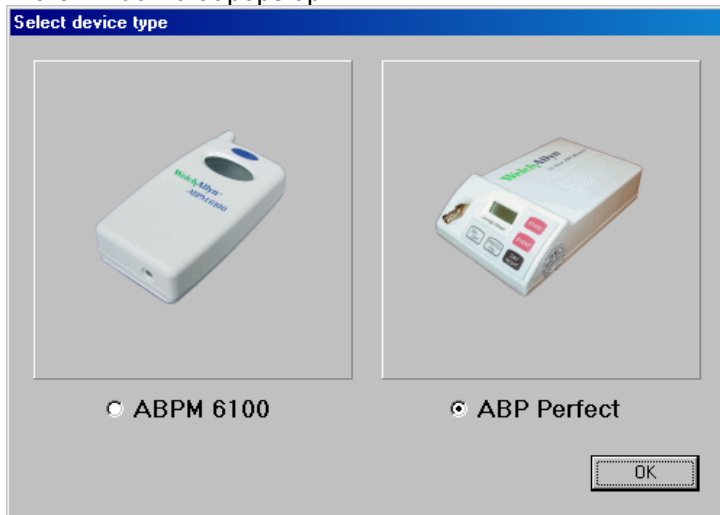

5. If the **Name new device** dialog box is displayed, enter the name of the blood pressure recorder.

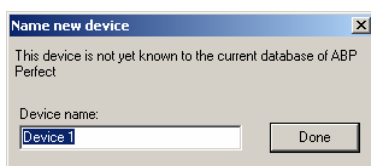

6. After this step, a number of situations can occur:


<p>If the recorder...</p> <ul style="list-style-type: none"> Is empty Does contain measurements from previous tests. 	<p>Continue with...</p> <ul style="list-style-type: none"> Enter patient and protocol information Store previously recorded measurements. Enter patient and protocol information
---	--
7. Disconnect the blood pressure recorder from your computer and place it in the pouch. The patient wears the recorder for 24 hours.
8. When the patient returns, connect the blood pressure recorder to the computer again, and read back the measurements.

Enter patient and protocol information

Before you can record an ABP test, you first need to enter patient and protocol information. This information is saved together with the recording.

You can enter patient and protocol information in the **New Ambulatory Blood Pressure Test** dialog box. This dialog box is displayed:

- Immediately when you start a new test with an empty ABP recorder.
- After deleting the measurements when you start a new test with an ABP recorder that was not empty.

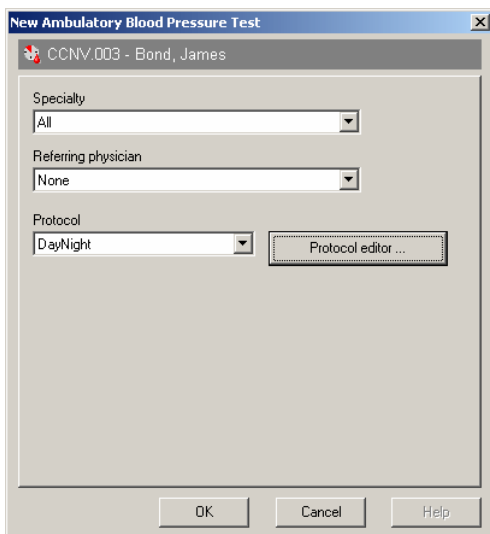

To enter patient and protocol information:

1. From the **Specialty** list, select the specialty of the physician that ordered the test.
2. From the **Referring** physician list, select the name of the physician that ordered the test.
3. From the **Protocol** list, select the protocol that you want to use for the test.
4. Click **OK**. The ABP module shows that the test is in progress.

Please refer to chapter 6.5 and 6.6 for more information on protocols and the protocol editor.

Store previously recorded measurements

The recorder can contain measurements from previously recorded tests. You can choose to save those measurements to the database before deleting them from the recorder.

How the measurements are saved depends on the patient for which the measurements were recorded.

If the measurements belong to...

The current patient

Another patient

An unknown patient

Then you can...


Read back the measurements and save them to the patient's test in progress.

Assign the measurements to a test in progress for that patient.

Create a new patient and assign the measurements to the new test for that patient.

To read back the measurements and save them to the patient's previous test:

1. If you try to record a test for a patient who already has a test in progress, the following message is displayed:

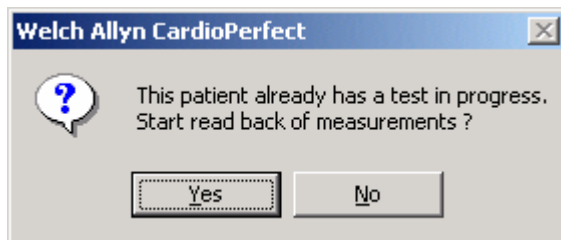


2. Click **Yes**. The measurements are read from the ABP recorder and saved to the patient's previous test.

Note: if you click No, you cannot start a new test for the current patient, because a patient can only have one test in progress.

To assign the measurements to a test of another patient:

1. If you want to record a test for the current patient, but the ABP recorder still contains measurements of another patient, the following message is displayed:

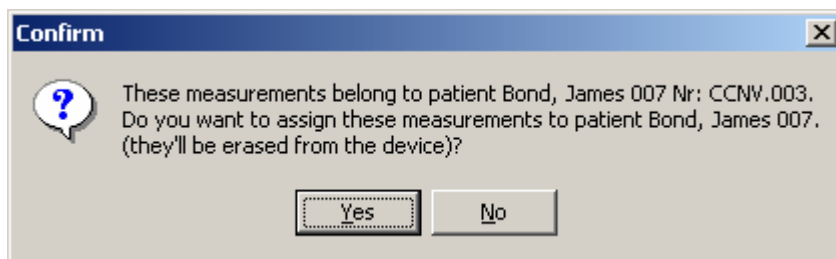


2. Click **Yes**. The measurements are assigned to the patient that is mentioned in the message. The **New ambulatory blood pressure test** dialog box is displayed, so that you can continue with recording a test for the current patient.

Note: if you click **No**, the measurements are deleted from the ABP recorder without being saved.

To assign the measurements to a new patient:

1. If you want to record a test for the current patient, but the ABP recorder still contains measurements of an unknown patient, the following message is displayed:

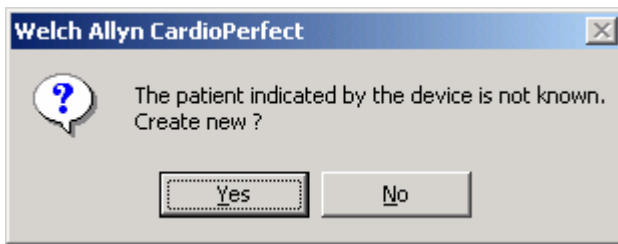

2. Click **Yes**. A new patient card is created. This card only contains a patient number. The **New ambulatory blood pressure test** dialog box is displayed, so that you can continue with recording a test for the current patient.


Note: if you click **No**, the measurements are deleted from the ABP recorder without being saved.

2.3 Read back measurements

When the patient has returned, you need to read back the measurements and transfer them to the computer. Once you have transferred the measurements to the computer, you can view them in the ABP module.

To read back the measurements and save them on your computer:

1. Connect the ABP recorder to your computer. The **In progress** dialog is displayed.
2. Press the **Read measurements** button. The **Ambulatory Blood Pressure Measurements read back** dialog box is displayed.

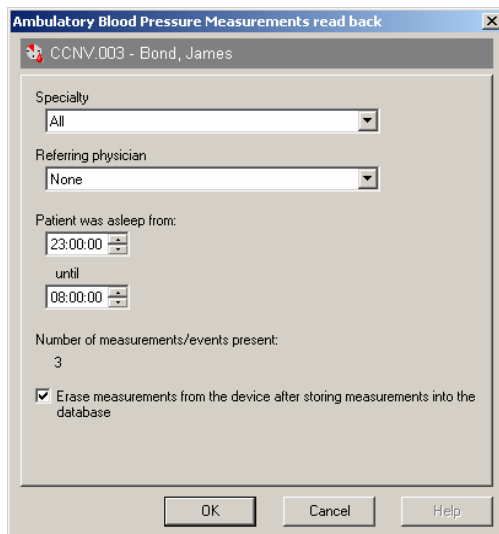

3. This dialog box displays the specialty and the name of the referring physician that ordered the test. In addition, it shows:
 - The period that the patient was asleep.
 - The number of measurements or events that were recorded.
 4. The sleeping period is set to the default settings that were entered in the ABP settings. You can change the values to that they reflect the patient's actual sleeping period.
 5. Decide whether you want to delete the measurements from the ABP recorder. If you want to delete the measurements, select the option at the bottom of the dialog box.
 6. Press **OK** to transfer the measurements to the computer and save them with the patient that is mentioned at the top of the dialog box.
- **Note:** if you click the **Cancel** button, the measurements are not transferred to the computer. The test remains in progress.


Tips for reading back measurements:

- You can also read back the measurements by clicking the **Read back** button on the toolbar, if the ABP module is activated. This way, you do not need to select the patient before reading back the measurements, which speeds up the reading back process.

3. Viewing an ABP test

3.1 View an ABP test

To view an ABP test:


1. In the patient list, select the patient whose test you want to view. The tests that have been recorded for that patient are listed in the test list.
2. From the test list, select the ABP test that you want to see. ABP tests are indicated with . The ABP module is started automatically, and the test is displayed in the workspace.

3.2 View and add information

You can view information that was entered when the test was recorded. Additionally, you can add your own comments.

To view information about the test:

- In the **Tools** menu, click **Information**. The **Information dialog** box is displayed.

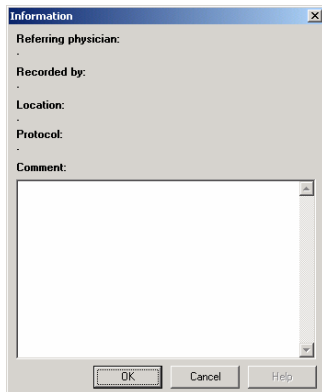

To enter your own comments:

- In the Comment section of the Information dialog box, start typing and click OK.

3.3 Views

With the ABP module, you can view ABP test results in seven different views:

- Measurements and events
- Hourly intervals
- Trends
- Summary
- Scatter points
- Rise and fall
- Exceeding norms
- Frequency distribution

3.4 Measurements view

The **Measurements** view displays a list of measurements and events. Measurements are automatically made by the ABP recorder. Events are created when the patient presses the button on the recorder.

Number	Date & Time	Systolic	MAP	Diastolic	Heart rate	Pulse pressure	RPP	Comment
1	29-5-1995 12:18	144	117	108	72	36	10368	Manual measurement
2	29-5-1995 12:30	128	107	100	75	28	9600	
3	29-5-1995 13:00	134	115	102	74	32	9916	
4	29-5-1995 13:30	155	122	112	70	43	10850	
5	29-5-1995 14:00	142	118	103	72	39	10224	
6	29-5-1995 14:30	142	113	98	74	44	10508	
7	29-5-1995 15:00	155	133	112	69	43	10695	
8	29-5-1995 15:30	145	118	105	78	40	11310	
9	29-5-1995 16:00	142	123	99	72	43	10224	
10	29-5-1995 16:30	145	133	106	68	39	9860	
11	29-5-1995 17:00	138	111	94	72	44	9936	
12	29-5-1995 17:30	155	129	108	78	47	12090	
13	29-5-1995 18:00	172	143	131	75	41	12900	
14	29-5-1995 18:30	166	148	124	78	42	12948	
15	29-5-1995 19:00	156	135	114	78	42	12168	
16	29-5-1995 19:30	148	113	102	76	46	11248	
17	29-5-1995 20:00	152	122	108	77	44	11704	
18	29-5-1995 20:30	142	125	94	82	48	11644	
19	29-5-1995 21:00	144	124	98	74	46	10656	

Information

For each measurement, the following information is presented:

- Number, date and time of the recording.
- Systolic and diastolic pressure.
- Mean arterial pressure (MAP)
- Heart rate
- Pulse pressure
- Rate pressure product (RPP= systolic pressure * heart rate)
- Comment

Norm values

Values that are above the norm values are displayed in bold red text.

Sleeping period

Measurements that were recorded when the patient was asleep are displayed in the grayed area of the view.

21	29-5-1995 22:00	138	122	92	73	46	10074	
22	29-5-1995 22:30	142	128	93	71	49	10082	
23	29-5-1995 23:00	133	107	94	69	39	9177	
24	29-5-1995 23:30	135	111	91	72	44	9720	
Event	29-5-1995 23:52	--	--	--	--	--	--	Pressed event button

Working with the Measurements view

In the **Measurements** view, you can:

- Assign measurements to the sleeping or waking period
- Add a comment to a measurement
- Temporarily hide a measurement from a test
- Delete a measurement

Assign measurements to the waking or sleeping period

The measurements in an ABP test are divided into two categories:

- Measurements that were taken when the patient was awake (waking period).
- Measurements that were taken when the patient was asleep (sleeping period).

This is done according to the default ABP settings. Usually this division consists of an 8-hour night and a 16-hour day. When you read back the measurements, you can change the sleeping period values so that they reflect the patient's actual sleeping period.

You can change the period to which a measurement was assigned. This can be necessary, for instance when the patient has not slept very well and decided to read a book. He might also have decided to sleep in.

To assign a measurement from the sleeping period to the waking period:

1. Right click on the measurement that you want to assign to the waking period.
2. In the shortcut menu, click **Set awake**. The gray shade is removed from the measurement to indicate that the patient was awake when this measurement was taken.

To assign measurements from the waking period to the sleeping period:

1. Right click on the measurement that you want to assign to the sleeping period.
2. In the shortcut menu, click **Set asleep**. A gray shade is added to the measurement to indicate that the patient was asleep when this measurement was taken.

Add a comment

When you want to make note of something in a measurement, you can add a comment to that measurement.

To add a comment:

1. In the **Measurements** view, click on **Comment** field of the measurement to which you want to add the comment. The field becomes active, and a pointer appears, click again on the field and a cursor appears.
2. Enter your comment. The comment is automatically saved with the test.

Temporarily hide a measurement from a test

When you do not want to view a measurement, but do not want to delete it from the test either, you can temporarily hide it.

To temporarily hide a measurement:

1. In the **Measurements** view, click the measurement that you want to hide.
2. Right click on the measurement. In the shortcut menu, click **Exclude**. The measurement number is put between brackets to indicate that this measurement is not used in the test at the moment. The measurement is not used in any of the measurements or graphs in the other views.

3. Click **Include** to make the measurement part of the test again.

Delete a measurement

When a measurement or event is not valid, you can delete it from the test.

To delete a measurement:

1. In the **Measurements** view, right click on the measurement.
2. In the shortcut menu, click **Delete**. The measurement is deleted from the test. It is not used in any of the views any longer.

3.5 Hourly intervals view

The **Hourly intervals** view displays the average of a number of values and their corresponding standard deviations at intervals of 1, 2, 3, 4, 6 or 8 hours.

Measurements								Hourly Intervals								Trends		Summary		Scatter points		Rise and fall		Exceeding norms		Frequency distribution	
1 Hour								2 Hours								3 Hours		4 Hours		6 Hours		8 Hours					
Numbers	Time	Systolic	SD	Diastolic	SD	Heart rate	SD																				
1 - 2	12 - 13 h	136	11.3	104	5.7	74	2.1																				
3 - 4	13 - 14 h	145	14.8	107	7.1	72	2.8																				
5 - 6	14 - 15 h	142	0.0	101	3.5	73	1.4																				
7 - 8	15 - 16 h	150	7.1	109	4.9	74	6.4																				
9 - 10	16 - 17 h	144	2.1	103	4.9	70	2.8																				
11 - 12	17 - 18 h	147	12.0	101	9.9	75	4.2																				
13 - 14	18 - 19 h	169	4.2	128	4.9	77	2.1																				
15 - 16	19 - 20 h	152	5.7	108	8.5	77	1.4																				
17 - 18	20 - 21 h	147	7.1	101	9.9	80	3.5																				
19 - 20	21 - 22 h	144	0.7	99	0.7	77	3.5																				
21 - 22	22 - 23 h	140	2.8	93	0.7	72	1.4																				
23 - 24	23 - 24 h	134	1.4	93	2.1	71	2.1																				
25 - 26	0 - 1 h	127	1.4	83	1.4	74	0.7																				
27 - 28	1 - 2 h	128	1.4	86	3.5	80	0.7																				
29 - 30	2 - 3 h	123	3.5	86	2.8	76	4.9																				
31 - 32	3 - 4 h	125	4.2	84	8.5	74	0.7																				
33 - 34	4 - 5 h	124	4.2	83	1.4	77	2.1																				
35 - 36	5 - 6 h	123	0.0	80	0.7	69	1.4																				
37 - 38	6 - 7 h	138	9.2	91	4.2	79	4.2																				
39 - 40	7 - 8 h	156	3.5	110	9.9	85	4.2																				

Information

The averages of the following values are displayed:

- Systolic pressure
- Diastolic pressure
- Heart rate

Working with the Hourly intervals view


In the **Hourly intervals** view, you can change the intervals for which the values are displayed.

To change the intervals for which the values are displayed:

- Click on one of the interval buttons at the top of the Hourly interval view. The interval is changed.

3.6 Trends view

The **Trends** view shows a graphical representation of the ABP test.

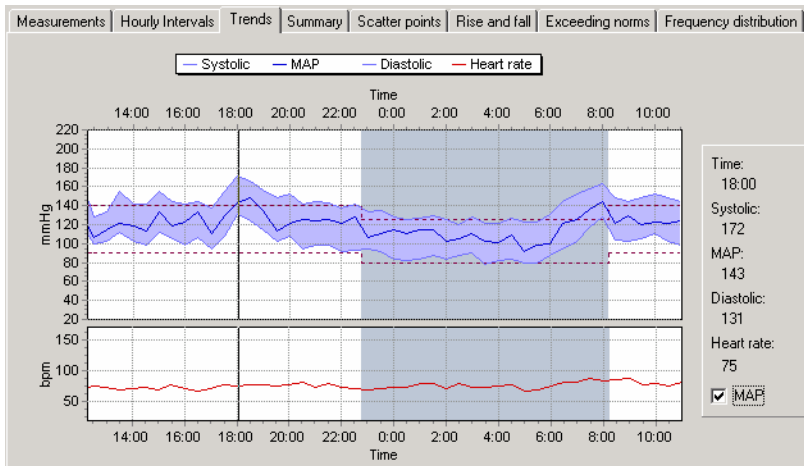

Information

- The upper chart displays the systolic pressure, MAP (when selected) and diastolic pressure
- The lower chart displays the heart rate

Norm values

Upper and lower norm values are displayed in dotted lines.

Sleeping period

Measurements that were recorded when the patient was asleep are displayed in the grayed area of the view.


Working with the trends view

In the **Trends** view, you can:

- View the exact values with the measurement slider
- Compare ABP tests
- Zoom in on parts of the trends graph (see §3.10)

View exact values in the Trends view


The **Trends** view displays information in a graph. You can see the exact values of a point in the graph with the measurements slider. This is a vertical line that you can slide over the graph. The exact values for the points in the graph are displayed on the right hand side of the **Trends** view.

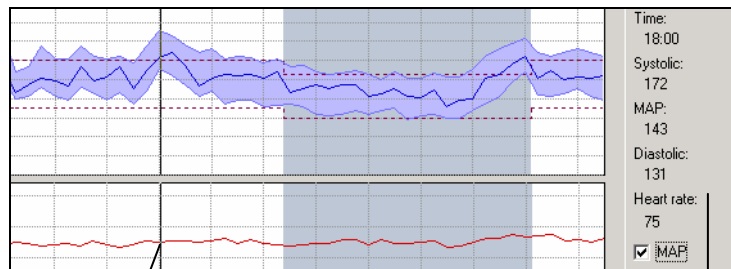

The measurements slider only gives exact values for 30 minutes intervals.


Compare ABP tests of the same patient

In the **Trends** view, you can compare the current ABP record with a previous recorded test of the same patient.

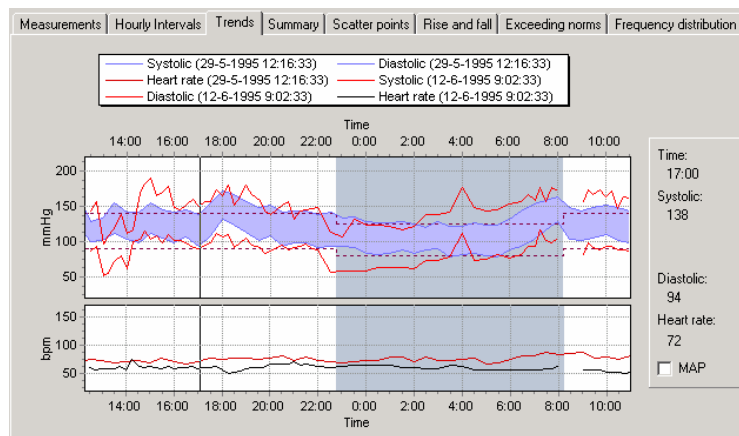

To make a comparison:

1. Right click on the graph.
2. In the shortcut menu, click **Comparison**.

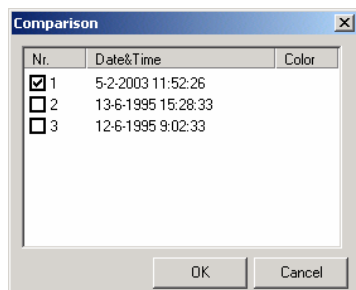

3. In the **Comparison** dialog box select the test with which you want to compare the current test and click OK.
4. The graphs for both tests are shown in the **Trends** view window. To return to the current test, click **Comparison** in the shortcut menu and click on the selected test.

3.7 Summary view

The Summary view displays statistics for general information, systolic pressure, diastolic pressure and heart rate. For each of the statistics, the values are given for the waking period, the sleeping period and the entire test.

Measurements	Hourly intervals	Trends	Summary	Scatter points	Rise and fall	Exceeding norms	Frequency distribution
General							
Nr. measurements	47	28	19				
Total time	22:42:00	13:12:00	9:30:00				
Average	141 / 98 mmHg	147 / 104 mmHg	132 / 90 mmHg				
SD	12.8 / 12.6 mmHg	9.0 / 8.8 mmHg	12.8 / 13.0 mmHg				
Pulse pressure	42.3 mmHg	42.5 mmHg	42.1 mmHg				
Systolic							
Nr. measurements	≥ 140 <input type="checkbox"/> 28 (60%) (<input checked="" type="checkbox"/> incl. equal)	≥ 140 <input type="checkbox"/> 24 (86%) (<input checked="" type="checkbox"/> incl. equal)	≥ 125 <input type="checkbox"/> 14 (74%) (<input checked="" type="checkbox"/> incl. equal)				
Minimum	120 mmHg	128 mmHg	120 mmHg				
Maximum	172 mmHg	172 mmHg	163 mmHg				
Diastolic							
Nr. measurements	≥ 90 <input type="checkbox"/> 35 (74%) (<input checked="" type="checkbox"/> incl. equal)	≥ 90 <input type="checkbox"/> 28 (100%) (<input checked="" type="checkbox"/> incl. equal)	≥ 80 <input type="checkbox"/> 17 (89%) (<input checked="" type="checkbox"/> incl. equal)				
Minimum	78 mmHg	92 mmHg	78 mmHg				
Maximum	131 mmHg	131 mmHg	128 mmHg				
Heart rate							
Average	76 bpm	76 bpm	76 bpm				
SD	4.9 bpm	4.6 bpm	5.3 bpm				
Minimum	68 bpm	68 bpm	68 bpm				
Maximum	88 bpm	87 bpm	88 bpm				
Awake/asleep decrease, early morning							
Decrease:	9.3% / 13.4% mmHg (Not a dipper)						
Morning average:	149.0 / 104.0 mmHg						

Information

The general statistics consist of:

- Number of measurements
- Total recording time
- Average blood pressure
- Standard deviation of the blood pressure
- Average pulse pressure

The systolic and diastolic pressure statistics consist of:

- Number of measurements that is greater than or equal to the norm value that is displayed in the field.
- Minimum value that was measured
- Maximum value that was measured

The heart rate statistics consist of:

- Average
- Standard deviation
- Minimum heart rate that was measured
- Maximum heart rate that was measured

Awake/asleep decrease

The decrease in blood pressure in the sleeping period compared to the blood pressure in the awake period is expressed in the decrease percentages at the bottom of the view.

Morning average

The average blood pressure in the first hour after the sleep period is expressed in the morning average at the bottom of the view.

Working with the summary view

In the **Summary** view, you can:


- Change the norm values for the systolic and diastolic blood pressure, and see how many measurements are above the norm value.

Change the norm values

The Summary view displays how many measurements are greater than or equal to the norm values. The norm values are set as a default in the ABP module. You can change the norm values for one test.

To change the norm values:

In the **Summary** view, enter a new norm value in one of the norm value fields.

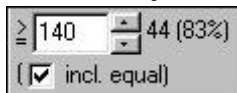


3.8 Scatter points view

A scatter plot reveals relationships or association between two variables. Such relationships manifest themselves by any non-random structure in the plot.

The **Scatter points** view displays the correlation between the following values:

- Systolic pressure against heart rate
- Diastolic pressure against heart rate
- MAP against heart rate
- Systolic pressure against diastolic pressure

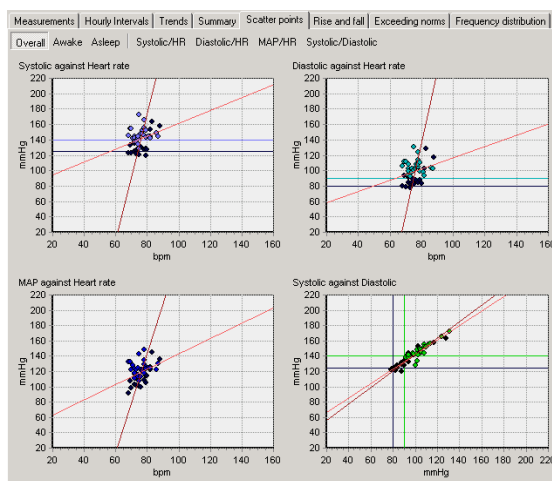

Information

One point on a graph represents one measurement. Black points were measured

during the sleeping period.

Sleeping period

Each correlation graph can be displayed for all measurements, measurements for the sleeping period or measurements for the awake period.

Working with the Scatter points view

In the **Scatter points** view, you can:

- Toggle between one graph or all graphs.
- Zoom in on a graph (see §3.10).

Toggle between one or all graphs


In the Scatter point view, the default view displays all four scatter point graphs. You can display each graph individually by clicking the graph names at the top of the Scatter point view.



You can also select whether you want to see the overall measurements, measurements for the sleeping period or measurements for the waking period.

3.9 Rise and fall view

The **Rise and fall** view displays values for the rise and fall of the blood pressure during the test.

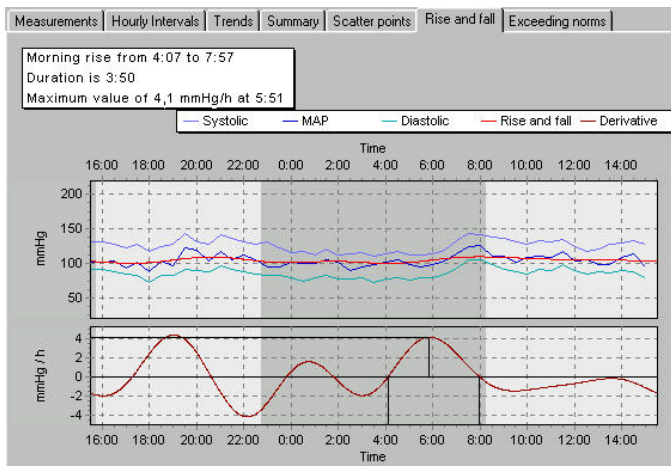

Information

The upper chart displays:

- The diastolic and systolic pressure and MAP values.
- A rise and fall curve. This curve is obtained by filtering the MAP curve.

The lower chart displays:

- A first derivation of the filtered MAP curve. This derivation clearly displays the rise and fall of the blood pressure.

Sleeping period

The measurements that were recorded during the sleeping period are displayed in the grayed area.

Working with the Rise and fall view

In the **Rise and fall** view, you can:


- Zoom in on a graph (see §3.10).

3.10 Zoom in on a graph

In the **Trends** view, the **Scatter points** view and the **Rise and fall** view, you can zoom in on a part of the graph.

To zoom in on a graph:

1. Position your mouse near the area that you want to enlarge.
2. Click your mouse button and keep it pressed. When you move the mouse from the top left corner to the bottom right corner of the graph, a square is drawn.

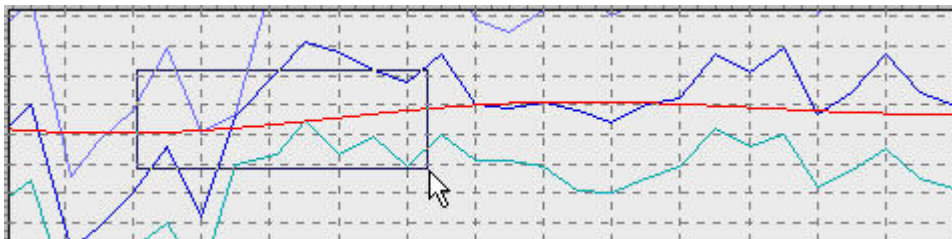

3. Drag the square over the area that you want to enlarge. The area is enlarged.

To reset the zoom:

1. Right click on the graph.
2. In the shortcut menu, click **Reset zoom**. The graph is restored to its original size.


3.11 Exceeding norms view

The **Exceeding norms** view displays the percentages of measurements that are

- Normal (less or equal than the norm value)
- Acceptable (range [100 – 125] % of the norm value)
- Critical (greater than 125% of the norm value)


See paragraph 6.2 ‘ABP settings- General tab’ on page 30 for the norm value.

The percentages are displayed in pie charts, both for systolic and diastolic pressure.

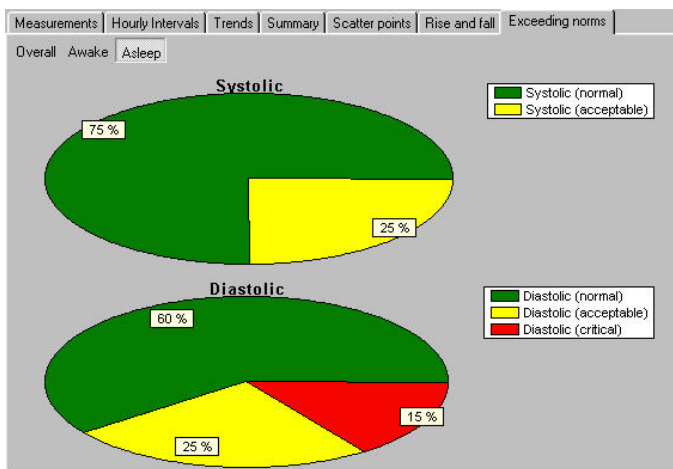

3.12 Frequency distribution

In the Frequency distribution window the frequencies of the systolic and diastolic pressure are shown divided in 10 mmHg bins. The Heart rate frequencies are shown divided in 10 bpm bins.

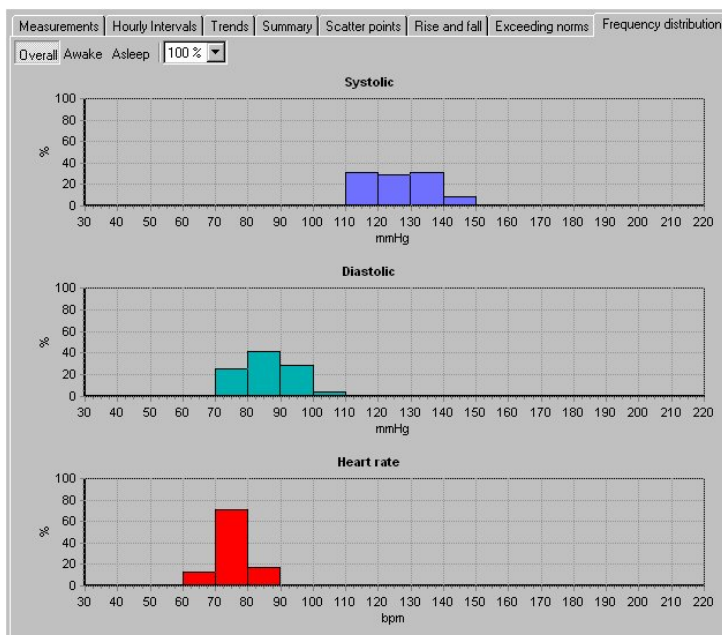

You can select whether you want to see the overall graphic, the graphic for the sleeping or the waking period and select which percentage is shown.


4. Interpreting an ABP test

4.1 Edit and confirm an interpretation

You can edit an interpretation in the Interpretation editor. In this editor, you can enter your own texts. Alternatively, you can use the statement tree to select and enter common interpretation statements. Once you edit an interpretation, you must confirm it. Otherwise, your changes will not be saved.

To edit and confirm an interpretation manually:

1. In the **Tools** menu, click **Interpretation**. The **Interpretation editor** is displayed.

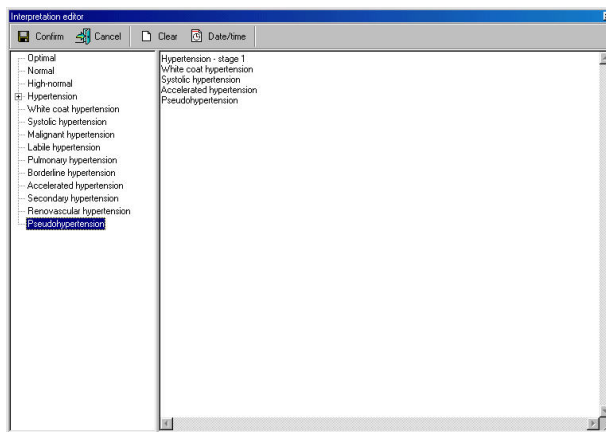

2. In the right hand pane, at the pointer, start typing.
3. Click the **Confirm** button to save your comments and to return to the ABP test.

To edit and confirm an interpretation with the statement tree:

1. In the **Tools** menu, click **Interpretation**. The Interpretation editor is displayed.
2. From the statement tree, click on one of the categories to display all possible statements for that category.
3. From a category, click on the statement that you want to include in the interpretation. The statement is added.
4. To delete a statement from the interpretation, select the statement text and press **BACKSPACE** to delete it.
5. Click the **Confirm** button to save your comments and to return to the ABP test.

Tips for editing and confirming an interpretation:


- You can automatically insert the current date and time by clicking the **Date/time** button.
- You can clear the interpretation editor by clicking the **Clear** button.

4.2 View interpretation history

When you change something in an interpretation, the ABP module does not change the original interpretation, but creates a new one. This way, you never lose any information. A copy of all interpretations is kept in the interpretation history.

To view the interpretation history:

1. In the **Tools** menu, click **History**. The **Interpretation history** window is displayed.

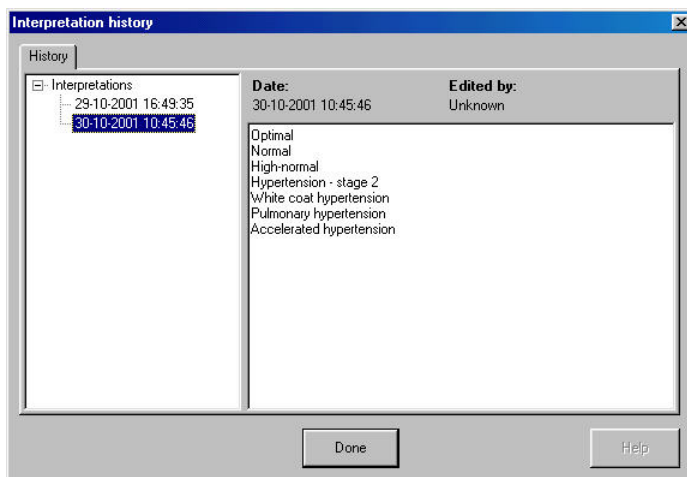

2. The left hand pane displays the interpretations sorted by date. The right hand pane shows the content of each interpretation, plus the interpretation date, time and editor.
3. Click on a date to view an interpretation.

5. Printing an ABP test

5.1 About printing ABP tests

With the ABP module, you can print ABP tests on plain paper with an regular inkjet or laser printer. You can choose from a number of formats in which you want to print the information. Each print format has a header that contains additional information. With the Print preview function, you can see what a printed ABP test will look like on paper.

Print formats

The ABP module can print the following formats:

- Trends
- Summary
- Scatter points
- Scatter points awake
- Scatter points asleep
- Rise and fall
- Measurements
- Hourly intervals
- Hourly intervals with an interval of 2, 3, 4, 6 or 8 hours
- Frequency distributions
- Frequency distributions asleep or awake
- Exceeding norms

Each format contains the view and a header that displays additional information. You can print several formats at the same time.

If among others the Summary report is selected in the default report templates in the ABP printing settings tab, the summary report will be printed first.

Please refer to the Workstation manual for further information on printing tests.

6. Customize the ABP module

6.1 About customizing

You can adjust various ABP settings to your own needs and wishes. You can, for example, change the norm values for diastolic and systolic pressure, set the dipping percentage or select the formats that you want to use with the Print selected formats function.

The features that you can customize are located in the ABP settings.

To open the ABP settings:

1. In the **File** menu, click **Settings**. The **Settings** dialog box is displayed.
2. Click **Ambulatory Blood Pressure**.

Protocols To add and edit protocols, use the Protocol editor.

6.2 ABP settings- General tab

Use the **General** tab to customize the norm values, default sleeping time and dipping definitions.

The **General** tab contains the following information:

Norm values	The norm values that are used by the ABP module to compare measurements. You can set norm values for diastolic and systolic pressure, for both sleeping and waking periods.
Default sleeping time	The period that is used as the default sleeping time. You can always manually assign measurements to sleeping and waking time.
Dipping	The percentage at which the blood pressure dips

6.3 ABP settings- Printing tab

Use the **Printing** tab to select the formats that you want to use for the **Print selected formats** function.

The **Printing** tab contains the following information:

Default report templates


Here, you can select the formats that you want to print when you use the Print selected formats function. You can select multiple formats.

Print after the measurements are read back from the device

If selected the ABP test is automatically printed when the measurements are read back from the device. The formats that are selected in the Default report templates area are printed.

6.4 ABP settings- Advanced tab

Use the **Advanced** tab to select the Recording device and the COM port for the ABP recorder.

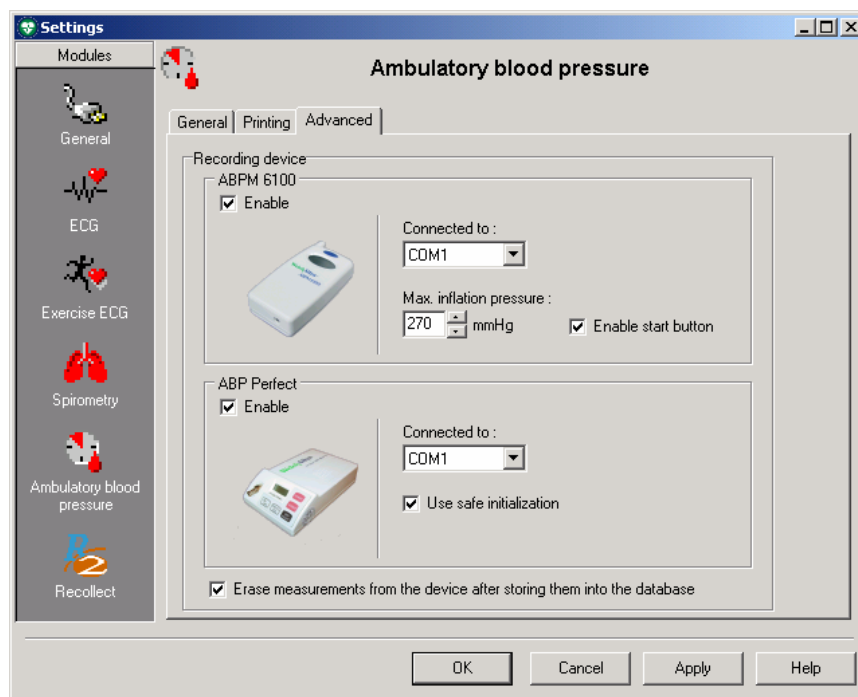

The **Advanced** tab contains the following information:

Recording device Enable the ABPM 6100, the ABP Perfect or both. If you have selected both a window will pop up when you want to make a new recording in which you can select which device to use.

Connected to From this list, you can select the port to which the ABP recorder is connected.

Use safe initialization	Use this option if the ABP Perfect recorder has problem communicating with the ABP program.
Max. inflation pressure	Allows you to select the maximum pressure that the unit will inflate the cuff to in mmHG
Enable start button	If selected a measurement can be manually started with the start button. If not, this can only be done in the first 30 minutes.
Auto erase	When you select this option, the measurements are automatically erased from the device after storing them into the database.

6.5 Protocols

A protocol is an important part of the ABP module. It contains information about when the ABP recorder must record a measurement.

A protocol contains a number of stages. Each stage contains the following information:

- **Start time:** the time the stage becomes active.
- **Time interval:** time in minutes between two measurements.
- **Number of measurements:** number of measurements that are recorded per hour during this stage. You can select 0, 1, 2, 3, 4, 5, 6, 10, 12, 15, 20 or 30 measurements per hour. If you set the number of measurements to 6, this means that a measurement will be recorded every 10 minutes.
- **Buzzer:** whether the buzzer should play a sound when a recording is made. This way, the patient can prepare himself and reduce the possibility of an error measurement. If an error measurement occurs, a repeated measurement is taken at exactly 3 minutes after the error measurement.

You can add and edit protocols in the Protocol editor.

Note:

The ABPM 6100 only supports 0, 1, 2, 3, 4, 6 or 12 measurements per hour. If the ABPM 6100 is selected and an unsupported measurement is used, a warning dialog will appear. If you select cancel a new measurement per hour can be selected. If you select ignore, the measurement will be listed however this protocol cannot be used with the ABPM 6100.

The maximum number of protocol entries is limited to 6. If either the maximum number of entries is exceeded or an unsupported entry has been inserted the ABPM 6100 refuses to upload.

6.6 Add or edit a protocol

With the protocol editor, you can add new protocols or edit existing ones. Adding a protocol consists of two parts:


- Creating the protocol
- Adding stages to the protocol.

Note:

The ABPM 6100 only supports 0, 1, 2, 3, 4, 6 or 12 measurements per hour.

To create a new protocol:

1. In the **Tools** menu, click **Protocol editor**. The **Protocol editor** is started.

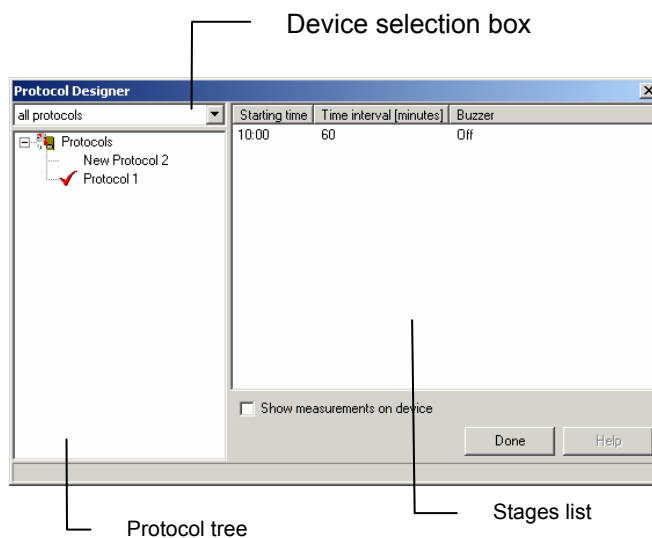

2. Select in the Device selection box, which type of protocol you want to create. Either a protocol for the ABPM 6100 or for the ABP Perfect or for both.
3. Right click on the **Protocol tree**.
4. In the shortcut menu, click **New**. A new empty protocol is added to the **Protocol tree**.
5. Enter a meaningful name for the new protocol.

To add a stage to the protocol:

1. Right click in the stages list.
2. In the shortcut menu, click **New**. The **Time interval editor** is started.
3. Enter the time at which the stage must start.
4. Enter the number of measurements that must be made per hour.
5. If you want the buzzer to play a sound before recording, select the **Buzzer** option.
6. If you want the measurement to be displayed on the ABP recorder right after it was recorded, select the **Show measurements on device** option.

To edit a stage:

1. In the stages list, double click on the stage that you want to edit. The **Time interval editor** is started.
2. Change the information

6.7 Customize abp.txt

You can customize the statements that are used in the Interpretation editor according to your own preferences.

To customize the statements:

1. Use Notepad to open the file abp.txt. As a default, this file is located in C:\program files\cardio control\mdw\modules\resources\0009 (English) or \0013 (Dutch).
2. Edit, add and delete the statements according to your own preferences.

Abp.txt consists of statements that are grouped in categories. The statements are listed under the category name they belong to. Statements indent one position.

To create a new category:

1. Type the name of the new category on the first position of a new line.
2. Enter the statements you want to add to this category on new lines, make sure that each statement indents one character.

6.8 Device overview

The device overview displays the ABP devices that are currently in use, the patients to whom they are assigned and the times when recordings have started.

To view the Device overview:

- In the Tools menu, click Device overview. The Device overview is displayed.

6.9 ABP error list

In the error list you can see the errors that can occur during the test recording as shown by the software.

The error list dialog consists of two items:

- Number of the error
- Explanation

An error can occur because of a number of reasons. Most of the reasons are listed in the explanations.

Error	Description
1	Unknown error
2	<p>This can indicate one of the following problems:</p> <ul style="list-style-type: none"> • After 2 seconds the pulse has not reached the falling level. • Heart frequency has not reached its defined range. • Variation of the heart frequency is too large. • Less than 4 measurements were taken. • The rising level was not reached within 2 seconds after the falling level. • Pulse bandwidth is out of range. • Movement artefact. • Maximum number of beats encountered in a stage.
3	<ul style="list-style-type: none"> • This can indicate one of the following problems: • The heart frequency was out of range in more than 3 pressure stages. • The difference between systole and diastole was too small. • The systole is considerably bigger than the pump pressure. • An oscillation value was smaller than the start or end value. • The pulse bandwidth is smaller than at start or end point. • The MAP is out of range. • The systole is out of range. • The diastole is too large. • The number of stages is too large. • The oscillation value is too large. • The total time is too long. • A function took too much time.
4	The battery voltage is too low.
5	No cuff was detected.
6	<p>Too many measurements were taken</p> <p>-or-</p> <p>No memory was available in EEPROM</p>
7	<p>No measurements are available.</p> <p>-or-</p> <p>The measurements were stopped.</p>
8	<p>Hardware failure</p> <p>-or-</p> <p>Error at zero alignment or pumping started unexpectedly.</p>

Note: The codes mentioned above are the codes as listed by the software. They can differ from the codes in the LCD display of the ABPM 6100. The latter ones are for reference only, the error codes used in the software module as listed above are the error codes which correctly identify the possible error status.

7. Troubleshooting

When creating a new ABP test, the computer tells me that there is no response from the hardware:

- Check if there are fresh batteries in the device
- Check if the device is connected to the COM port of the computer
- Check if the COM-port settings in the CPWS settings menu correspond with the actually used COM port

While using the ABP Perfect:

- Check if “co” is displayed on the display of the ABP Perfect (see below for further checking)
- Try switching on “Use safe initialization” in the advanced part of the ABP settings menu

The ABP Perfect unit does not display “co” after it has been connected to the computer:

- Check if there are fresh batteries in the device
- Switch the device off and on once more to make sure the device was switched on after the device has been connected to the computer

Measurement fails repeatedly:

- Make sure the chosen cuff fits the patients arm correctly
- Check if the cuff is (still) positioned correctly
- Try a measurement with the cuff on the other arm
- If the pump does not seem to work, try again with fresh batteries
- To determine whether the device works at all, try a measurement on a (test) person who has been successfully tested before.

An error code appears on the display of the ABP unit:

- See the list of possible errors in the CPWS help-file and on page 34.

8. Installing the ABP module

The Welch Allyn Cardio Perfect ABP software module can be used with the following devices:

- **Mobil-O-Graph ABP Perfect:**
OEM ABP Monitor: I.E.M. GmbH, Stolberg, Germany, Mobil-o-Graph



- **ABPM 6100:**
WA ABP monitor: Welch Allyn, Inc.; Skaneateles Falls, NY, USA



The ABP module is made up of two elements: the recorder, and the software that runs on the computer to which the recorder is connected. Before you can start recording tests, you need to:

- Connect the recorder to the computer.
- Configure the software.

Please refer to the ABPM 6100 hardware manual for instructions on how to handle and maintain the ABPM 6100 recorder.

8.1 Connect the recorder to the computer

To connect the ABP Perfect recorder to the computer:

1. Switch off the recorder.
2. Connect the interface cable to the data connector of the recorder. Each time the cable from the data connector is connected or disconnected, the recorder must be switched off.
3. Connect the serial connector of the interface cable to a free serial port or COM port on your computer.
4. Switch on the recorder.
5. The recorder is ready when its LCD display shows the word 'co'.

To connect the ABPM 6100 recorder to the computer:

1. Connect the interface cable to the data connector of the recorder. Each time the cable from the data connector is connected or disconnected, the recorder must be switched off.
2. Connect the serial connector of the interface cable to a free serial port or COM port on your computer.
3. Switch on the recorder.

Note: Please refer to the ABPM Hardware Manual section 4. If your PC has no COM PORT.

8.2 Configure Welch Allyn CardioPerfect Workstation

After connecting the recorder, you need to configure Welch Allyn CardioPerfect Workstation.

To configure Welch Allyn CardioPerfect Workstation for use with the recorder:

1. Start Welch Allyn CardioPerfect Workstation.
2. In the **File** menu, click **Settings** and click **Ambulatory blood pressure**.
3. Click the **Advanced** tab.
4. Select device.
5. Set the COM-port number to the number of the serial port to which the ABP recorder is connected.
6. Click **OK** to save the settings.

For further information on installation and configuration please refer to the Workstation Installation and Configuration manual.

9. Function keys

The Welch Allyn CardioPerfect module is, just like all Windows applications, designed for working with the mouse. However, there might be situations in which working with the keyboard can be quicker. Therefore a number of functions within the Welch Allyn CardioPerfect module can also be selected directly using the keyboard. Here is a list of all available keyboard shortcuts in this module, for a more general function key description please refer to the Workstation manual:

ABP specific functions	
Key	Function
[SHIFT]+[CTRL]+[B]	Starts a new ABP recording.
F2	Start new test
[CTRL]+[I]	Opens the Interpretation window

Index

ABP		MAP	17, 20, 23, 24
Installation	38	filtering	24
ABP recorder	39	Measurement	18, 19
ABP test	11	Delete	19
ABP window	9	Temporarily hide	18
Add comment	18	Measurements view	17
Advanced tab	32	Name new device	11
Assign measurements	14	New ABP Test dialog	12
Awake/asleep	22	Norm values	23
Buzzer	34	Change	23
Change	23	Patient information	12
norm values	23	Print formats	30
Clear button	28	Printing tab	32
COM	32	Protocol	33, 34
Comment	18	edit	34
Confirm	28	Protocol information	12
Confirm interpretation	28	Protocol list	12
Connect ABP recorder	39	Read back measurements	13, 14
Date/time button	28	Record	11, 12
Delete	19	ABP test	11, 12
measurement	19	Reset zoom	21, 25
Edit	34	Rise	21, 25
protocol	34	RRP	17
Exceeding	26	Scatter points view	23
norms view	26	Shortcut keys	40
Exclude	18	Sleeping period	17, 18
General tab	31	Store previously recorded	13
Graph	21, 25	Summary	23
Hourly intervals view	19	Summary view	22
Include	18	Trends	21, 25
Installation	38	Trends view	20
Interpretation history	29	View interpretation history	29
Interpretations	9, 28	Waking period	18
confirm	28	Zoom	21, 25