

CP 200™ 12-Lead Resting Electrocardiograph



Directions for Use

WelchAllyn®

Advancing Frontline Care™

Copyright 2005, Welch Allyn, Inc. All rights are reserved. No one is permitted to reproduce or duplicate, in any form, this manual or any part thereof without permission from Welch Allyn.

Caution: Federal US law restricts sale of the device identified in this manual to, or on the order of, a licensed physician.

Welch Allyn assumes no responsibility for any injury, or for any illegal or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or indications for use published in this manual.

Welch Allyn is a registered trademark of Welch Allyn, Inc., and CP 200 and CardioPerfect are trademarks of Welch Allyn, Inc.

SD is a trademark of Toshiba.

Software in this product is Copyright 2005, Welch Allyn, Inc., or its vendors. All rights are reserved. The software is protected by United States of America copyright laws and international treaty provisions applicable worldwide. Under such laws, the licensee is entitled to use the copy of the software incorporated within this instrument as intended in the operation of the product in which it is embedded. The software may not be copied, decompiled, reverse-engineered, disassembled or otherwise reduced to human-perceivable form. This is not a sale of the software or any copy of the software; all right, title and ownership of the software remains with Welch Allyn or its vendors.

For information about any Welch Allyn product, please call Welch Allyn Technical Support:

USA	1 800 535 6663 + 1 315 685 4560	Australia	+ 61 29 638 3000 800 074 793
Canada	1 800 561 8797	China	+ 86 216 327 9631
European Call Center	+ 353 46 906 7790	France	+ 33 15 569 5849
Germany	+ 49 747 792 7186	Japan	+ 81 33 219 0071
Latin America	+ 1 315 685 2644	Netherlands	+ 31 15 750 5000
Singapore	+ 65 6419 8100	South Africa	+ 27 11 777 7555
United Kingdom	+ 44 207 365 6780	Sweden	+ 46 85 853 6551

Reorder Number (multi-language CD): 401151

Mat. Number (manual only): 701557, Ver: D

Welch Allyn
4341 State Street Road, PO Box 220
Skaneateles Falls, NY 13153-0220

www.welchallyn.com

Printed in USA

Contents

1 - Introduction	1
About This Manual	2
Product Overview	2
Intended Use	2
Indications for Use	3
Contraindications	3
Standard Features & Benefits	3
Options	4
Accessories	5
Controls, Indicators, and Connectors	6
About the Main Menu	10
Moving Through the Menu	11
About the Patient Cable and Leads	12
Symbols	13
Using the Electrocardiograph Safely	15
General Warnings	15
General Cautions	17
Getting Help	18
2 - Setting Up the Electrocardiograph	19
Inspecting the Electrocardiograph	20
Connecting the Patient Cable	20
Loading the Thermal Chart Paper	21
Powering the Electrocardiograph	22
Verifying Proper Operation	23
3 - Reviewing the System Settings	25
"System Settings" Menu Tree	26
Reviewing the Device Configuration Settings	27
Reviewing the Device Information	29
Reviewing the Medication List	30
Reviewing the History List	31
4 - Reviewing the ECG Settings	33
"ECG Settings" Menu Tree	34
Reviewing the Auto Report Settings	35
Reviewing the Format Settings for Auto Reports	37
Reviewing the Interpretation and Copy Settings for Auto Reports	39
Reviewing the Patient Data Fields Available for Auto Reports	40

Reviewing the Rhythm Report Settings	42
Reviewing the Miscellaneous ECG Settings	43
5 - Performing ECG Tests	45
Connecting the Leads to the Patient	46
Recording an Auto ECG	49
Recording a Normal Auto ECG	50
Recording a Stat Auto ECG	55
Recording a Rhythm ECG	56
Searching for Saved Patient Data	57
Adjusting the ECG Waveforms	61
6 - Performing Administrative Tasks	63
Managing Saved Tests	64
Managing the Scheduled Patients List	68
Managing Data Security	69
Working With the User List and User Login	70
Working With the Audit Trail	72
7 - Maintaining the Electrocardiograph	73
Inspecting the Equipment	74
Cleaning the Equipment	74
Testing the Equipment	75
Recharging a Fully Discharged Battery	76
Replacing the Battery	77
Replacing the Battery (DC) Fuse	79
Replacing the AC Fuses	80
Storing the Equipment	81
Discarding the Equipment	81
8 - Troubleshooting	83
Problem-Solving Suggestions	84
Limited Warranty	87
Service Policy	88
A - Specifications	89
B - EMC Guidance and Manufacturer's Declarations	91
Glossary	95
Index	97

1

Introduction

About This Manual	2
Product Overview	2
Intended Use	2
Indications for Use	3
Contraindications	3
Standard Features & Benefits	3
Options	4
Accessories	5
Controls, Indicators, and Connectors	6
About the Main Menu	10
Moving Through the Menus	11
About the Patient Cable and Leads	12
Symbols	13
Using the Electrocardiograph Safely	15
Getting Help	18

About This Manual

This manual is written for clinical professionals with a working knowledge of medical procedures and terminology as required for monitoring cardiac patients.

Before using the CP 200 electrocardiograph for clinical applications—or before setting up, configuring, troubleshooting, or servicing the electrocardiograph—you must read and understand this manual and all other information accompanying the electrocardiograph and related options or accessories.

Product Overview

The Welch Allyn CP 200 electrocardiograph can display, print, save, and send ECGs electronically. It features a full alphanumeric keyboard, a color display to preview ECGs and edit settings, storage for up to 50 ECG and 50 spirometry records, full-size user-programmable reports, and the ability to operate on either battery or AC power.

For centralized ECG data storage, the CP 200 electrocardiograph can connect to a Welch Allyn CardioPerfect™ workstation, which in turn can connect with other electronic patient-information systems, such as billing and medical records.

For details, see the following sections:

- [“Standard Features & Benefits”](#) on page 3
- [“Options”](#) on page 4
- [“Specifications”](#) on page 89

Intended Use

The CP 200 electrocardiograph is specifically intended for acquiring, viewing, storing, and printing ECG signals from adult and pediatric patients. It will be used in clinical settings by trained healthcare providers.

The optional interpretation algorithm analyzes these ECG signals to generate measurements and interpretive statements for adults. The interpretive results are intended only as guidance for qualified physicians and must not be relied upon as diagnoses.

The electrocardiograph provides an optional interface to a pulmonary function device.

Communication of ECG and spirometry data with a central data-management system is optional.

Indications for Use

The electrocardiograph is one of the tools that clinicians use to evaluate, diagnose, and monitor patient cardiac function.

The 12-lead interpretive algorithm provides a computer-generated analysis of potential patient abnormalities which must be used only as guidance for consideration by qualified physicians, together with other relevant clinical information, for purposes of making diagnoses.

Contraindications

The 12-lead interpretive algorithm is not intended for use on pediatric patients.

Standard Features & Benefits

Full alphanumeric keypad

Enter or search for patient information quickly and easily.

Color LCD display

View and adjust the ECG waveforms prior to printing, to save time and paper. With spirometry option, also view patients' spirometry efforts and results.

Storage for up to 50 ECG and 50 spirometry records

Review, edit, print, or save recent records.

Unlimited storage on SD™ memory cards

Use SD memory cards to save as many ECG or spirometry records as you like. (Cards are not included.)

Battery operation

Use the electrocardiograph almost anywhere. On battery power, you can print up to 100 ECGs continuously before needing to recharge.

User-definable ECG report formats

Customize one or two formats for efficient reporting.

Removable leads for ECG patient cable

Replace leads individually if needed.

Compatibility with CardioPerfect workstation software

Store and manage data electronically by transferring records to a Welch Allyn CardioPerfect workstation in one of two ways:

- via an SD memory card (card not included)
- via a hardwired connection (cable not included)

Options

These options are available both for initial purchases and for upgrades.

- **Spirometry**

With a disposable, single-use flow transducer, the optional spirometer performs FVC and SVC tests.

- **Automatic ECG interpretation**

The optional MEANS interpretation algorithm, developed by the University of Rotterdam in the Netherlands, provides automatic analysis of ECG tests. For more information, see the *MEANS Physicians' Manual* on the CD that came with your electrocardiograph.

- **Carts**

Two specially designed carts are available for convenient transport and use of the electrocardiograph, as shown here with the optional cable arm and shelf.

Figure 1. Office Cart


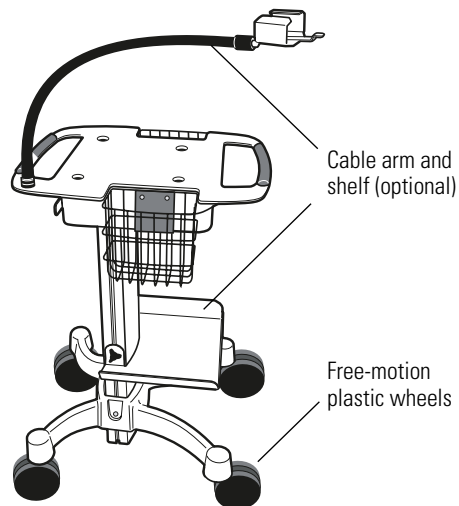


Figure 2. Hospital Cart

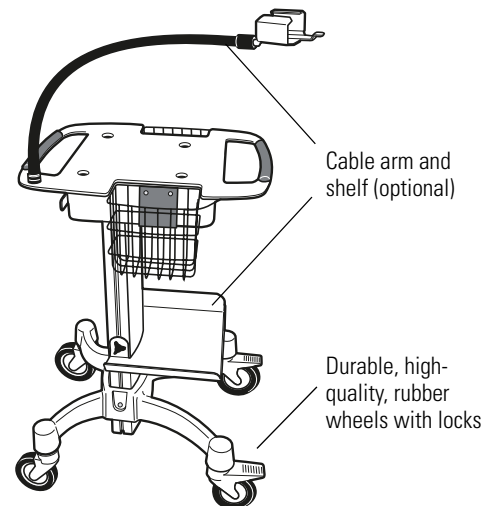

Accessories

To order accessories, call Welch Allyn. For phone numbers, see [page ii](#).

Item	Customer Order Number	Quantity
Resting tab electrodes	45008-0000	1000
Resting tab electrode adaptors	58581-0000	10
Thermal chart paper (1 case = 5 pads, 200 sheets each)	94018-0000	1 case
Welch cups	RE-ELEC-CUP	6
Limb lead clamps, IEC	RE-ELEC-CLP	4
Limb lead clamps, AHA	401432	4
<hr/>		
Patient cable (Figure 12 on page 12)		
• AHA	400293	1
• IEC	400294	1
• IEC, vacuum adapter	401128	1
<hr/>		
Lead wires (10 wires per set)		
• AHA banana	401129	1 set
• IEC banana	401122	1 set
• AHA pinch	401123	1 set
• IEC pinch	401124	1 set
<hr/>		
Battery (Figure 47 on page 77)	100660	1
Dust cover	401428	1
<hr/>		
Carts		
• Utility cart	08265-0000	1
• Office cart (Figure 1 on page 4)	401393	1
• Hospital cart (Figure 2 on page 4)	401394	1
• Cable arm & shelf option (page 4)	401161	1
<hr/>		
Connectivity kit to the CardioPerfect workstation	100638	1
Interpretation upgrade option	100623	1
<hr/>		
Spirometry option	100400	1
<hr/>		
Product information		
• Electrode placement wall poster	71300-0000	1
• <i>CP 200 12-Lead Resting Electrocardiograph Directions for Use</i>	701557	1
• CP 200 product information multi-language CD	401151	1

Controls, Indicators, and Connectors

This section describes the controls, indicators, and connectors that are part of the electrocardiograph.

Figure 3. Top

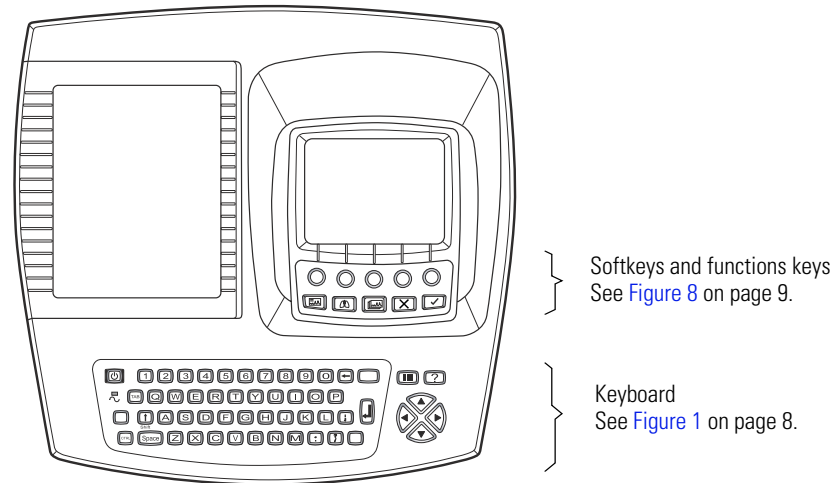


Figure 4. Back

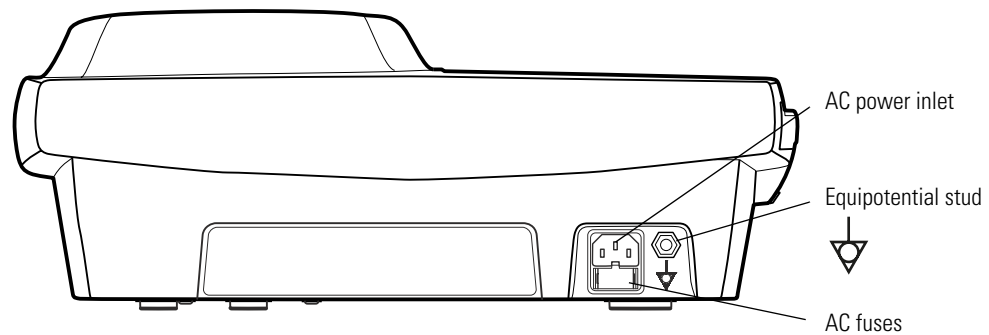


Figure 5. Right Side

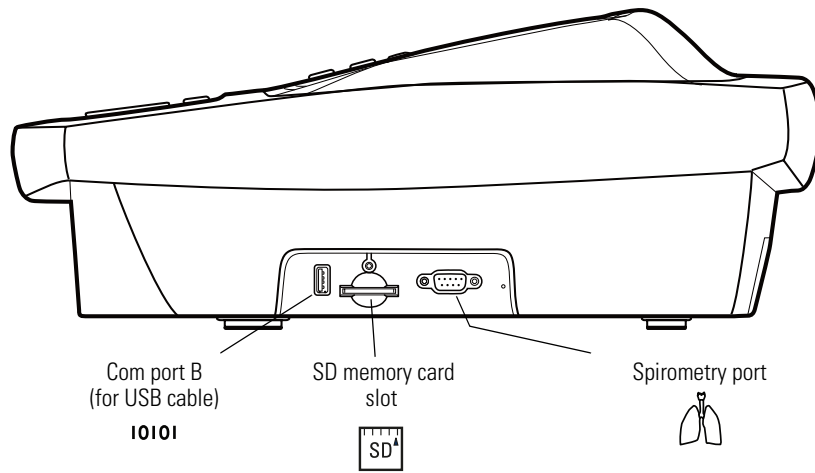


Figure 6. Front

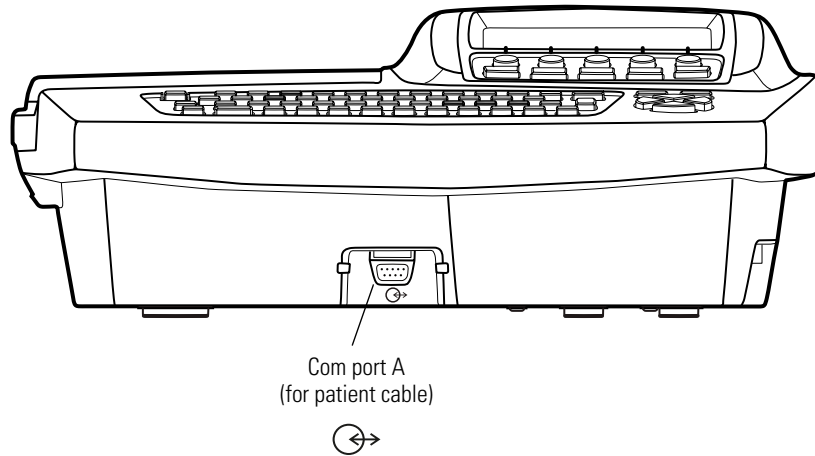


Figure 7. Left Side

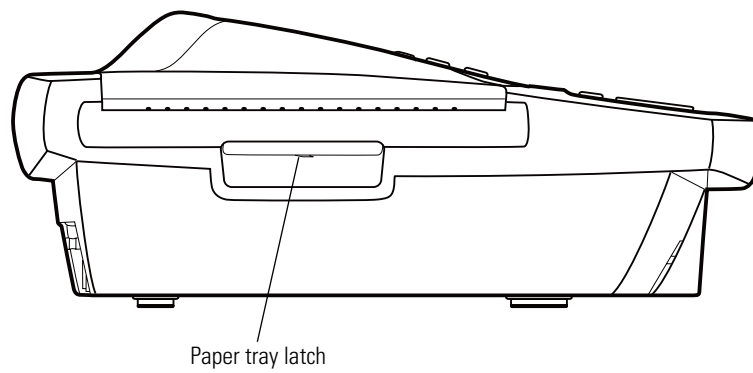
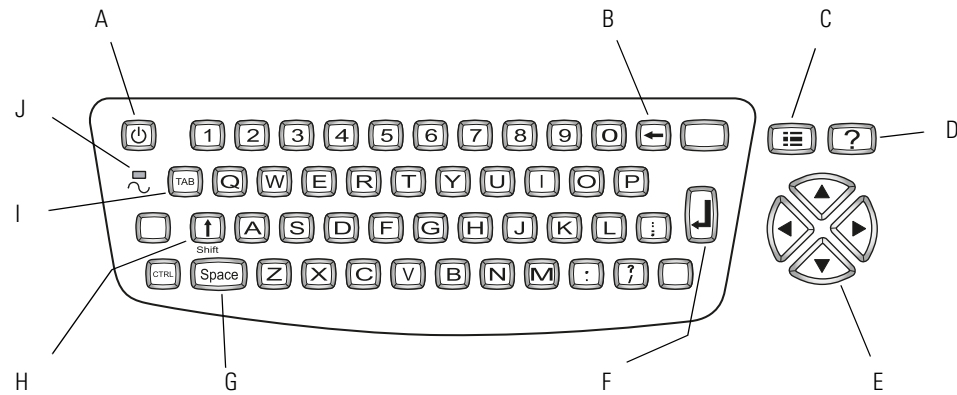
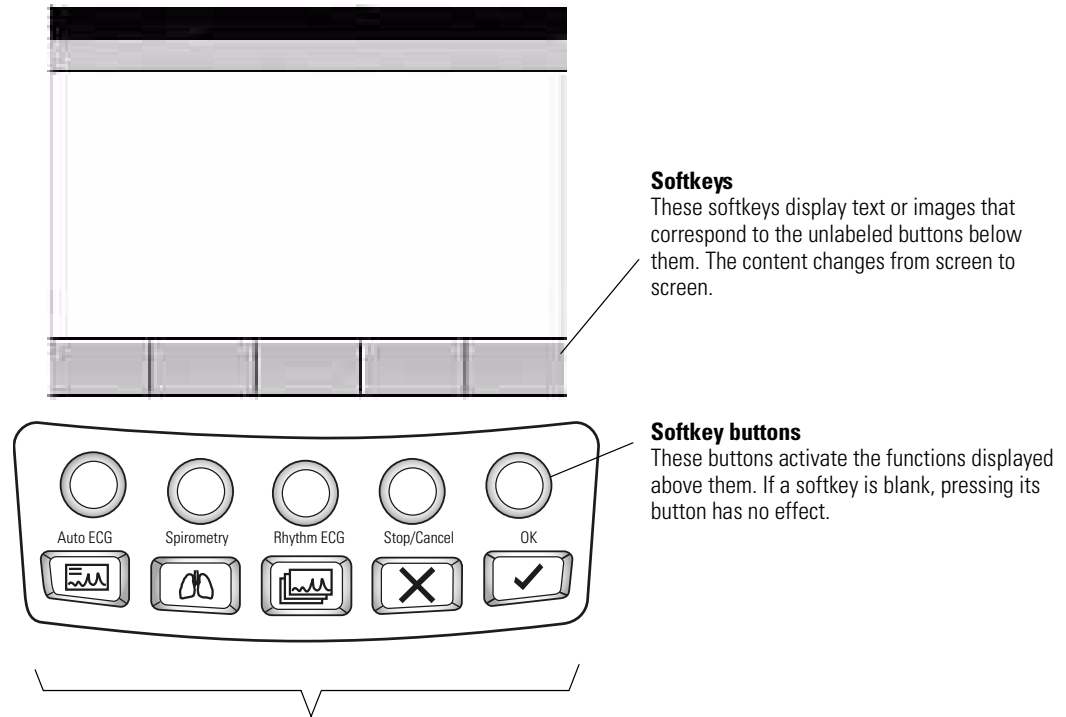


Table 1. Keyboard

Key	Function
A. On/Off	See "Powering the Electrocardiograph" on page 22.
B. Backspace	Deletes the character to the left of the cursor.
C. Menu	See "About the Main Menu" on page 10.
D. Help	See "Getting Help" on page 18.
E. Navigation arrows	See "Moving Through the Menus" on page 11.
F. Enter	See "Moving Through the Menus" on page 11.
G. Space	Enters a space.
H. Shift	Capitalizes letters.
I. Tab	Moves through the data-entry fields.
J. Green LED	Lights up when the electrocardiograph is connected to AC power.

Figure 8. Softkeys and Function Keys**Softkeys**

These softkeys display text or images that correspond to the unlabeled buttons below them. The content changes from screen to screen.

Softkey buttons

These buttons activate the functions displayed above them. If a softkey is blank, pressing its button has no effect.

Function Keys**Auto ECG**

Begins Auto ECGs, normal and stat.
See ["Recording an Auto ECG"](#) on page 49.

**Spirometry**

Begins spirometry tests.
See spirometry manual.

**Rhythm ECG**

Begins Rhythm ECGs.
See ["Recording a Rhythm ECG"](#) on page 56.

**Stop/Cancel**

Stops any current activity.
See ["Moving Through the Menus"](#) on page 11.

**OK**

Accepts data that you have entered, or chooses a highlighted item.
See ["Moving Through the Menus"](#) on page 11.

About the Main Menu


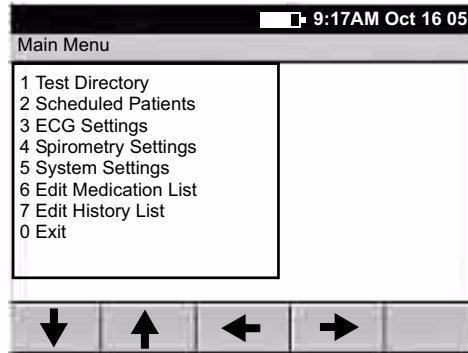
The main menu appears when you press the Menu key .

Figure 9. Main Menu



Submenu	Purpose	Procedure
Test Directory	View, change, print, or send saved tests.	See “Managing Saved Tests” on page 64.
Scheduled Patients	View the scheduled patients list, add patients to the list, or delete patients from the list.	See “Managing the Scheduled Patients List” on page 68.
ECG Settings	Review or change ECG settings: Auto Report format, Rhythm Report format, and so on.	See “Reviewing the ECG Settings” on page 33.
Spirometry Settings	Review or change spirometry settings: display settings, print settings, and so on.	See spirometry manual.
System Settings	Review or change system settings: device configuration, device info, user setup, and so on.	See “Reviewing the System Settings” on page 25.
Edit Medication List	Edit the list of medication choices available to choose during patient data entry.	See “Reviewing the Medication List” on page 30.
Edit History List	Edit the list of clinical conditions available to choose during patient data entry.	See “Reviewing the History List” on page 31.

Moving Through the Menus

Figure 10. Standard Menu


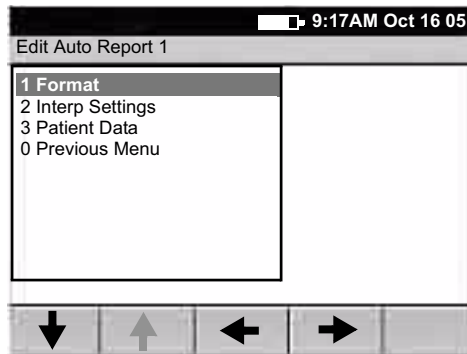


Figure 11. Parent Menu With Submenu

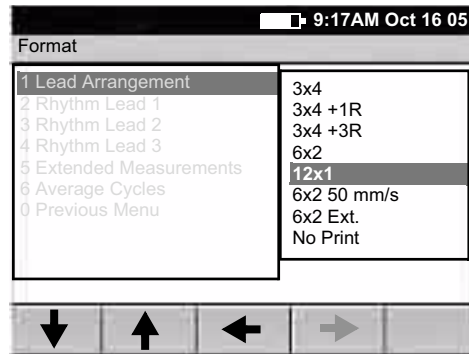


Desired Actions	Keys to Press
To move up or down a list	▲ or ▼ (keyboard or softkey arrows)
To open a standard menu (Figure 10)	or or ► or item's number or letter
To move from parent menu to submenu on same screen (Figure 11)	or
To perform an action To accept data To check or uncheck a checkbox	or
To return to parent menu from submenu on same screen (Figure 11)	or (To select the highlighted submenu item.) or ◀ (To make no change.)
To move back through the menus	◀ or zero key
To move through data-entry fields	
To return to the ECG Preview screen from a standard menu (Figure 10)	

Note Keyboard and softkey arrows work the same way.

About the Patient Cable and Leads

The patient cable processes the patient's ECG data and transmits it to the electrocardiograph. To make handling convenient, the ten leads are arranged to point toward the appropriate parts of the body. The cable rake, which slides easily, prevents the chest leads from tangling.

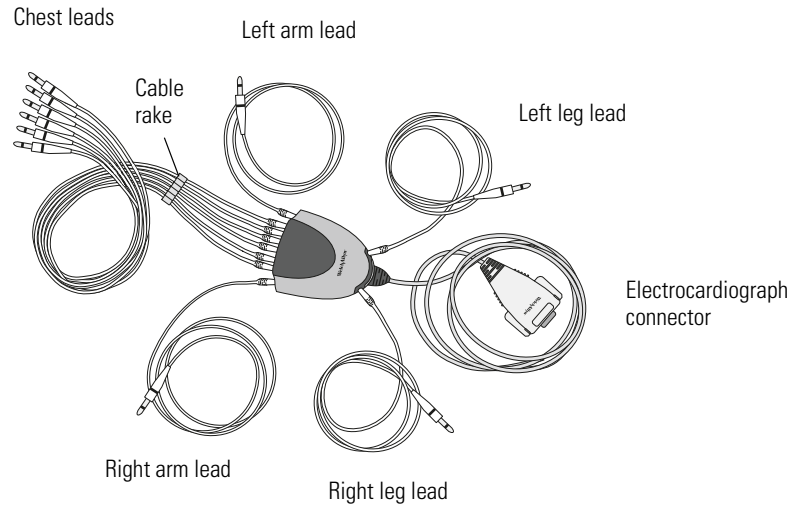


Figure 12. Patient Cable and Leads









Symbols

The symbols illustrated on the following pages may appear on the electrocardiograph, on the packaging, on the shipping container, or in this manual.



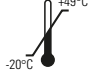

Documentation Symbols

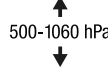-  **WARNING** Indicates conditions or practices that could lead to illness, injury, or death.
 -  **Caution** In this manual, indicates conditions or practices that could damage the equipment or other property.
 -  **Caution** On the product, means “Consult accompanying documentation.”
-

Shipping, Storing, and Environment Symbols




















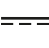



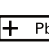


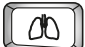

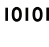- | | |
|---|--|
| <ul style="list-style-type: none">  This end up  Fragile  Temperature limits | <ul style="list-style-type: none">  Keep dry  Relative humidity limit  Altitude limits |
|---|--|
-

Certification Symbols

- | | |
|---|--|
| <p> Meets essential requirements of European Medical Device Directive 93/42/EEC</p> | <p> Complies with applicable U.S. and Canadian medical safety standards</p> |
| <p> European Regulatory Manager
Welch Allyn LTD.
Navan Business Park
Dublin Road
Navan, County Meath, Republic of Ireland
Tel.: 353-46-90-67700
Fax: 353-46-90-67756</p> | <p> Australian registered importer</p> |
-

Operation Symbols

 For details on the keys, see [Figure 1](#) on page 8.

	On/standby (off)		Help
	Navigation arrows		Backspace
	Enter		Shift
	Auto ECG		Menu
	Rhythm ECG		Stop/Cancel
	Spirometry		OK
	Spirometry port		Com port B (for USB cable)
	SD memory card slot		Com port A (for patient cable)
	Alternating current		Battery charge level
	Dangerous voltage		Direct current
	AC fuse replacement information		Ground equipotential
	Do not reuse.		Sealed lead-acid battery
	Do not dispose of this product as unsorted municipal waste. Prepare this product for reuse or separate collection as specified by Directive 2002/96/EC of the European Parliament and the Council of the European Union on Waste Electronic and Electrical Equipment (WEEE). If this product is contaminated, this directive does not apply. For more specific disposal information, see www.welchallyn.com/weee , or contact Welch Allyn Customer Service at +44 207 365 6780.		Recycle.
			Defibrillation-proof Type CF applied parts. (While the electrocardiograph is safety-rated "CF" for direct cardiac contact, it is not intended to be connected directly to the patient's heart. Only surface contact with the patient's skin is intended.)
			Type BF applied part

Using the Electrocardiograph Safely

Before using or servicing the electrocardiograph, you must read and understand the following safety-related information.

General Warnings

The following warning statements apply to electrocardiograph use in general. Warning statements that apply specifically to particular procedures, such as connecting the patient cable or performing an ECG test, appear in the corresponding sections of the manual.

Warning statements indicate conditions or practices that could lead to illness, injury, or death.



Warnings Related to the Environment

WARNING To ensure patient and device safety, leave 5 feet (1.5 meters) of open area around the patient.

WARNING To avoid a possible explosion, do not use the electrocardiograph in the presence of flammable anesthetics: mixtures containing air, oxygen, or nitrous oxide.

WARNING When transporting the electrocardiograph on a cart, tuck the patient cable away from the wheels so that it does not present a hazard.

Warnings Related to Accessories and Other Equipment

WARNING For operator and patient safety, peripheral equipment and accessories that can come in direct patient contact must be in compliance with all appropriate safety, EMC, and regulatory requirements. See [“EMC Guidance and Manufacturer’s Declarations”](#) on page 91.

WARNING All signal input and output (I/O) connectors are intended for connection of only devices complying with IEC 60601-1, or other IEC standards (for example, IEC 60950), as appropriate to the device. Connecting additional devices to the electrocardiograph may increase chassis or patient leakage currents. To maintain operator and patient safety, consider the requirements of IEC 60601-1-1. Measure the leakage currents to confirm that no electric shock hazard exists.

WARNING The electrocardiograph has not been designed for use with high-frequency (HF) surgical equipment and does not protect against hazards to the patient.



Warnings Related to Using the Electrocardiograph

WARNING This device captures and presents data reflecting a patient's physiological condition. When reviewed by a trained physician or clinician, this data can be useful in determining a diagnosis. However, the data should not be used as a sole means for determining a patient's diagnosis.

WARNING To avoid serious injury or death, take these precautions during patient defibrillation:

- Avoid contact with the electrocardiograph, patient cable, and patient.
- Verify that the patient leads are properly connected. See "[Connecting the Patient Cable](#)" on page 20.
- Place defibrillator paddles properly in relation to electrodes.
- After defibrillation, pull each patient lead out of the patient cable and inspect the tips for charring (black carbon marks). If there is any charring, the patient cable and individual leads must be replaced. If there is no charring, fully reinsert the leads into the patient cable. (Charring can occur only if a lead is not fully inserted into the patient cable before defibrillation.)

WARNING To prevent the spread of infection, take these precautions:

- Dispose of single-use components (for example, electrodes) after using them once.
- Regularly clean and disinfect all components that come in contact with patients. See "[Cleaning the Equipment](#)" on page 74.
- Avoid ECG testing for patients with open, infectious sores.

WARNING Avoid positioning any leads or cables so that they could easily trip someone or become wrapped around a patient's neck.

WARNING Satisfactory maintenance procedures must be implemented, or equipment failure and health hazards may result.

WARNING Only qualified service personnel should attempt to repair the electrocardiograph. In case of a malfunction, call Technical Support and precisely describe the problem. For phone numbers, see [page ii](#).

General Cautions

The following caution statements apply to electrocardiograph use in general. Caution statements that apply specifically to particular procedures, such as connecting the patient cable or performing an ECG test, appear in the corresponding sections of the manual.

Caution statements indicate conditions or practices that could damage the equipment or other property.



Caution When removing the electrocardiograph from storage, allow it to thermally stabilize to surrounding environmental conditions before using it.

Caution To prevent possible damage to the keypad, do not use sharp or hard objects to press keys. Only use fingertips.

Caution Do not expose the patient cable to strong ultra-violet radiation.

Caution Do not pull or stretch the patient cable. Doing so could result in mechanical or electrical failures. Form the patient cable into a loose loop before storing.

Caution Avoid positioning the patient cable where it might get pinched or stepped on. If the cable's impedance is altered, measurements might no longer be accurate, and repair might be necessary.

Caution Using the equipotential terminal for anything but grounding purposes may contribute to damage of the device.

Caution Use only parts and accessories supplied with the device and available through Welch Allyn. The use of accessories other than those specified may result in degraded performance of this device.

Caution Portable and mobile RF communications equipment can affect the performance of the electrocardiograph.


Caution The electrocardiograph meets the Class A requirements of IEC 60601-1-2:2000 regarding incidental emission of radio frequency interference. As such it is suitable for use in commercial grade electrical environments. If the electrocardiograph is used in residential grade electrical environments and you experience incidental interference with other equipment that uses radio frequency signals to operate, minimize the interference as described under ["EMC Guidance and Manufacturer's Declarations"](#) on page 91.

Caution Other medical equipment—including but not limited to defibrillators, ultrasound machines, pacemakers, and other stimulators—may be used simultaneously with the electrocardiograph. However, such devices may disturb the electrocardiograph signal.

Caution The power cord must be disconnected from AC power before cleaning, maintaining, or servicing.

Getting Help

You can get help with the electrocardiograph in a variety of ways beyond this manual.

- Press the Help key  from the **ECG Preview** screen or **Lead Off** screen for a list of topics available to print.
- Review the other information that came with the electrocardiograph. For list, see [“Product information”](#) on page 5.
- Contact Welch Allyn. For phone numbers, see [page ii](#).

2

Setting Up the Electrocardiograph

Inspecting the Electrocardiograph	20
Connecting the Patient Cable	20
Loading the Thermal Chart Paper	21
Powering the Electrocardiograph	22
Verifying Proper Operation	23

Inspecting the Electrocardiograph

1. Look for obvious signs of shipping damage. If you find any damage, contact Technical Support. For phone numbers, see [page ii](#).
2. Verify that you have received all appropriate options and accessories. See “Options” on page 4 and “Accessories” on page 5.

Connecting the Patient Cable



WARNING Conductive parts of the patient cable, electrodes and associated connections of defibrillation-proof Type CF applied parts, including the neutral conductor of the patient cable and electrode, should not come into contact with other conductive parts, including earth ground.

WARNING To avoid injury to the patient or damage to the device, never plug patient leads into any other device or wall outlet.

1. Insert all leads into their proper positions, as labeled on the connectors.

Insert connectors fully so that no part of the metal ring remains exposed. For example, see [Figure 13](#). (To see the whole patient cable with all leads inserted, see [Figure 12](#) on page 12.)



WARNING Failure to insert all connectors fully may result in a loss of energy being delivered to the patient during defibrillation and damage to the patient cable itself. For other warnings related to defibrillation, see [page 16](#).

2. Plug the patient cable into the port on the front of the electrocardiograph. See [Figure 14](#).

Figure 13. Inserting the Leads


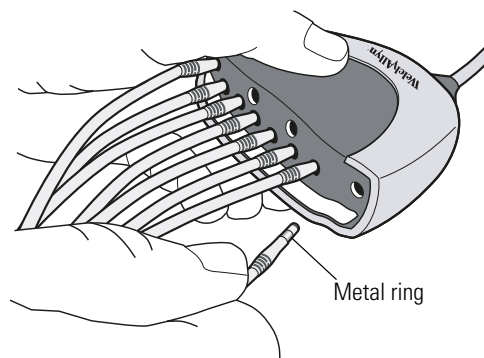


Figure 14. Plugging in the Connector

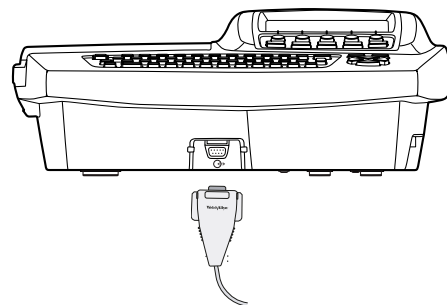

Loading the Thermal Chart Paper

1. Squeeze the latch. Pull the paper door to the left. See [Figure 15](#).
If any paper remains in the tray, remove it.
2. Remove the outer packaging, including the cardboard bottom, from a new pack of paper. Pull the top sheet back so that the paper's grid side faces up and the Welch Allyn name is on the bottom of the paper.
3. Slide the paper into the tray. See [Figure 16](#).
If humidity is high, remove up to 10 sheets so that the paper fits properly.
4. Lay the top sheet over the paper door. Push the door to the right until it clicks. See [Figure 17](#).

Figure 15. Opening the Paper Door

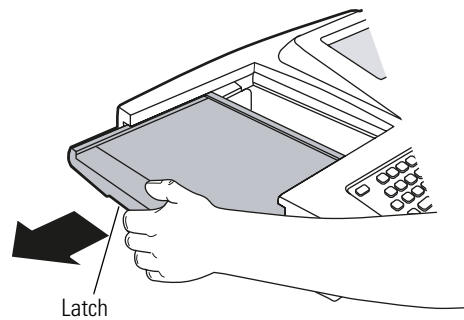


Figure 16. Loading the Paper


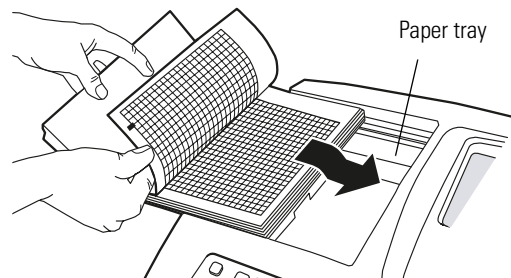


Figure 17. Closing the Paper Door

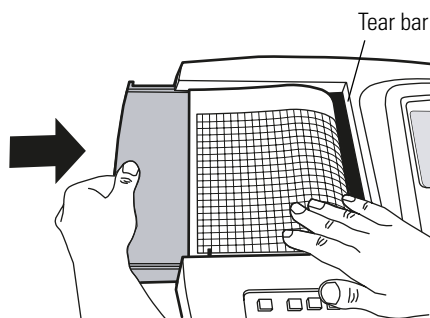

Tips for handling thermal paper:

- Store in a cool, dry, dark place.
- Avoid exposure to bright light or UV sources.
- Avoid exposure to solvents, adhesives, or cleaning fluids.
- Do not store with vinyls, plastics, or shrink wraps.

Powering the Electrocardiograph

The electrocardiograph can run on AC or battery power.



WARNING To ensure that electrical safety is maintained when using AC power, the device must be plugged into a hospital-grade outlet.

WARNING Where the integrity of external protective earth conductor arrangement is in doubt, use battery power.




Caution Medical electrical equipment needs special precautions regarding EMC and must be installed and used according to the information provided in [“EMC Guidance and Manufacturer’s Declarations”](#) on page 91.

To Connect to AC Power


Plug one end of the power cord into the electrocardiograph’s AC power inlet. Plug the other end into an AC outlet. The green LED on the keyboard lights up, indicating that power is connected. See [Figure 18](#).

To Keep the Battery Charged

Leave the electrocardiograph connected to AC power whenever possible. Battery charge status is indicated on the screen by an icon: . When the charge gets low, the icon flashes. When the charge gets too low to operate, a warning message appears and the electrocardiograph beeps every 15 seconds for 1 minute, then it turns off.

For more, see [“Recharging a Fully Discharged Battery”](#) on page 76.

Figure 18. AC Power Inlet and Green LED

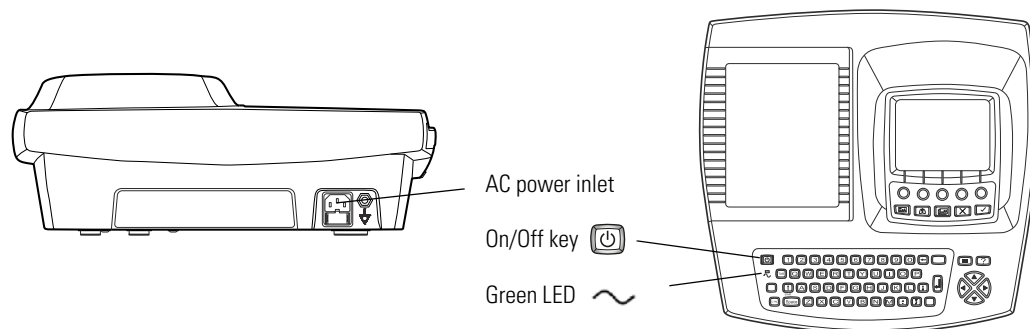

To Turn the Electrocardiograph On

Press .

To Turn the Electrocardiograph Off

Press  and hold.

Note If Power-Save is enabled, the electrocardiograph turns off automatically after several idle minutes. To learn how to enable or disable Power-Save, see [“Reviewing the Device Configuration Settings”](#) on page 27.

Verifying Proper Operation

Once your electrocardiograph is set up, verify proper operation by using an ECG simulator to acquire and print a standard 12-lead ECG of known amplitude. See [Step 2](#) on page 75.

Note As part of your initial set-up, you may want to adjust the display contrast. To learn how, see [“Reviewing the Device Configuration Settings”](#) on page 27.

You may also want to change other software settings, as described in the following chapters:

- [“Reviewing the System Settings”](#) on page 25
- [“Reviewing the ECG Settings”](#) on page 33

3


Reviewing the System Settings

“System Settings” Menu Tree	26
Reviewing the Device Configuration Settings	27
Reviewing the Device Information	29
Reviewing the Medication List	30
Reviewing the History List	31

This chapter documents the system settings, which affect both ECG and spirometry functions. For information on the following related tasks, see the procedures identified here:

- **Reviewing ECG settings**
See [“Reviewing the ECG Settings”](#) on page 33.
- **Reviewing spirometry settings**
Spirometry manual.
- **Printing all settings**
See [“Reviewing the Device Information”](#) on page 29.

“System Settings” Menu Tree

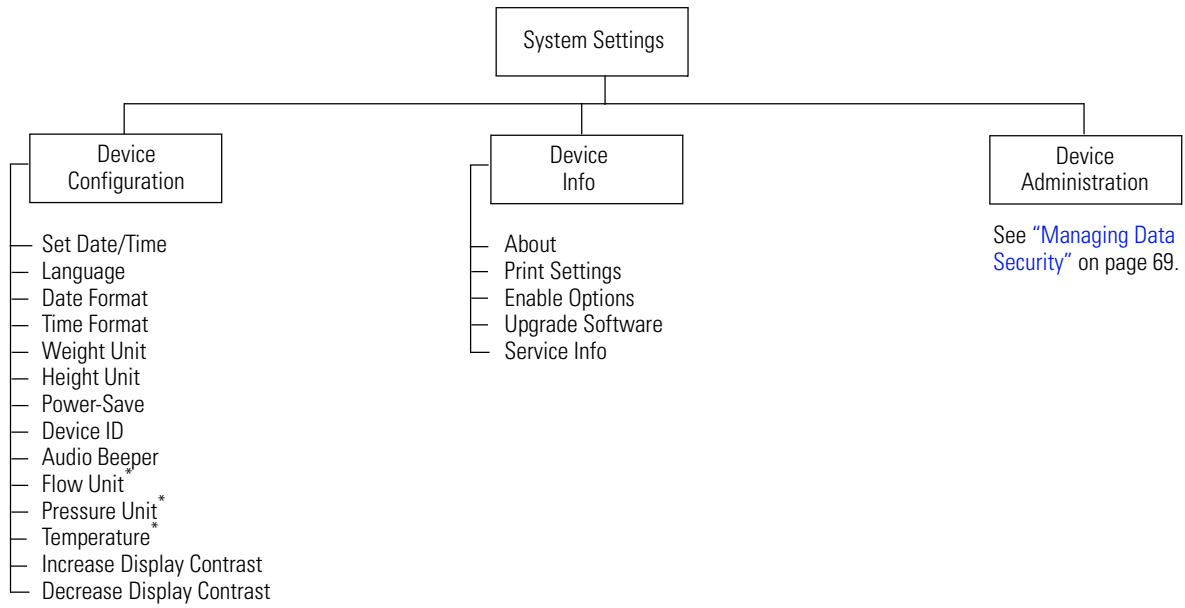


* Applicable for spirometry only.

Reviewing the Device Configuration Settings

1. Press the Menu key .
2. Choose **System Settings > Device Configuration**.

The following screen appears.

Figure 19. “Device Configuration” Screen

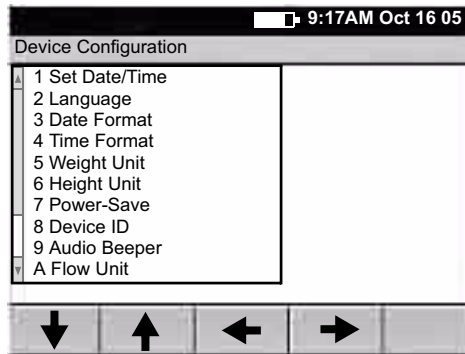

3. Change any desired settings.

Setting	Description
Set Date/Time	Current date and time.
Language	List of languages available. Changes take effect when the next screen appears.
Date Format	MM/DD/YY (month/day/year) DD/MM/YY (day/month/year)
Time Format	24-hour or AM/PM.
Weight Unit	Kilograms (kg) or pounds (lb).
Height Unit	Centimeters (cm), inches (in), or feet and inches (ft, in).
Power-Save	On or off. When on, the electrocardiograph turns itself off after several idle minutes.
Device ID	Electrocardiograph identification. Enter up to 20 characters.
Audio Beeper	On or off. When on, beeps to indicate errors, such as incorrect input, improper external connections, or a printer error. Beeps may also indicate a low battery.
Flow Unit	L/sec or L/min. For spirometry only. Determines the y-axis units for flow/volume curves.
Pressure Unit	mmHg, mbar, inHg, kPa. For spirometry only. Determines the units for the calibration menu's atmospheric pressure values.
Temperature	Fahrenheit or Celsius. For spirometry only. Determines the units for the calibration menu's temperature values.


Setting (Continued)	Description (Continued)
Increase Display Contrast	Each time you select this choice, the display contrast immediately increases until you reach maximum contrast.
Decrease Display Contrast	Each time you select this choice, the display contrast immediately decreases until you reach minimum contrast.

Reviewing the Device Information

1. Press the Menu key .
2. Choose **System Settings** > **Device Info**.

The following screen appears.

Figure 20. "Device Info" Screen

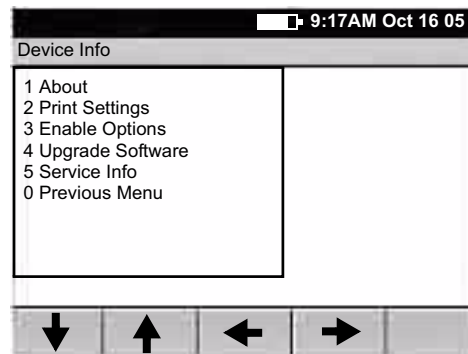


3. Select the desired item:

Item	Description
About	Displays the following information about the electrocardiograph: <ul style="list-style-type: none"> • serial number • modules configured • version numbers
Print Settings	Prints your ECG, spirometry, and system settings as well as medication & history lists.
Enable Options	Contact Technical Support. For phone numbers, see page ii .
Upgrade Software	Contact Technical Support. For phone numbers, see page ii .
Service Info	Accessible to service support only.


Reviewing the Medication List

The medication list determines which medications are available to choose during patient data entry.

1. Press the Menu key .
2. Choose **Edit Medication List**.

The following screen appears.


Figure 21. “Edit Medication List” Screen

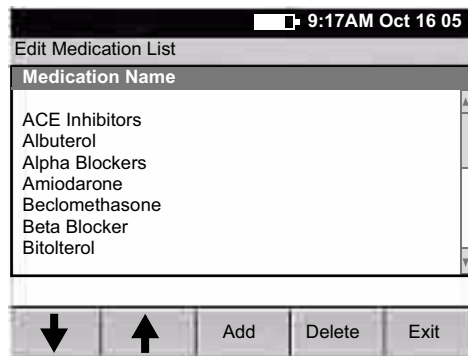

3. Press the desired softkeys:
 - **Add**
Lets you add medications, up to a total of 40.
 - **Delete**
Deletes the highlighted medication.
 - **Exit**
Returns to the main menu.


Reviewing the History List

The history list determines which clinical conditions are available to choose during patient data entry.

1. Press the Menu key .
2. Choose **Edit History List**.

The following screen appears.

Figure 22. "Edit History List" Screen

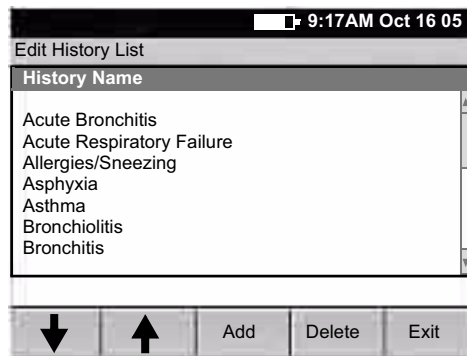

3. Press the desired softkeys:
 - **Add**
Lets you add conditions, up to a total of 40.
 - **Delete**
Deletes the highlighted condition.
 - **Exit**
Returns to the main menu.

4


Reviewing the ECG Settings

“ECG Settings” Menu Tree	34
Reviewing the Auto Report Settings	35
Reviewing the Rhythm Report Settings	42
Reviewing the Miscellaneous ECG Settings	43

This chapter documents the ECG settings. For information on the following related tasks, see the procedures identified here:

- **Reviewing system settings** (which affect both ECG and spirometry functions)
See [“Reviewing the System Settings”](#) on page 25.
- **Reviewing spirometry settings**
Spirometry manual.
- **Printing all settings**
See [“Reviewing the Device Information”](#) on page 29.

“ECG Settings” Menu Tree

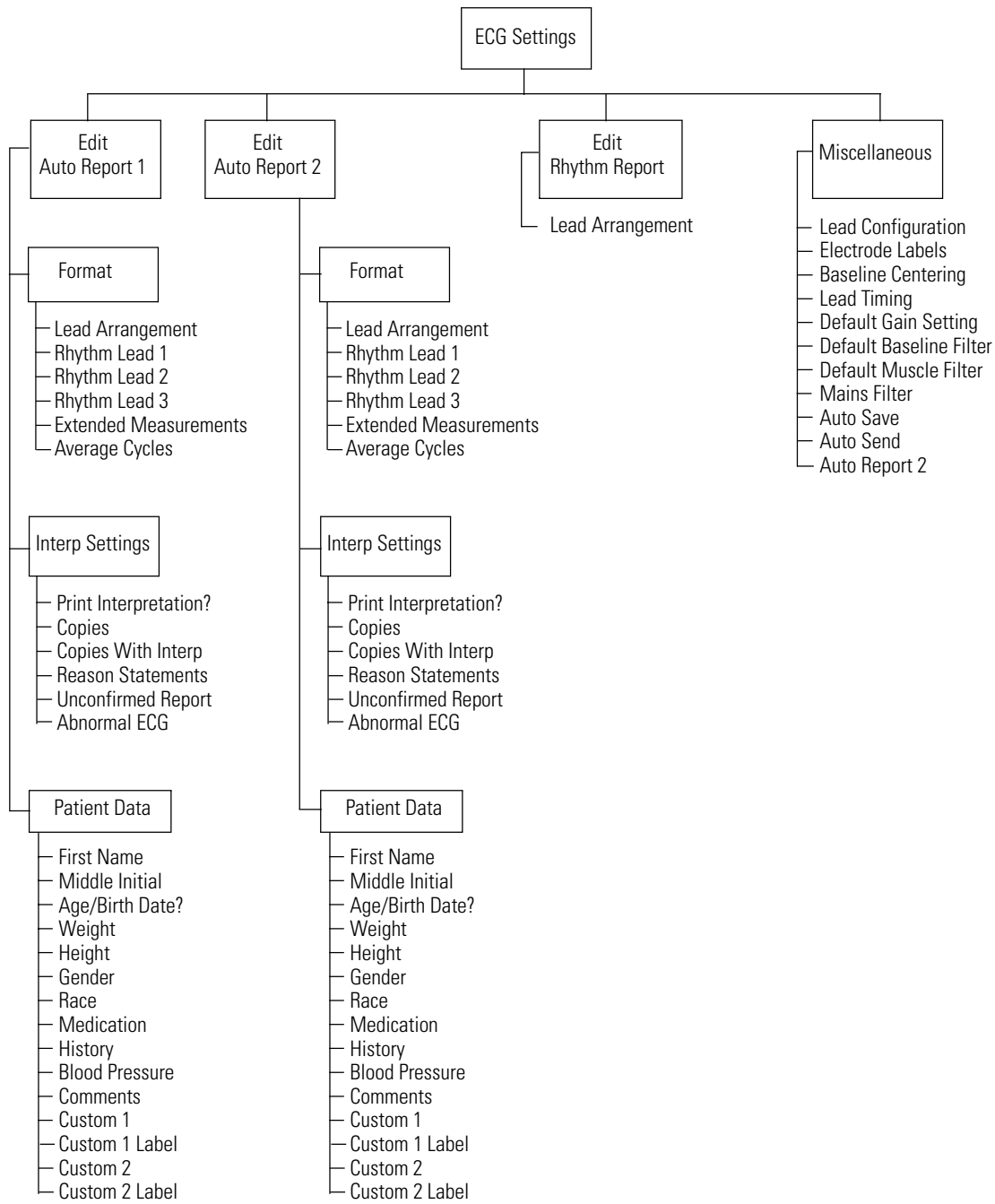


Reviewing the Auto Report Settings

An Auto ECG is a report of ECG data in one of two user-defined formats: Auto Report 1 or Auto Report 2. For an example, see [Figure 23](#). To learn how to set up or interpret a report, see the references on [page 36](#).

Note If you want a second predefined format to be available, enable Auto Report 2. To learn how, see [“Reviewing the Miscellaneous ECG Settings”](#) on page 43.

Figure 23. Auto Report Example — 3x4 +3R Lead Arrangement

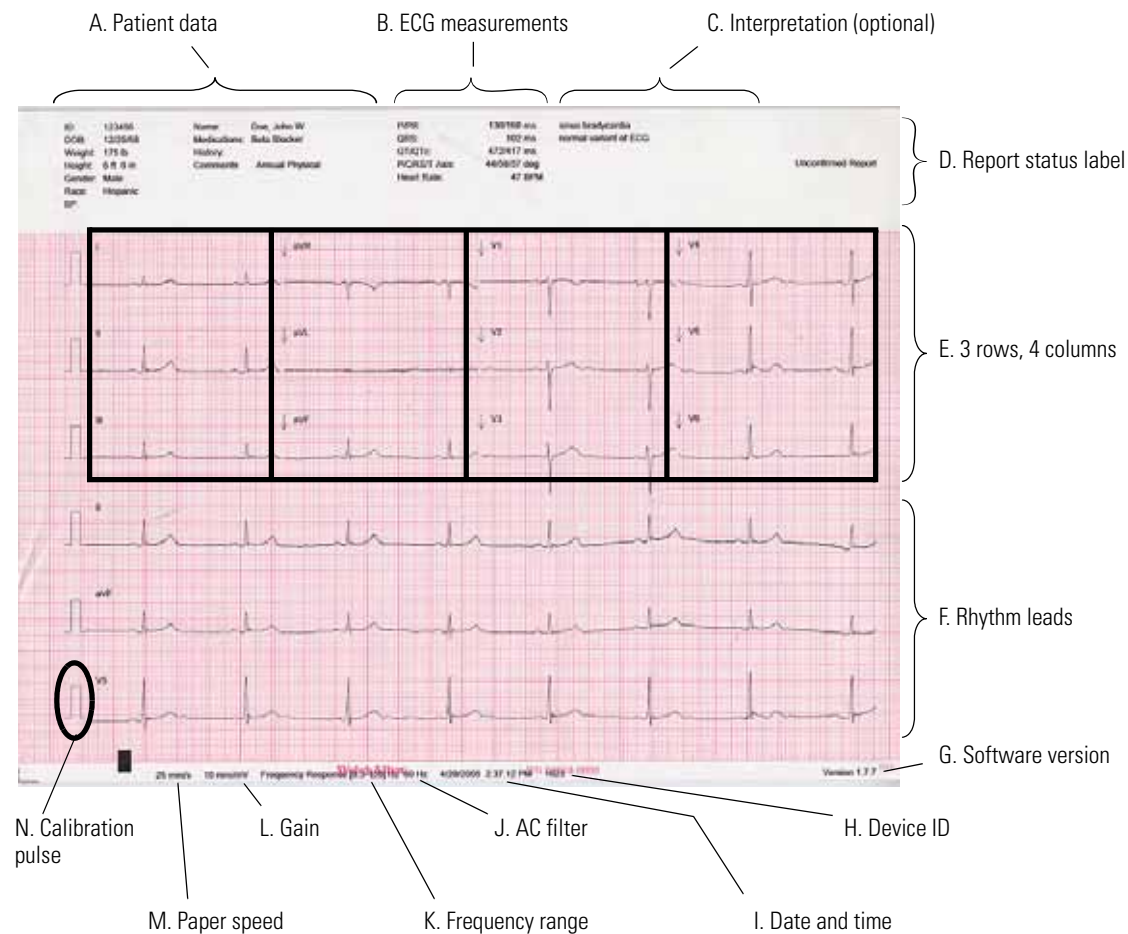


Item (in Figure 23 on page 35)	Description
A. Patient data	See “Reviewing the Patient Data Fields Available for Auto Reports” on page 40.
B. ECG measurements	Standard.
C. Interpretation (optional)	See “Reviewing the Interpretation and Copy Settings for Auto Reports” on page 39.
D. Report status label	See “Reviewing the Interpretation and Copy Settings for Auto Reports” on page 39.
E. 3 rows, 4 columns	See “Reviewing the Format Settings for Auto Reports” on page 37.
F. Rhythm leads	See “Reviewing the Format Settings for Auto Reports” on page 37.
G. Software version	See also “Reviewing the Device Information” on page 29.
H. Device ID	See “Device ID” on page 27.
I. Date and time	See “Set Date/Time” on page 27.
J. AC filter	See “Mains Filter” on page 43.
K. Frequency range	Lower limit: baseline filter on = 0.5, off = 0.3 Upper limit: muscle filter on = 35, off = 150 See “Baseline Filter” on page 61 and “Muscle Filter” on page 61.
L. Gain	See “Gain” on page 61.
M. Paper speed	See “Speed” on page 61.
N. Calibration pulse	Amplitude reference — represents the current height of a one-millivolt signal. Is adjusted for the selected gain: 5 mm/mV = 0.5 x 10 mm/mV = 1 x 20 mm/mV = 2 x

Reviewing the Format Settings for Auto Reports

1. Press the Menu key .
2. Choose **ECG Settings** > **Edit Auto Report 1** (or 2) > **Format**.

The following screen appears.

Figure 24. Auto Report “Format” Screen

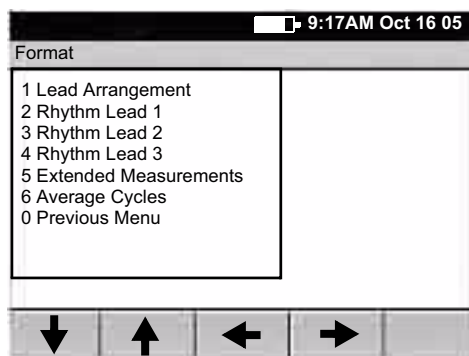

3. Change any desired settings.

For a report example, see [Figure 23](#) on page 35.

Setting	Description
Lead Arrangement	<p>Arrangement of the leads on the report.</p> <ul style="list-style-type: none"> • 3x4 3 rows x 4 columns • 3x4 +1R 3 rows x 4 columns + 1 rhythm lead • 3x4 +3R 3 rows x 4 columns + 3 rhythm leads • 6x2 6 rows x 2 columns • 12x1 12 rows x 1 column • 6x2 50 mm/s 6 rows x 2 columns, 50 mm/s • 6x2 Ext. 6 rows x 2 columns, extended printouts (two pages, 20 seconds of ECG data) • No Print No report prints
Rhythm Lead 1	Rhythm lead to print at the bottom of 3x4 +1R and 3x4 +3R reports.
Rhythm Lead 2	Second rhythm lead to print at the bottom of 3x4 +3R reports.
Rhythm Lead 3	Third rhythm lead to print at the bottom of 3x4 +3R reports.
Extended Measurements	On or off. When on, an additional page prints with the report. Extended measurements include the values for several common parameters, such as Q, R, and S amplitude and ST values. The amplitudes are expressed in microvolts. The durations are expressed in milliseconds. The measurements cannot be edited.


Setting (Continued)	Description (Continued)
Average Cycles	If desired, an additional page prints with the report. Average cycles show the dominant waveforms for all 12 leads. <ul data-bbox="659 363 1333 443" style="list-style-type: none"><li data-bbox="659 363 1333 388">• 3x4 50 mm/s + 3R 3 rows x 4 columns + 3 rhythm leads, 50 mm/s<li data-bbox="659 388 1333 413">• 6x2 50 mm/s + 6R 6 rows x 2 columns + 6 rhythm leads, 50 mm/s<li data-bbox="659 413 1333 443">• No Print Average cycles page does not print.

Reviewing the Interpretation and Copy Settings for Auto Reports

1. Press the Menu key .
2. Choose **ECG Settings** > **Edit Auto Report 1** (or 2) > **Interp Settings**.

The following screen appears.

Figure 25. "Interpretation Settings" Screen

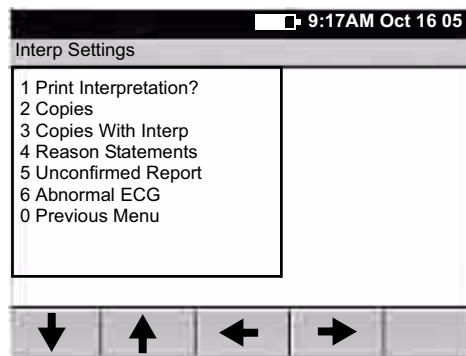

3. Change any desired settings.

For a report example, see [Figure 23](#) on page 35.

Setting	Description
Print Interpretation?	On or off. Determines whether interpretation is printed and saved with reports.
Copies	Number of copies to print automatically in addition to the original report: 0, 1, 2, 3, 4, or 5.
Copies With Interp	On or off. Determines whether interpretation is printed on the automatic copies.
Reason Statements	On or off. Determines whether reasons (criteria) are printed with the interpretation statements.
Unconfirmed Report	On or off. Determines whether the label "Unconfirmed Report" is printed on reports.
Abnormal ECG	On or off. Determines whether the label "Abnormal ECG" is printed on reports. Available only for systems using automatic interpretation.

Reviewing the Patient Data Fields Available for Auto Reports

You can determine which fields appear during patient data entry for Auto ECGs.


Note Spirometry tests use a separate set of data-entry fields, as described in the spirometry manual.

To Choose the Fields

1. Press the Menu key .
2. Choose **ECG Settings > Edit Auto Report 1** (or 2) > **Patient Data**.

The following screen appears.

Figure 26. “Patient Data” Screen for ECGs

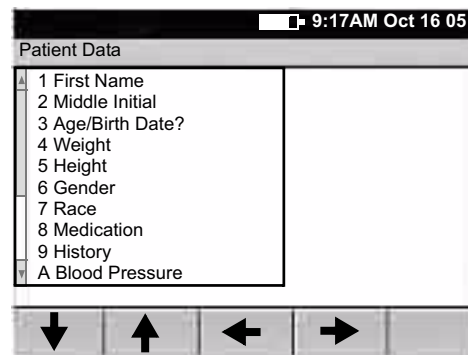

The **Patient ID** and **Last Name** fields always appear on the **Enter New Patient** screen, as shown in [Figure 33](#) on page 50. Since these two fields cannot be disabled, they do not appear on this user-selectable list.

3. Change any desired settings.

Disabled items (set to *off* or *no*) neither display nor print.

Field	Description
First Name	Yes or no. If yes, this field is enabled.
Middle Initial	Yes or no. If yes, this field is enabled.
Age/Birth Date?	Birth Date, age, or off. Determines whether and how this data is labeled and entered. For instructions on changing the date format (MM/DD/YY or DD/MM/YY), see “Reviewing the Device Configuration Settings” on page 27.
Weight	Yes or no. If yes, this field is enabled for entering patients’ weight. For instructions on changing the weight units (kg or lb), see “Reviewing the Device Configuration Settings” on page 27.
Height	Yes or no. If yes, this field is enabled for entering patients’ height. For instructions on changing the height units (cm, in., or ft and in.), see “Reviewing the Device Configuration Settings” on page 27.
Gender	Yes or no. If yes, this field is enabled. Data-entry choices: Male, Female, or Unknown.
Race	Yes or no. If yes, this field is enabled. Data-entry choices: Blank, Caucasian, Black, Hispanic, Asian, Unknown.

Field (Continued)	Description (Continued)
Medication	Yes or no. If yes, this field is enabled. During data entry, choose up to three items from the list of patient medications. To learn how to edit this list, see “Reviewing the Medication List” on page 30.
History	Yes or no. If yes, this field is enabled. During data entry, choose up to three items from the list of patient clinical conditions. To learn how to edit this list, see “Reviewing the History List” on page 31.
Blood Pressure	Yes or no. If yes, this field is enabled for entering blood pressure in standard ### / ### format.
Comments	Yes or no. If yes, this field is enabled for entering comments.
Custom 1	Yes or no. If yes, this field is enabled for entering data of your choice.
Custom 1 Label	You may define a label for the Custom 1 field if desired.
Custom 2	Yes or no. If yes, this field is enabled for entering data of your choice.
Custom 2 Label	You may define a label for the Custom 2 field if desired.

Reviewing the Rhythm Report Settings


Rhythm Reports can print either 3, 6, or all 12 leads at a time.

To Change the Number of Leads That Print

1. Press the Menu key .
2. Choose **ECG Settings > Edit Rhythm Report > Lead Arrangement**.

The following screen appears.

Figure 27. “Lead Arrangement” Submenu

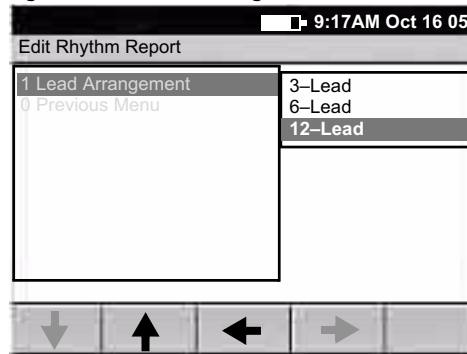

3. Choose the number of leads you want to print at a time: **3**, **6**, or **12**.


For instructions on cycling through 3-lead or 6-lead groupings while printing a rhythm report, see [Step 1](#) on page 56.

Reviewing the Miscellaneous ECG Settings

1. Press the Menu key .
2. Choose **ECG Settings > Miscellaneous**.

The following screen appears.

Figure 28. “Miscellaneous” Screen for ECG Settings

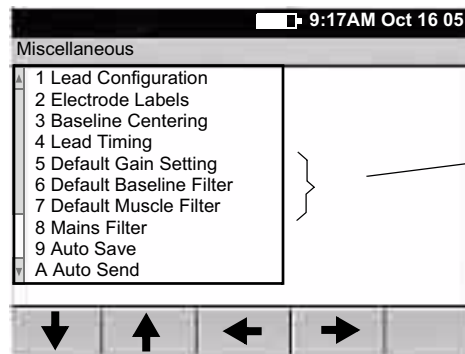

These three default settings—gain, baseline filter, and muscle filter—determine the values used every time you begin a new test, even if these values have been temporarily changed during ECG testing.

3. Change any desired settings.


Setting	Description
Lead Configuration	Standard (I II III, aVR aVL aVF, V1 V2 V3, V4 V5 V6) or Cabrera (aVL I –aVR, II aVF III, V1 V2 V3, V4 V5 V6).
Electrode Labels	AHA or IEC.
Baseline Centering	On or off. When on, aligns the isoelectric line of all leads.
Lead Timing	Simultaneous or sequential. “Simultaneous” prints ECG data that was captured simultaneously for all lead groups. “Sequential” prints ECG data that was captured at sequential intervals for each lead group in turn.
Default Gain Setting	5 mm/mV, 10 mm/mV, 20 mm/mV, or Auto. (AUTO is available for Auto ECGs only, not rhythm ECGs. AUTO is usually the best setting, but some waveforms may be easier to read on other settings.) For details, see “Gain” on page 61.
Default Baseline Filter	On or off. For details, see “Baseline Filter” on page 61.
Default Muscle Filter	On or off. For details, see “Muscle Filter” on page 61.
Mains Filter	Off, 50 Hz, 60 Hz. Use of this filter is recommended. For suggestions on eliminating AC interference, see page 85 .
Auto Save	On or off. When on, the electrocardiograph automatically saves all ECGs (except <i>stat</i> ECGs) to its test directory. When off, every time you print an ECG test you are asked if you want to save. For description of the test directory, see “Managing Saved Tests” on page 64.

Setting (Continued)	Description (Continued)
Auto Send	<p data-bbox="688 285 1422 338">Memory card, workstation, or off. Automatically sends all ECGs (except stat ECGs) to the option of your choice.</p> <p data-bbox="688 365 1422 417">If set to memory card, an SD memory card must be in place during testing. For slot location, see Figure 5 on page 7.</p> <p data-bbox="688 445 1422 495">If set to workstation, a USB cable must connect a CardioPerfect workstation to the electrocardiograph's Com port B (10101). For port location, see Figure 5 on page 7.</p>
Auto Report 2	On or off. When on, a second predefined report format is available.

5**Performing ECG Tests**

Connecting the Leads to the Patient	46
Recording an Auto ECG	49
Recording a Rhythm ECG	56
Searching for Saved Patient Data	57
Adjusting the ECG Waveforms	61

Connecting the Leads to the Patient

1. Help the patient get comfortable. Patient preparation is important for a successful ECG.
 - a. Describe the procedure. If desired, press the Help key  and print the page entitled “What Is An ECG?” for the patient to read.
 - b. Help the patient get warm and relaxed. Excessive patient movement could interfere with the operation of the electrocardiograph.
 - c. Put the patient in a reclining position with the head slightly higher than the heart and legs.



WARNING ECG electrodes could cause skin irritation. Examine the skin for signs of irritation or inflammation.

2. Prepare electrode locations. See [Figure 29](#) on page 47.
 - a. Shave if necessary.
 - b. Clean with alcohol or acetone.
 - c. Allow to dry.
3. Attach the electrodes and lead wires securely.
 - For reusable electrodes:


Straps must neither slide nor be so tight as to cause discomfort.

The electrode paste, gel, or creme must cover an area the size of the electrode but no larger, especially on the chest.
 - For disposable tab electrodes:

Place the electrode tab between the “jaws” of the electrode adapter, keeping the tab flat.

Gently tug on the adapter to ensure that it is properly placed on the electrode. (Each time you remove and reattach an electrode, the conductive gel becomes weaker and less effective.)

Figure 29. Electrode Placement Locations

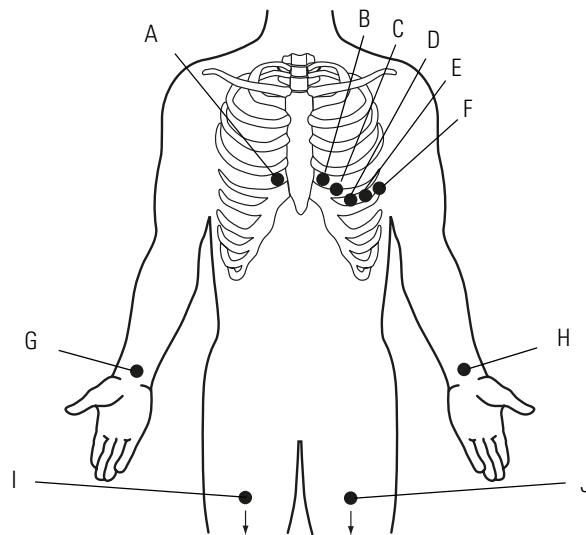

	Electrodes		Locations
	AHA	IEC	
A	V1 red	C1 red	Fourth intercostal space at right sternal border.
B	V2 yellow	C2 yellow	Fourth intercostal space at left sternal border.
C	V3 green	C3 green	Midway between V2 and V4.
D	V4 blue	C4 brown	Fifth intercostal space at left of midclavicular line.
E	V5 orange	C5 black	Anterior axillary line at same horizontal level as V4.
F	V6 purple	C6 purple	Mid-axillary line on same horizontal level as V4 and V5.
G	RA white	R red	Just above right wrist on inside of arm.
H	LA black	L yellow	Just above left wrist on inside of arm.
I	RL green	N black	Just above right ankle.
J	LL red	F green	Just above left ankle.


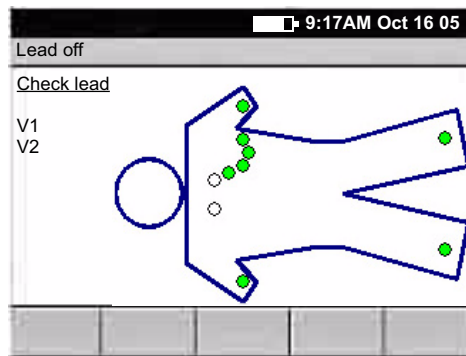
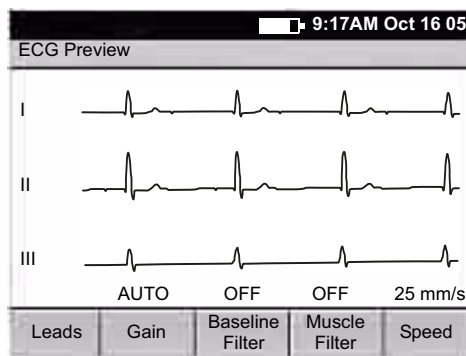
4. If the electrocardiograph's display is blank, press .
5. If the **Lead Off** screen appears, as shown here, reattach any leads that are flashing.

Figure 30. "Lead Off" Screen



The most common ECG problems are poor electrode contact and loose leads. When all leads have been connected for three seconds, the following screen appears.

Figure 31. "ECG Preview" Screen



6. (Optional) Use the softkeys as desired.
For details, see ["Adjusting the ECG Waveforms"](#) on page 61.
7. Go to the procedure for the type of ECG test you want to perform.
 - ["Recording an Auto ECG"](#) on page 49
 - ["Recording a Rhythm ECG"](#) on page 56


Recording an Auto ECG

An Auto ECG is a report typically showing a 10-second acquisition of 12 leads of ECG information combined with patient data, interpretation, and measurements matrix. Two user-defined formats are available: Auto Report 1 or Auto Report 2. To learn how to set up the Auto ECG report format, see [“Reviewing the Auto Report Settings”](#) on page 35.

As shown in the following diagram, there are the two types of Auto ECG: normal and stat. For details, see these procedures:

- [“Recording a Normal Auto ECG”](#) on page 50
- [“Recording a Stat Auto ECG”](#) on page 55


Figure 32. Auto ECG Testing, Process Diagram

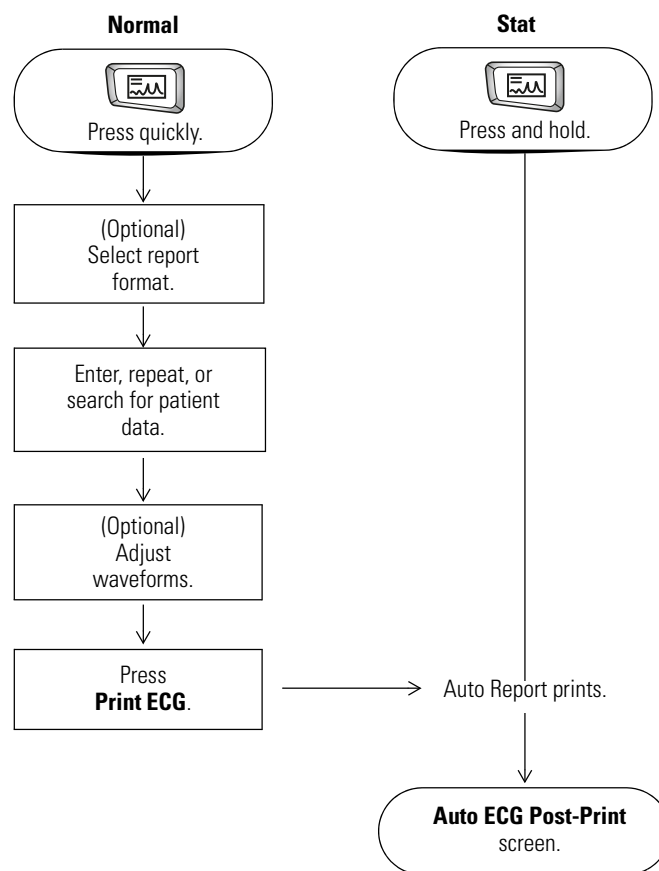

Recording a Normal Auto ECG

For a normal Auto ECG, you enter patient data and do other optional tasks before printing, as shown in [Figure 32](#) on page 49.

To Record a Normal Auto ECG

1. Press the **Auto ECG** key  quickly.
Do not hold it down, or a stat ECG would begin.
2. If prompted, choose **Auto Report 1** or **Auto Report 2**.
3. If asked “Repeat same patient?,” press the desired softkey.


- **Yes** — to perform another ECG for the same patient.

The **Auto ECG Acquisition** screen appears. Go to [Step 5](#) on page 51.

- **No** — to clear the current patient data.

The following screen appears.

Figure 33. “Enter New Patient” Screen

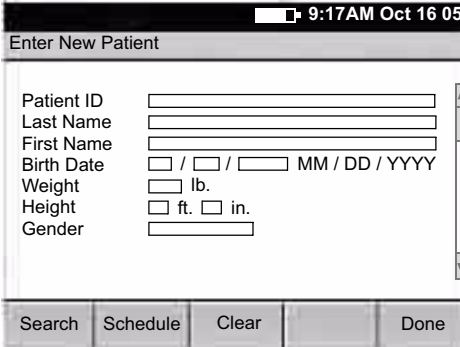


For details about these data fields—including how to choose which fields display and print—see [“Reviewing the Patient Data Fields Available for Auto Reports”](#) on page 40.

4. Enter or search for the patient data.
 - If you want to find a patient whose data has already been entered, go to [“Searching for Saved Patient Data”](#) on page 57.
 - If you want to enter the data, fill in the fields.

When finished, press the desired softkey:

Clear deletes the entered data and returns to the **Patient ID** field.

Done accepts the entered data and goes to the **Auto ECG Acquisition** screen. See [Figure 34](#) on page 51.

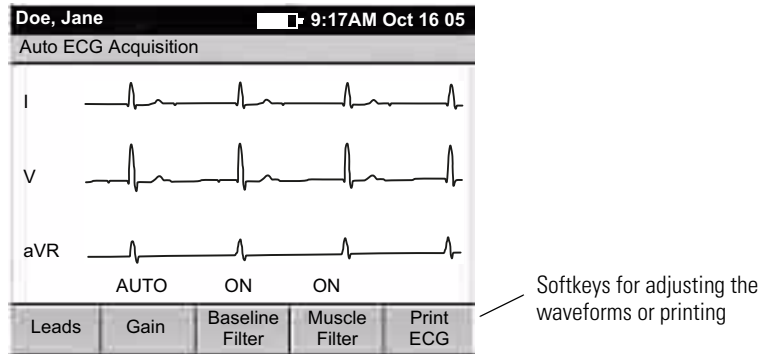Figure 34. "Auto ECG Acquisition" Screen

5. Verify ECG quality on the screen.



WARNING Do not perform ST segment analysis on the ECG screen display, since these ECG representations are scaled. Make manual measurements of ECG intervals and magnitudes on printed ECG reports only.


6. (Optional) Adjust the waveforms.
See ["Adjusting the ECG Waveforms"](#) on page 61.
7. Press **Print ECG**.
8. If prompted, choose whether to wait for the electrocardiograph to acquire 10 seconds of filtered, processed data before printing.

If you override the wait time and print the available data immediately, be aware that the printed data will be insufficient in quality or quantity or both.

The report prints.

Note If a red stripe appears along the edge of your report, replace the paper. See ["Loading the Thermal Chart Paper"](#) on page 21.

After printing, the **Auto ECG Post-Print** screen appears. [Figure 35](#) on page 52.

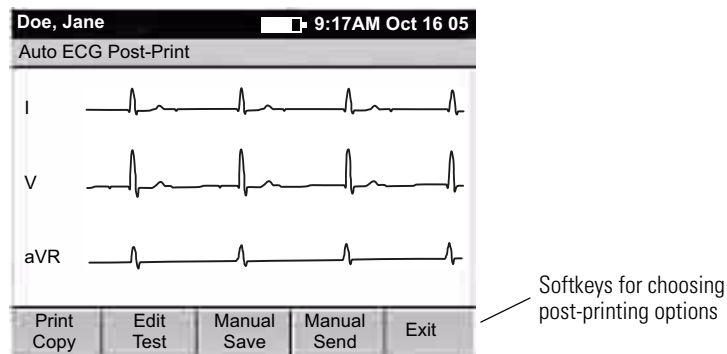Figure 35. “Auto ECG Post-Print” Screen

9. (Optional) If you want to repeat the test, go back to [Step 1](#) on page 50.
10. Press the desired softkey.

Softkey	Effect
Print Copy	Prints a copy of the test. To learn how to print multiple copies of all tests automatically, see “Reviewing the Interpretation and Copy Settings for Auto Reports” on page 39.
Edit Test	Brings up the Edit Test – Patient Data screen. From here you can edit or confirm the patient data and interpretation statements. See “To Edit or Confirm a Test Directly After Printing” on page 54.
Manual Save	Saves the test to the electrocardiograph’s test directory. See “Managing Saved Tests” on page 64. This Manual Save softkey appears only when Auto Save is disabled. For more on Auto Save, see “Reviewing the Miscellaneous ECG Settings” on page 43.
Manual Send	Displays two softkey options: <ul style="list-style-type: none"> • Memory Card An SD memory card must be in place. For slot location, see Figure 5 on page 7. • Workstation A USB cable must connect the CardioPerfect workstation to the electrocardiograph’s Com port B (IOIOI). For port location, see Figure 5 on page 7. <p>This Manual Send softkey appears only when Auto Send is disabled. For more on Auto Send, see “Reviewing the Miscellaneous ECG Settings” on page 43.</p>
Exit	The ECG Preview screen appears if all leads are connected to the patient.



Caution The requirements of AAMI EC11, Section 3.2.7.2, Frequency and Impulse Response, for an impulse triangle waveform may be impacted by up to 5 milliseconds of small amplitude dampened ringing immediately after the impulse when the muscle filter (35 Hz) is turned on or a small amplitude offset when the baseline filter (0.5 Hz) is turned on. These requirements are unaffected by any other combination of filters turned on or off. Measurements performed by the optional interpretation algorithm are unaffected by any filter selections.

To Edit or Confirm a Test Directly After Printing

Note A qualified physician must review and confirm all tests before patients are treated. If changes are needed for any saved test, you can edit two types of data:

- patient data
- interpretation statements

1. From the **Auto ECG Post-Print** screen ([Figure 35](#) on page 52), press **Edit Test**.

The **Edit Test – Patient Data** screen appears.

2. (Optional) Edit the patient data.
3. Press the desired softkey.

Softkey	Effect
Interp	<p>Saves any changes, and displays the Edit Test – Interpretation screen.</p> <ol style="list-style-type: none"> 1. (Optional) Edit the interpretation statements to be saved with the test. 2. Press the desired softkey: <ul style="list-style-type: none"> • Patient Data saves any changes, and returns to the Edit Test – Patient Data screen. • Confirm saves any changes, sets the test status to “confirmed,” and returns to the Auto ECG Post-Print screen. • Cancel discards any changes, and returns to the Auto ECG Post-Print screen. • Done saves any changes, and returns to the Auto ECG Post-Print screen. <p>For details on interpretation, see “Automatic ECG interpretation” on page 4.</p>
Cancel	Discards any changes, and returns to the Auto ECG Post-Print screen.
Done	Saves any changes, and returns to the Auto ECG Post-Print screen.

Recording a Stat Auto ECG

A stat Auto ECG is an immediate printout in Auto Report 1 format.

Stat mode bypasses patient data entry, as shown in [Figure 32](#) on page 49. A temporary ID number is assigned to the patient to identify stat tests. After printing, you may enter the patient data by editing the test.

In stat mode, Auto Send and Auto Save are always disabled, even if they are enabled in the ECG settings. If you want to send or save a stat Auto ECG, you may do so manually after you print.

To Record a Stat Auto ECG

1. Press and hold the **Auto ECG** key .

The electrocardiograph begins acquiring ECG data. After it has acquired 10 seconds of quality data, it prints a report.

2. Go to [Step 8](#) on page 51.


Continue the procedure as if you have just pressed the **Print ECG** softkey.

Recording a Rhythm ECG

A Rhythm ECG is a continuous, real-time printout of a rhythm strip with a user-defined lead arrangement. For details on reviewing or changing the lead arrangement in your reports, see [“Reviewing the Rhythm Report Settings”](#) on page 42.

Rhythm ECGs are printouts only. They cannot be saved or sent electronically.

Figure 36. Rhythm ECG Testing, Process Diagram

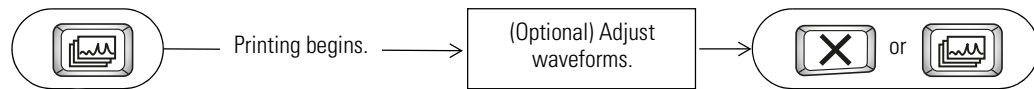


To Record a Rhythm ECG

1. Press the **Rhythm ECG** key  .

Printing begins.

The screen displays 3 leads at a time from the leads currently printing. The printout includes either 3, 6, or 12 leads at a time. To find out how to change this number, see [“Reviewing the Rhythm Report Settings”](#) on page 42.

Figure 37. “Rhythm ECG” Screen

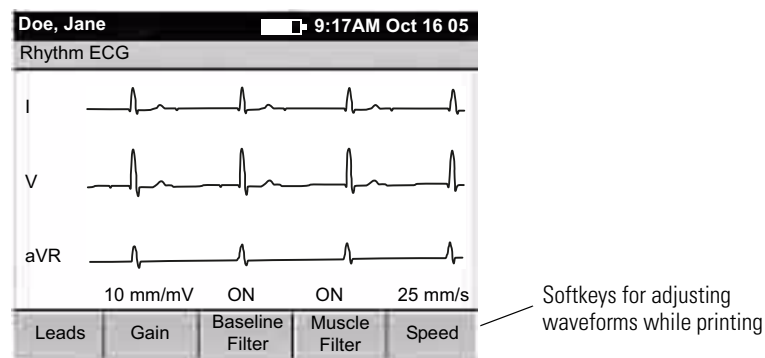

1. (Optional) Press the softkeys to adjust the waveforms.

See [“Adjusting the ECG Waveforms”](#) on page 61.

2. Press  or  to stop printing.

The **ECG Preview** screen appears if all leads are connected to the patient.

(Optional) Adjust the waveforms.

Searching for Saved Patient Data

During an ECG or spirometry test, instead of entering patient data manually you can search for patient data that has been saved in either of two places:

- **In the scheduled patients list**

The scheduled patients list identifies up to 40 patients whose data has been entered into the electrocardiograph’s memory for an ECG or spirometry test that day. For more, see [“Managing the Scheduled Patients List”](#) on page 68.


- **In the test directory**

The test directory is the collection of tests saved in the electrocardiograph’s memory. This directory holds up to 50 ECG and 50 spirometry tests. For more, see [“Managing Saved Tests”](#) on page 64.

The following diagram illustrates the procedure for both types of searching. For step-by-step instructions, see these sections:

- [“To Search the Test Directory”](#) on page 58
- [“To Search the Scheduled Patients List”](#) on page 60

Figure 38. Patient Search, Process Diagram

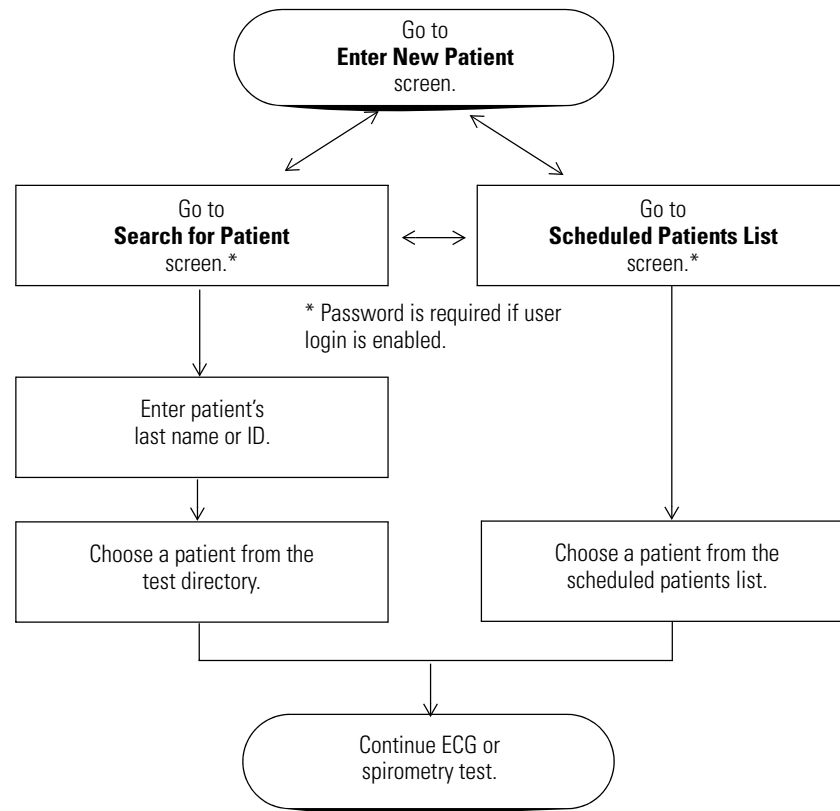

To Search the Test Directory

1. From the **Enter New Patient** screen (Figure 33 on page 50), press **Search**.

If prompted to log in, enter your user ID and password.

The following screen appears.

Figure 39. "Search for Patient" Screen

Search for Patient

Patient ID:

Last Name:

New Patient Schedule Search


Brings up any matching names in the test directory.

2. Enter the patient's complete identification number or last name (partial or complete).
3. Press **Search**.

If one or more patients match your entry, their names appear.

Note The other two softkeys exit this search:

- **New Patient** returns to the **Enter New Patient** screen. See Figure 33 on page 50.
- **Schedule** brings up the **Scheduled Patients List** screen. Go to ["To Search the Scheduled Patients List"](#) on page 60.

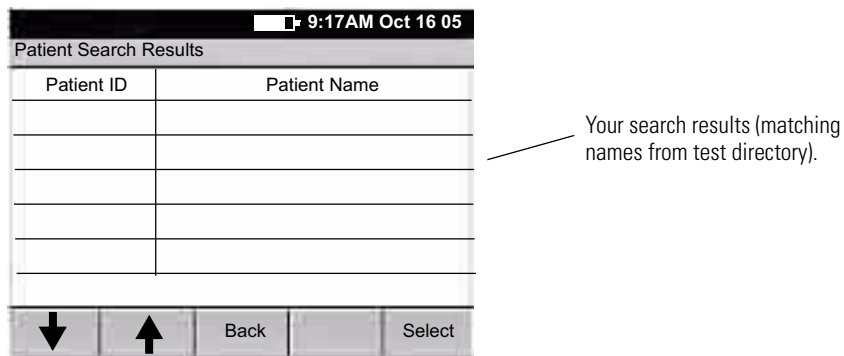Figure 40. "Patient Search Results" Screen

4. Take the desired action.

- Press **Back**.

The **Search for Patient** screen reappears.


- Highlight a patient name, and press **Select**.

The next screen in your procedure appears. (For ECG tests, go to [Step 5](#) on page 51. For spirometry tests, see the spirometry manual.)

To Search the Scheduled Patients List

1. From the **Enter New Patient** screen (Figure 33 on page 50), press **Schedule**.
If prompted to log in, enter your user ID and password.
The following screen appears.

Figure 41. “Scheduled Patient List” Screen (for Searching)

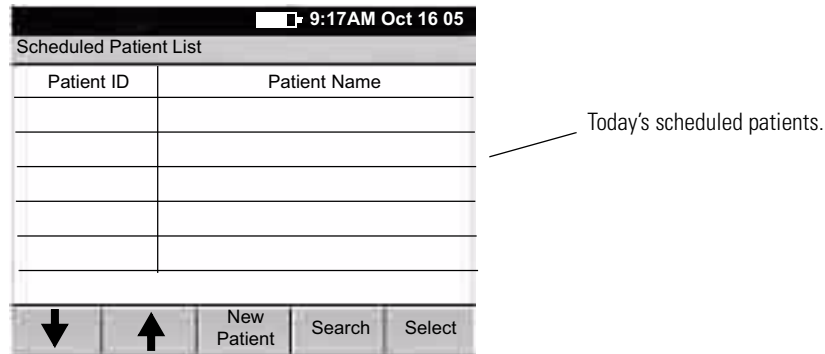

2. Highlight the desired name. Press **Select**.





The next screen in your procedure appears. (For ECG tests, go to [Step 5](#) on page 51. For spirometry tests, see the spirometry manual.)

Note The other two softkeys exit this search:

- **New Patient** returns to the **Enter New Patient** screen. See [Figure 33](#) on page 50.
- **Search** brings up the **Search for Patient** screen. Go to [“To Search the Test Directory”](#) on page 58.

Adjusting the ECG Waveforms

The following softkeys appear on the **Auto ECG Acquisition** and **Rhythm ECG** screens (Figure 34 on page 51 and Figure 37 on page 56). Use these softkeys before printing an Auto ECG or while printing a rhythm ECG.

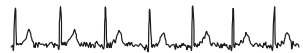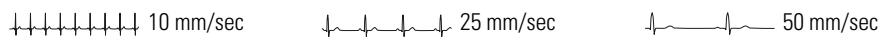Softkey	Effect
Leads	Cycles through the leads in groups of three on the screen. For rhythm reports, also cycles through the lead groups that are printing (if <12). To learn how to change the number of leads per group—3, 6, or 12—on rhythm reports, see “Reviewing the Rhythm Report Settings” on page 42. To learn how to change between standard and Cabrera lead groupings, see “Reviewing the Miscellaneous ECG Settings” on page 43.
Gain	Cycles through the gain settings in mm/mV (5, 10, 20, AUTO), zooming in and out. (AUTO is available for Auto ECGs only, not rhythm ECGs. AUTO is usually the best setting, but some waveforms may be easier to read on other settings.) To learn how to change the the gain’s default setting, see “Reviewing the Miscellaneous ECG Settings” on page 43.
	
Baseline Filter	Toggles between the two baseline filter settings (on or off). This filter reduces “wandering baseline,” an upward and downward fluctuation of the waveforms. It is preferable, if possible, to eliminate or reduce wandering baseline by addressing the causes, as explained on page 84 . To learn how to change the the filter’s default setting, see “Reviewing the Miscellaneous ECG Settings” on page 43. Caution: You cannot perform ST segment analysis on waveforms that were recorded with the baseline filter turned on. For details, see the caution on page 53 .
	 Wandering baseline
Muscle Filter	Toggles between the two muscle filter settings (on or off). This filter reduces muscle tremor interference: random, irregular voltage superimposed on the waveforms. It is preferable, if possible, to eliminate or reduce muscle tremor by addressing the causes, as explained on page 84 . To learn how to change the the filter’s default setting, see “Reviewing the Miscellaneous ECG Settings” on page 43. Caution: You cannot perform ST segment analysis on waveforms that were recorded with the muscle filter turned on. For details, see the caution on page 53 .
	 Muscle tremor interference
Speed (available for Rhythm ECGs only)	Cycles through the paper speed settings in mm/sec (10, 25, 50).
	

6

Performing Administrative Tasks

Managing Saved Tests	64
Managing the Scheduled Patients List	68
Managing Data Security	69


Managing Saved Tests

The test directory is the collection of tests saved in the electrocardiograph's memory. This directory holds up to 50 ECG and 50 spirometry tests. When the directory gets full, the electrocardiograph prompts you to delete the oldest tests before saving new ones.


When performing Auto ECG or spirometry tests, you may select patients from this directory rather than manually entering their data at the time of the test.

This section explains how to manage—view, edit, print, and send—saved tests. These tasks are all accomplished through the **Test Directory** menu, as illustrated here.

Figure 42. "Test Directory" Menu Tree

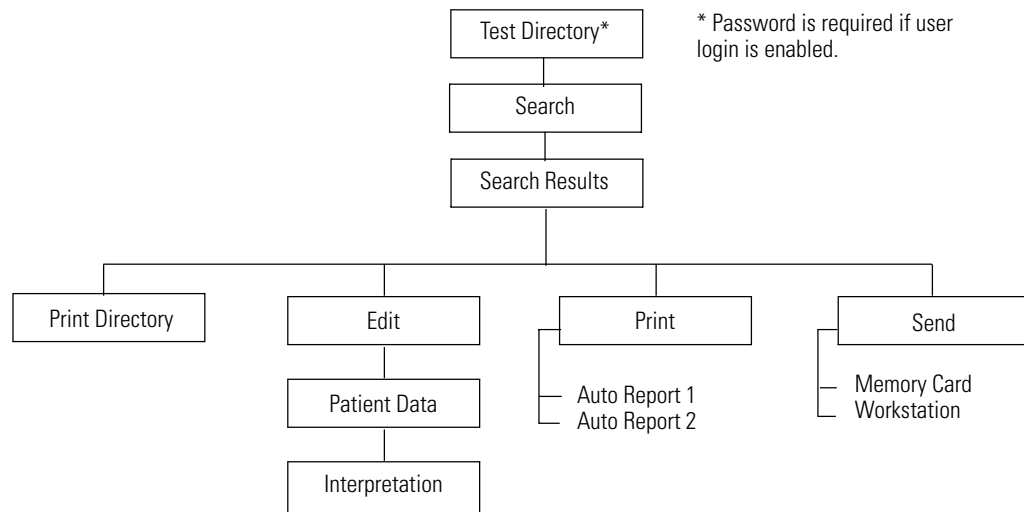


To View, Print, or Send Saved Tests

1. Press the Menu key .
2. Choose **Test Directory**.

If prompted to log in, enter your user ID and password.

The following screen appears.

Figure 43. Test Directory “Search” Screen

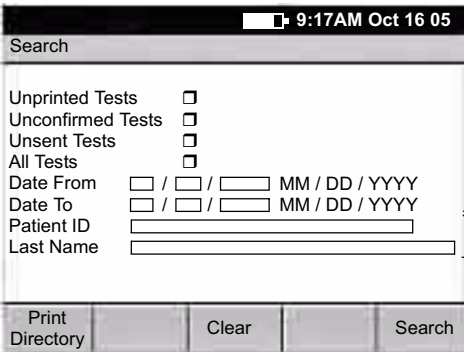

Search

9:17AM Oct 16 05

Unprinted Tests

Unconfirmed Tests

Unsent Tests

All Tests

Date From / / MM / DD / YYYY

Date To / / MM / DD / YYYY

Patient ID

Last Name

Print Directory Clear Search

Use these criteria to find and perform actions on a batch of tests.

Use these criteria to find and perform actions on a single test.

3. (Optional) Press **Print Directory** to print the contents of the test directory.
4. Select or fill in the search criteria, as explained in [Figure 43](#).
5. Press **Search**.

If any saved tests match your criteria, the **Search Results** screen appears.

6. Determine whether you need to select a test.
 - If you searched for a batch of tests, all tests are automatically selected.
 - If you searched for a particular patient, select the desired test.
7. Press the desired softkey.
 - **Print** — prints the selected test(s).
 - **Send** — sends the selected test(s) to your choice:

Memory Card

An SD memory card must be in place.

Workstation

A USB cable must connect a CardioPerfect workstation to the electrocardiograph's Com port B (10101).

To Edit or Confirm Saved Tests

Note A qualified physician must review and confirm all tests before patients are treated. If changes are needed for any saved test, you can edit two types of information:

- patient data
- interpretation statements

For an example of each type of information as it appears on a report, see [Figure 23](#) on page 35.

1. Select the desired test(s) from the test directory.

For instructions, see [“To View, Print, or Send Saved Tests”](#) on page 65, [Step 1](#) through [Step 6](#).

2. Press **Edit**.

The **Edit Test – Patient Data** screen appears.

If you have selected one test, the patient data displays for that test. If you have selected multiple tests, the patient data displays for the *first* test.

3. (Optional) Edit the patient data.
4. Press the desired softkey.


Softkey	Effect
Interp	<p>Saves any changes, and displays the Edit Test – Interpretation screen for the same test.</p> <ol style="list-style-type: none"> 1. (Optional) Edit the interpretation. 2. Press the desired softkeys: <ul style="list-style-type: none"> • Patient Data saves any changes, and returns to the Edit Test – Patient Data screen for the same test. • Confirm saves any changes, sets the test status to “confirmed,” and displays patient data for the next test, if any. • Previous Test saves any changes, and displays patient data for the previous test. Appears only when previous tests are available in your search results. • Next Test saves any changes, and displays patient data for the next test. Appears only when next tests are available in your search results. • Done saves any changes, and returns to the Test Directory Search screen.

Softkey	Effect (Continued)
Previous Test	Saves any changes, and displays the previous test. Appears only when previous tests are available in your search results.
Next Test	Saves any changes, and displays the next test. Appears only when next tests are available in your search results.
Done	Saves any changes, and returns to the Test Directory Search screen.

Managing Data Security

The CP 200 electrocardiograph incorporates security features that can keep patient data private and confidential. To implement your data security plan, use the **Device Administration** menu, which is illustrated here and described in the following pages.

Figure 45. “Device Administration” Menu Tree

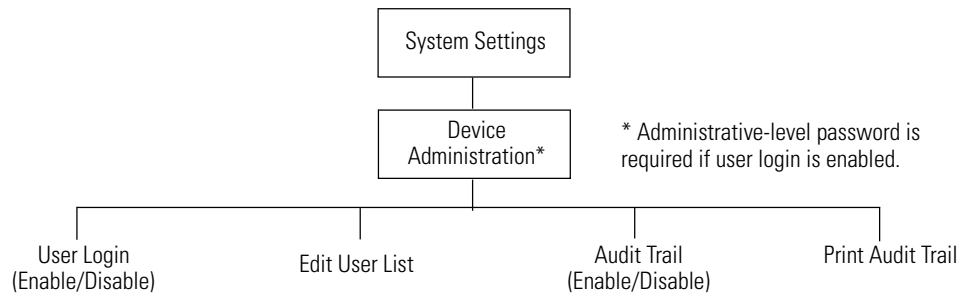

Working With the User List and User Login

The user list identifies all individuals who are authorized to access patient data.

The user list can hold up to 25 users. After the list reaches 25 users, you can add names only if some names are set to inactive status. New names replace inactive names. If all 25 names are active, you receive a message that the user list is full.

Two levels of access can be assigned:


- **User level**

When the user login feature is enabled, all users on the user list have access to the test directory and the scheduled patient list, including searching and editing privileges.

- **Administrator level**

When the user login feature is enabled, only administrator-level users have access to the administrative functions. For an outline of these functions, see menu tree in [Figure 45](#) on page 69.

To Enable or Disable User Login

1. Press the Menu key .
2. Choose **System Settings > Device Administration**.
If prompted to log in, enter an administrator-level user ID and password.
3. Choose **User Login**.
4. Follow the prompts to enable or disable user login.

To Change a Password

You can change a password in either of two ways:

- At any **User Login** screen, press **Change Password**.
Follow the prompts.
- Ask an administrator-level user to change the password.
See [“To Review or Edit the User List”](#) on page 71.

To Review or Edit the User List

1. Press the Menu key .
2. Choose **System Settings > Device Administration**.

If prompted to log in, enter an administrator-level user ID and password.

Note If no administrator-level password has been assigned, the system's default user ID and password — available from Welch Allyn — works here. These defaults exist solely to provide access to the user list in the event that user login is enabled before an administrator is assigned.

3. Choose **User List**.

The user list appears.

4. Press the desired softkeys.

Softkey	Effect
Add	<p>Lets you add new users. For each user, provide the following information:</p> <ul style="list-style-type: none"> • User ID (1 to 10 characters) • Password (5 to 10 characters, displayed as asterisks for security) • Password Expiration <ul style="list-style-type: none"> - Never - 180 days • User's Name • User Level <ul style="list-style-type: none"> - User - Administrator • Active? <ul style="list-style-type: none"> - Yes (user cannot be replaced) - No (user can be replaced when list is full)
Edit	Lets you edit the information for the highlighted user.
Exit	Returns to the Device Administration screen.


Working With the Audit Trail

An audit trail, a collection of information about user activity, may be helpful or even required for record-keeping. It tracks who did what when, including these types of activity:


- Accessing the test directory
- Searching the scheduled patient list
- Editing patient data
- Accessing administrative functions

When the audit trail is enabled, the electrocardiograph collects this type of information in a file in its memory. When the allotted memory begins to get full, you receive prompts to print the audit trail. You must print the audit trail to delete (purge) this information and make room for new information to be collected.

To Enable or Disable the Audit Trail

1. Press the Menu key .
2. Choose **System Settings > Device Administration**.
If prompted to log in, enter an administrator-level user ID and password.
3. Choose **Audit Trail**.
4. Follow the prompts to enable or disable the audit trail.

To Print an Audit Trail

1. Press the Menu key .
2. Choose **System Settings > Device Administration**.
If prompted to log in, enter an administrator-level user ID and password.
3. Choose **Print Audit Trail**.
4. Follow the prompts.
After you verify that printing was successful, the audit information is deleted.

7

Maintaining the Electrocardiograph

Inspecting the Equipment	74
Cleaning the Equipment	74
Testing the Equipment	75
Recharging a Fully Discharged Battery	76
Replacing the Battery	77
Replacing the Battery (DC) Fuse	79
Replacing the AC Fuses	80
Storing the Equipment	81
Discarding the Equipment	81

Inspecting the Equipment



WARNING To ensure patient safety and proper operation, perform the following inspections daily.

- Verify that patient leads are fully inserted. For details, see [“Connecting the Patient Cable”](#) on page 20.
- Check for cracked or broken patient cable, patient leads, power cord, communications cables, display, and enclosure.
- Check for bent or missing pins on all cables.
- Check all cable and cord connections; reseal if any connectors are loose.

Cleaning the Equipment



WARNING To prevent the spread of infection, the electrocardiograph and patient cable must be kept clean, especially the components that come in contact with patients.



Caution Do not let soap or water come into contact with the electrocardiograph's internal printer, connectors, or jacks.

Do not attempt to clean the electrocardiograph or the patient cable by submersing them in liquid, autoclaving, or steam cleaning. Do not pour alcohol directly on the equipment or soak any components in alcohol. If alcohol or liquids spill into the electrocardiograph while cleaning, arrange to have it checked before using it again. For Welch Allyn phone numbers, see [page ii](#).

Monthly, or more often if needed, follow these cleaning instructions:

1. Disconnect the power plug from the AC outlet.
2. Wipe the exterior of the patient cable and electrocardiograph and with a damp cloth using mild detergent diluted in water.
3. Use 70% isopropyl alcohol to disinfect patient cable, lead wires, and equipment.
4. Dry all components with a clean, soft cloth or paper towel.
5. Wait at least 10 minutes to allow all traces of alcohol to evaporate before turning the electrocardiograph back on.

Testing the Equipment



WARNING Only qualified service personnel should perform leakage current tests.

Whenever the electrocardiograph is serviced or problems are suspected, Welch Allyn recommends the following test procedures:

1. Verify continued electrical safety of the device, using IEC 60601-1 or ANSI/AAMI ES1 methods and limits. Test for the following:
 - Patient leakage current
 - Chassis leakage current
 - Earth leakage current
 - Dielectric strength (AC and patient circuits)

2. Verify that the electrocardiograph is working properly, using an ECG simulator to acquire and print a standard 12-lead ECG of known amplitude.
 - Printing should be dark and even across the page.
 - There should be no evidence of print-head dot failure (no printing breaks forming horizontal streaks).
 - Paper should move smoothly and consistently during printing.
 - Waveforms should appear normal, with proper amplitude, and without distortion or excessive noise.
 - Paper should stop with perforations near the tear bar, indicating proper cue-sensor operation. For tear-bar location, see [Figure 17](#) on page 21.

Recharging a Fully Discharged Battery

If the electrocardiograph does not turn on when unplugged, the battery may be fully discharged.

Note Regardless of the battery condition, you can use the electrocardiograph whenever it is plugged in.

To Recharge the Battery

1. Plug the electrocardiograph into AC power.
2. Verify that the green LED on the keyboard lights up. (See [Figure 18](#) on page 22.)
If the LED does not light up, go to [“Replacing the AC Fuses”](#) on page 80.
3. Keep the electrocardiograph plugged in for 12 hours.

The first time you turn the electrocardiograph on after a full battery discharge, you are prompted to reenter the date and time.

If the electrocardiograph still does not turn on when it is unplugged from AC power, you may need to replace the battery or battery fuse. See [“Replacing the Battery”](#) on page 77 or [“Replacing the Battery \(DC\) Fuse”](#) on page 79.

Replacing the Battery

If you have recharged the battery and the electrocardiograph still does not turn on when unplugged, or if the battery loses its charge quickly, replace the battery as follows. (For part number, see [“Accessories”](#) on page 5.)

1. Unplug the electrocardiograph from AC power if connected.
2. Turn the electrocardiograph upside-down.
3. Unscrew and remove the battery door. See [Figure 46](#).
4. Lift out the battery. See [Figure 47](#).

Figure 46. Removing the Battery Door



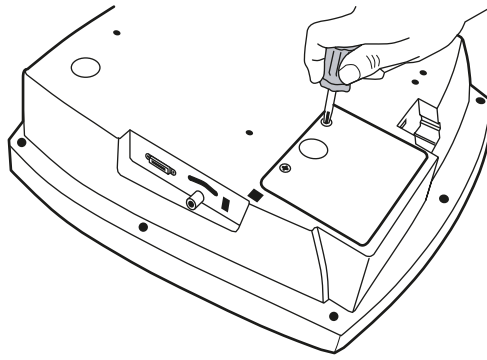


Figure 47. Lifting Out the Battery

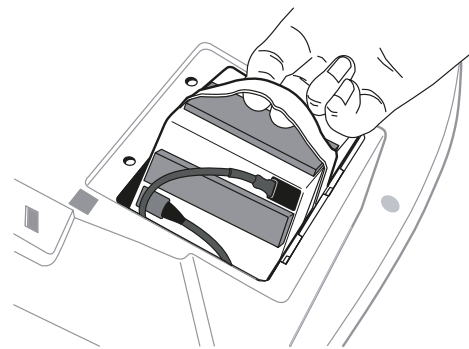


5. Inspect the fuse. See [Figure 48](#).
 - If the “Z” wire is intact, go to [Step 6](#).
 - If the “Z” wire is broken or dark, replace the fuse. See [“Replacing the Battery \(DC\) Fuse”](#) on page 79.

Figure 48. Battery Fuse Plugged In

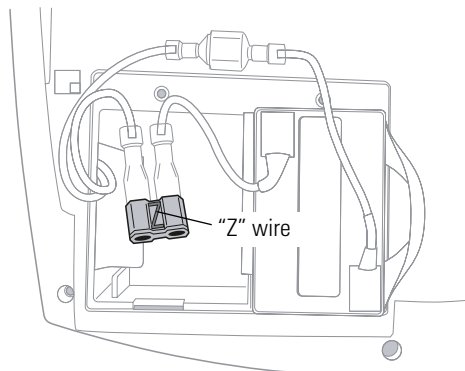


6. Unplug the two battery connectors. See [Figure 49](#).
You may need to pull hard. Use pliers gently if necessary.

Figure 49. Battery Connectors

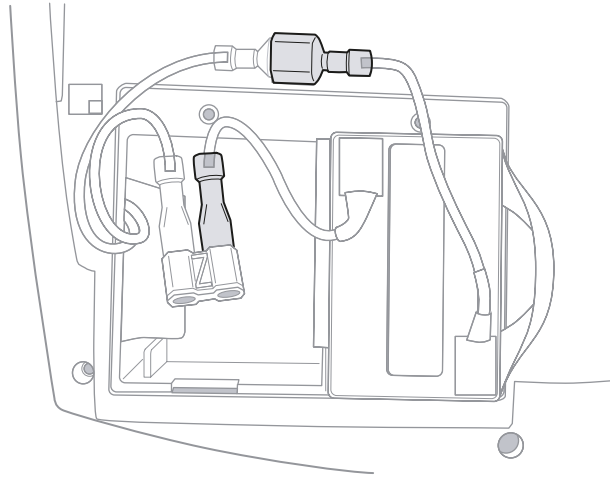

7. Connect a new battery by matching connector sizes. Do not force connectors to mismatch.
8. Carefully tuck in all cables. Place the battery into the battery compartment. Fold the strap so that it will not stick out when you replace the battery door. See [Figure 50](#).

Figure 50. Folding the Battery Strap

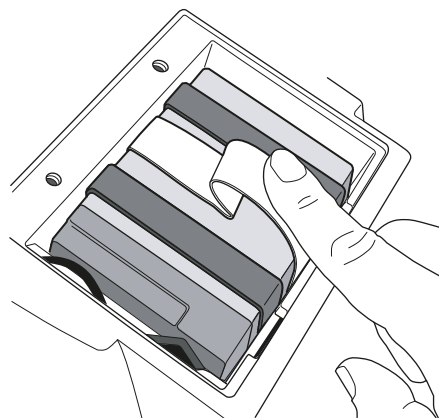


9. Replace the battery door and screws. Turn the electrocardiograph rightside-up. It turns on automatically and displays a prompt to enter date and time.
10. Enter the date and time.
The electrocardiograph is ready to use.
11. Discard the old battery appropriately.
 - In the USA, call 1-800-SAV-LEAD for instructions on how to recycle it.
 - International users, contact your local authorities concerning recycling.

Replacing the Battery (DC) Fuse

If the battery (DC) fuse requires replacing, as described in [Step 5](#) on page 77, follow these steps. For the fuse value, see ["Fuses"](#) on page 89.

1. Remove and discard the fuse. See [Figure 51](#).
You may need to pull hard. Use pliers gently if necessary.
2. Connect a new fuse. It goes in either way.
3. Go to [Step 8](#) on page 78.

Figure 51. Battery Fuse Removed

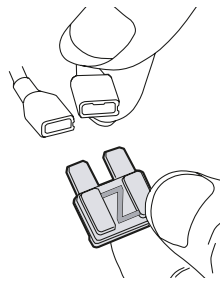

Replacing the AC Fuses

If the green LED on the keyboard does not light up when the electrocardiograph is connected to AC power, you may need to replace one or both of the AC fuses, as follows. For the fuse value, see “Fuses” on page 89.

1. Unplug the electrocardiograph from AC power if connected.



WARNING Failure to unplug could result in electrocution.

2. Use needle-nosed pliers to remove the fuse case. See [Figure 52](#).
3. Inspect the fuses. If either fuse is dark or has a broken wire, replace the fuse. See [Figure 53](#).
4. Insert the fuse case. Line it up with the opening; it goes in only one way.

Figure 52. Removing AC Fuse Case


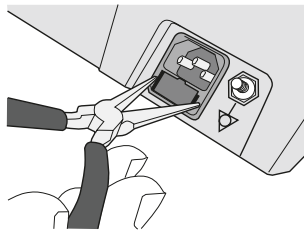


Figure 53. AC Fuses Removed

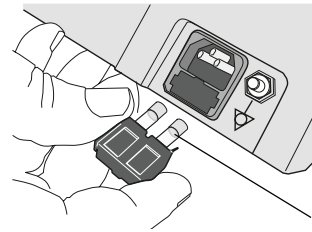

Storing the Equipment

When storing the electrocardiograph, cords, and accessories, observe the environmental storage conditions. See “Specifications” on page 89.

Discarding the Equipment

Discard the old battery appropriately.



- In the USA, call 1-800-SAV-LEAD for instructions on recycling it.
- International users, contact your local authorities concerning recycling.

Discard the electrocardiograph, cords, and accessories according to local laws.



Do not dispose of this product as unsorted municipal waste. Prepare this product for reuse or separate collection as specified by Directive 2002/96/EC of the European Parliament and the Council of the European Union on Waste Electronic and Electrical Equipment (WEEE). If this product is contaminated, this directive does not apply. For more specific disposal information, see www.welchallyn.com/weee, or contact Welch Allyn Customer Service at +44 207 365 6780.

8

Troubleshooting

Problem-Solving Suggestions	84
Limited Warranty	87
Service Policy	88

Problem-Solving Suggestions

This section includes several tables:

- Lead quality problems ([Table 2](#) on page 84)
- System failure problems ([Table 3](#) on page 85)
- System messages ([Table 4](#) on page 86)

If you try these suggestions and still have problems, contact Technical Support. For phone numbers, see [page ii](#).

Table 2. Lead Quality Problems


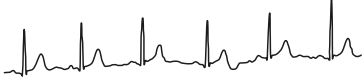
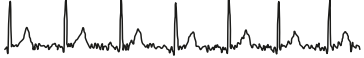
Condition	Causes	Actions
<p>A red dot is flashing on the Lead Off screen.</p> <p>OR</p> <p>Lead-off information is displayed on the screen.</p> <p>OR</p> <p>One or more leads prints as a square wave:</p> 	<ul style="list-style-type: none"> • Electrode contact may be poor. • A lead may be loose. <p>Note: Square waves may indicate that the electrocardiograph is inoperative, but they are more likely to indicate loose leads.</p>	<ul style="list-style-type: none"> • Reattach the lead. • Replace the electrode. • Verify that the electrode area has been properly prepared: shaved, cleaned with alcohol or acetone, allowed to dry. • Verify that electrodes have been properly stored and handled.
<p>Wandering baseline (an upward and downward fluctuation of the waveforms):</p> 	<ul style="list-style-type: none"> • Electrodes that are dirty, corroded, loose, or positioned on a bony area. • Insufficient or dried electrode gel. • Oily skin or body lotions. • Rising and falling of chest during rapid or apprehensive breathing. 	<ul style="list-style-type: none"> • Clean skin with alcohol or acetone. • Reposition or replace electrodes. • Help patient relax. • If wandering baseline persists, turn the baseline filter on. See "Baseline Filter" on page 61.
<p>Muscle tremor interference (random, irregular voltage superimposed on the waveforms). May resemble or coincide with AC interference:</p> 	<ul style="list-style-type: none"> • Patient is uncomfortable, tense, nervous. • Patient is cold and shivering. • Exam bed is too narrow or short to comfortably support arms and legs. • Arm or leg electrode straps are too tight. 	<ul style="list-style-type: none"> • Help patient get comfortable. • Check all electrode contacts. • If interference persists, turn the muscle-tremor filter on. See "Muscle Filter" on page 61. • If interference still persists, the problem is probably electrical in nature. See the following suggestions for reducing AC interference.










Table 2. Lead Quality Problems (Continued)

Condition	Causes	Actions
AC interference (even-peaked, regular voltage superimposed on the waveforms). May resemble or coincide with muscle-tremor interference.	<ul style="list-style-type: none"> • Electrodes that are dirty, corroded, loose, or positioned on a bony area. • Insufficient or dried electrode gel. • Patient or technician touching an electrode during recording. • Patient touching any metal parts of an exam table or bed. • Broken lead wire, patient cable, or power cord. • Electrical devices in the immediate area, lighting, concealed wiring in walls or floors. • Improperly grounded electrical outlet. • Incorrect AC filter frequency setting or AC filter is turned off. 	<ul style="list-style-type: none"> • Check all electrode contacts and lead wires. • Verify that the patient is not touching any metal. • Verify that the AC power cable is not touching the patient lead cable. • Verify that the proper AC filter is selected. See “Mains Filter” on page 43. • If interference persists, unplug the electrocardiograph from AC power and run it on the battery. If this solves the problem, you’ll know that the noise was introduced through the power line. • If interference still persists, the noise may be caused by other equipment in the room or by poorly grounded power lines. Try moving to another room.

Table 3. System Failure Problems

Condition	Causes	Actions
Won't turn on when plugged into AC power.	<ul style="list-style-type: none"> • Faulty AC power connection. • Blown AC fuses. • No AC power. 	<ul style="list-style-type: none"> • Check the AC power source. • Check the AC fuses. See “Replacing the AC Fuses” on page 80.
Won't turn on when unplugged from AC power.	<ul style="list-style-type: none"> • Battery disconnected or incorrectly connected. • Battery low, not charging, depleted, or bad. • Blown battery fuse. 	<ul style="list-style-type: none"> • Check battery connections. See “Replacing the Battery” on page 77. • Recharge the battery. See “Recharging a Fully Discharged Battery” on page 76. • Replace battery. See “Replacing the Battery” on page 77. • Replace battery fuse. See “Replacing the Battery (DC) Fuse” on page 79.
Shuts down during printing	Battery low or bad.	<ul style="list-style-type: none"> • Recharge the battery. See “Recharging a Fully Discharged Battery” on page 76. • Replace battery. See “Replacing the Battery” on page 77.
Prints fewer than 10 reports on a full battery charge.	Degraded battery.	Replace battery. See “Replacing the Battery” on page 77.

Table 4. System Messages (alphabetical order)

System Message	Problems	Actions
"A test is being printed that has not been saved and will be lost. Continue shutdown?"	 was pressed while printing an Auto ECG.	To shut down without saving the test, press  . To cancel the shutdown, press  .
"Audit Trail is too large. Please print and purge."	Audit information is approaching its maximum allotted storage.	Print and purge the audit trail. See "Working With the Audit Trail" on page 72.
"Insufficient battery power to begin. Please connect AC power and try again."	Battery low or bad.	<ul style="list-style-type: none"> Recharge the battery. See "Recharging a Fully Discharged Battery" on page 76. Replace battery. See "Replacing the Battery" on page 77.
"Insufficient space available"	Not enough space on memory card.	<ul style="list-style-type: none"> Delete some tests from the card at a PC. Use a different card.
"Memory card error"	Problem writing to memory card.	<ul style="list-style-type: none"> Verify that the write-protect tab is in the unprotected position. Reseat the card in its slot. Use a different card.
"Out Of Paper"	<ul style="list-style-type: none"> Printer is out of paper. Printer door is open. 	<ul style="list-style-type: none"> Load paper. See "Loading the Thermal Chart Paper" on page 21. Close the printer door. See "Loading the Thermal Chart Paper" on page 21.
"Paper Error"	Paper was loaded incorrectly.	Reload the paper. See "Loading the Thermal Chart Paper" on page 21.
"Powering down"	Low battery.	Recharge the battery. See "Recharging a Fully Discharged Battery" on page 76.
"Problem loading the following settings: <System> <ECG> * * * Using default settings."	Problem loading your settings at startup. May indicate memory problems.	Contact Technical Support. For phone numbers, see page ii .
"Shutdown?"	 was pressed while printing a Rhythm ECG.	To shut down, press  . To cancel the shutdown, press  .
"Temperature Error"	Printer head temp is too high.	Allow to cool, then try again.
"Test has not been saved. Continue shutdown?"	 was pressed after printing an Auto ECG.	To shut down without saving the test, press  . To cancel the shutdown, press  .
"Unable to communicate with workstation"	<ul style="list-style-type: none"> Configuration not correct. Cable not properly connected. 	<ul style="list-style-type: none"> Verify communication settings on the PC. Check cable connections.

Limited Warranty

Welch Allyn, Inc., warrants that the Cardiopulmonary line of electrocardiographs, including the CP 100 and CP 200 models (the Products) meet the labeled specifications of the Products and will be free from defects in materials and workmanship that occur within 3 years after the date of purchase, except that accessories used with the Products are warranted for 90 days after the date of purchase. Such accessories include: lead wires, cabling, electrodes, and battery.

The date of purchase is: 1) the date specified in our records, if you purchased the Product directly from us, 2) the date specified in the warranty registration card that we ask you to send to us, or 3) if you don't return the warranty registration card, 30 days after the date on which the Product was sold to the dealer from whom you bought the Product, as documented in our records.

This warranty does not cover damage caused by: 1) handling during shipping, 2) use or maintenance contrary to labeled instructions, 3) alteration or repair by anyone not authorized by Welch Allyn, and 4) accidents.

If a Product or accessory covered by this warranty is determined to be defective because of defective materials, components, or workmanship, and the warranty claim is made within the warranty period described above, Welch Allyn will, at its discretion, repair or replace the defective Product or accessory free of charge. If your Product requires repairs covered by this warranty, upon your request Welch Allyn will loan to you, at no cost, a substitute Product for use until your repaired Product is returned.

You must obtain a return authorization from Welch Allyn to return your Product before you send it to Welch Allyn's designated service center for repair. Contact Welch Allyn Technical Support. For phone numbers, see [page ii](#).

THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. WELCH ALLYN'S OBLIGATION UNDER THIS WARRANTY IS LIMITED TO REPAIR OR REPLACEMENT OF PRODUCTS CONTAINING A DEFECT. WELCH ALLYN IS NOT RESPONSIBLE FOR ANY INDIRECT OR CONSEQUENTIAL DAMAGES RESULTING FROM A PRODUCT DEFECT COVERED BY THE WARRANTY.

Service Policy

All repairs on products under warranty must be performed or approved by Welch Allyn. Unauthorized repairs will void the warranty. In addition, whether or not covered under warranty, any product repair shall exclusively be performed by Welch Allyn certified service personnel.

If the product fails to function properly—or if you need assistance, service, or spare parts—contact the nearest Welch Allyn Technical Support Center. For phone numbers, see [page ii](#).

Before contacting Welch Allyn, try to duplicate the problem, and check all accessories to ensure that they are not causing the problem. When calling, please be prepared to provide:

- Product name and model number and complete description of the problem.
- Serial number of your product (if applicable).
- Complete name, address and phone number of your facility.
- For out-of-warranty repairs or spare parts orders, a purchase order (or credit card) number.
- For parts orders, the required spare or replacement part numbers.

If your product requires warranty, extended warranty, or non-warranty repair service, please call first the nearest Welch Allyn Technical Support Center. A representative will assist you troubleshooting the problem and will make every effort to solve it over the phone, avoiding potential unnecessary returns.

In case a return cannot be avoided, the representative will record all necessary information and will provide a Return Material Authorization (RMA) number, as well as the appropriate return address. An RMA number must be obtained prior to any return.


If you have to return goods for service, follow these recommended packing instructions:

- Remove all hoses, cables, sensors, power cords, and ancillary products (as appropriate) before packing, unless you suspect they are associated with the problem.
- Wherever possible use the original shipping carton and packing materials.
- Include a packing list and the Welch Allyn Return Material Authorization (RMA) number.

It is recommended that all returned goods be insured. Claims for loss or damage to the product must be initiated by the sender.

A

Specifications

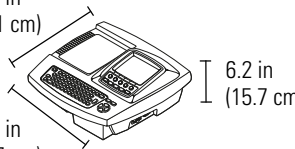Item	Specification
Dimensions	16.2 in (41.1 cm)  15.6 in (39.7 cm) 6.2 in (15.7 cm)
Weight	11.6 lb (5.3 kg)
Keyboard type	Elastomer keypad with complete alphanumeric keys Soft-key menu and dedicated function keys
Paper type	8.25 x 11 inches (21 x 28 cm), Z-fold thermal paper, 200 sheets
Thermal printer (internal)	Computer-controlled dot array, 8 dots/mm
Thermal paper speeds	10, 25, 50 mm/s
Gain settings:	
Auto ECGs	5, 10, 20 mm/mV, AUTO
Rhythm ECGs	5, 10, 20 mm/mV
Report print formats:	
Auto ECGs	3x4, 3x4+ 1R, 3x4 +3R, 6x2, 12x1, 6x2 50 mm/s, 6x2 Ext.
Rhythm ECGs	3, 6, or 12 leads at a time
Average cycles	3x4 50 mm/s + 3R, 6x2 50 mm/s + 6R
ECG storage	Up to 50 ECGs
Frequency range	0.3 to 150 Hz
Digital sampling rate	> 1,000 samples/second/channel
Pacemaker detection	ANSI/AAMI EC11
Power requirement	Universal AC power supply 100–240 V ~, 50/60 Hz ~, 2.2 A maximum
Fuses	
AC	Time-delay type, 2 amp 250 V rating, Littlefuse 0215002 or equivalent.
DC (battery fuse)	Fast-acting type, 10 amp 32 V rating, Bussman ATC-10 or equivalent.
Lead configurations	Standard, Cabrera
Rechargeable battery	Lead acid gel, 6 volt, 5 AH Prints up to 100 continuous ECGs per charge 12-hour recharging

Item (Continued)	Specification (Continued)	
Filters	0.5 Hz high-performance baseline filter 35 Hz muscle-tremor filter AC-interference filter 50 Hz or 60 Hz	
Safety, EMC, and regulatory compliance	ANSI/AAMI EC11* CAN/CSA C22.2 No. 601.1 CAN/CSA C22.2 No. 601.1.1 CAN/CSA C22.2 No. 601.1.2 CAN/CSA C22.2 No. 601.1.4 CAN/CSA C22.2 No. 601.2.25	UL60601-1 IEC/EN 60601-1 IEC/EN 60601-1-1 IEC/EN 60601-1-2 IEC/EN 60601-1-4 IEC/EN 60601-2-25
Standard connectivity	Com port, SD memory card slot (for use with cards \geq 64 MB)	
Connectivity with electronic medical records	Supported through the Welch Allyn CardioPerfect workstation software	
Electrodes	Rigorously tested for conductivity, adhesion, and hypoallergenic qualities, and exceed all AAMI standards	
Cables and wires	Meet or exceed IEC 60601	
Environmental operating conditions:		
Temperature	+10° C to +40° C (+50° F to +104° F)	
Relative humidity	15 – 95% noncondensing	
Altitude	700 – 1060 hPa	
Environmental storage conditions:		
Temperature	-20° C to +49° C (-4° F to +120° F)	
Relative humidity	15 – 95% noncondensing (30 – 70% for printing)	
Altitude	500 – 1060 hPa	
Protection against electric shock	Class I, internally powered Type BF	
Protection against ingress of water, per IEC 60529	IPX0	
Mode of operation	Continuous	

*Per AAMI EC11:1991/(R)2001 Diagnostic Electrocardiographic Devices, Section 3.1.2.1 Disclosure of cautionary information/ performance characteristics paragraph c) Accuracy of input signal reproduction, the manufacturer shall disclose the methods used to establish overall system error and frequency response. Welch Allyn has used methods A & D, as prescribed in section 3.2.7.2 and 4.2.7.2 of this same standard, to verify overall system error and frequency response. Because of the sampling characteristics and the asynchronism between sample rate and signal rate, digital ECG systems such as the CP 100 and CP 200 may produce a noticeable modulating effect from one cycle to the next, particularly in pediatric recordings. This phenomenon is not physiologic.

Specifications are subject to change without notice.



EMC Guidance and Manufacturer's Declarations

Table 5. Electromagnetic Emissions

The CP 200 electrocardiograph is intended for use in the electromagnetic environment specified below. The customer or user of the CP 200 electrocardiograph should assure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment - Guidance
RF emissions CISPR 11	Group 1	The CP 200 electrocardiograph uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	


Table 6. Electromagnetic Immunity

The CP 200 electrocardiograph is intended for use in the electromagnetic environment specified below. The customer or user of the CP 200 electrocardiograph should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions, and voltage variations on power supply input lines. IEC 61000-4-11	>95% dip in 0.5 cycle 60% dip in 5 cycles 30% dip for 25 cycles >95% dip in 5 seconds	>95% dip in 0.5 cycle 60% dip in 5 cycles 30% dip for 25 cycles >95% dip in 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the CP 200 electrocardiograph requires continued operation during power mains interruptions, it is recommended that the CP 200 electrocardiograph be powered from an uninterruptible power supply or battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Table 7. Electromagnetic Immunity

The CP 200 electrocardiograph is intended for use in the electromagnetic environment specified below. The customer or user of the CP 200 electrocardiograph should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2.5 GHz</p>	<p>3 Vrms</p> <p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the CP 200 electrocardiograph, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> <p>$d = (1.17) \sqrt{P}$</p> <p>$d = (1.17) \sqrt{P}$ 80 MHz to 800 MHz</p> <p>$d = (2.33) \sqrt{P}$ 800 MHz to 2.5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the CP 200 electrocardiograph is used exceeds the applicable RF compliance level above, the electrocardiograph should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the electrocardiograph.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Table 8. Recommended Separation Distances Between Portable and Mobile RF Communications Equipment and the CP 200 Electrocardiograph

The CP 200 electrocardiograph is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the CP 200 electrocardiograph can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the CP 200 electrocardiograph as recommended below, according to the maximum output power of the communications equipment.

Rated Max. Output Power of Transmitter (W)	Separation Distance According to Frequency of Transmitter (m)		
	150 kHz to 80 MHz $d = (1.17) \sqrt{P}$	80 MHz to 800 MHz $d = (1.17) \sqrt{P}$	800 MHz to 2.5 GHz $d = (2.33) \sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.70	3.70	7.37
100	11.70	11.70	23.30

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Glossary

AHA. An abbreviation for *American Heart Association*. The AHA electrode labeling scheme is most commonly used in the United States.

audit trail. A collection of information about user activity. The audit trail, which can be enabled or disabled, may be helpful or even required for record-keeping.

Auto ECG. A report typically showing a 10-second acquisition of 12 leads of ECG information combined with patient data, interpretation, and measurements matrix. Two user-defined formats are available: Auto Report 1 and Auto Report 2. See also [Rhythm ECG](#).

Auto Save. A feature that automatically saves all Auto ECGs (except *stat* ECGs) to the test directory.

Auto Send. A feature that automatically sends all Auto ECGs (except *stat* ECGs) to the option of your choice: SD memory card or CardioPerfect workstation.

average cycles. Dominant waveforms for all 12 leads. Printed on a separate page, if enabled.

CardioPerfect workstation. A PC using Welch Allyn CardioPerfect software. Stores ECG and spirometry test data. Can communicate with other electronic patient-information systems, such as billing and medical records.

confirmed, unconfirmed. Status of an Auto ECG report's interpretation. Indicates whether the report has been reviewed and accepted by a qualified physician.

electrocardiogram (ECG). A record of the electrical currents associated with heart muscle activity. Sometimes called *EKG* for the German term, *Elektrokardiogramm*. See also [Auto ECG](#); [Rhythm ECG](#).

electrocardiograph. An instrument used to create electrocardiograms (ECGs).

extended measurements. Values for several common parameters, such as Q, R, and S amplitude and ST values. The amplitudes are expressed in microvolts. The durations are expressed in milliseconds. The measurements cannot be edited. Printed on a separate page, if enabled.

IEC. An abbreviation for *International Electrotechnical Commission*. The IEC electrode labeling scheme is most commonly used in Europe.

lead. (1) An electrocardiograph wire connected to an electrode, which is attached to a patient's skin. There are 10 lead wires. (2) A waveform that represents ECG data from a particular view of the heart. The data from the 10 lead wires is converted into 12 waveforms, also called leads: I, II, III, aVR, etc.

MEANS. An acronym for *modular ECG analysis system*. The optional MEANS interpretation algorithm, developed by the University of Rotterdam, Netherlands, provides automatic analysis of ECG tests.

patient list. See [scheduled patients list](#).

Power-Save. A user-selectable feature that automatically turns the electrocardiograph off after several idle minutes.

Rhythm ECG. A continuous, real-time printout of ECG data. See also [Auto ECG](#).

scheduled patients list. A list of patients whose data has been entered into the electrocardiograph's memory for an ECG or spirometry test that day. This list can hold up to 40 patients. At midnight every night, the list is automatically cleared. When performing Auto ECG or spirometry tests, you may select patients from this list rather than manually entering their data at the time of the test.

test directory. A location in the electrocardiograph's memory that holds up to 50 ECG and 50 spirometry tests. When performing Auto ECG or spirometry tests, you may select patients from this directory rather than manually entering their data at the time of the test.

workstation. See [CardioPerfect workstation](#).

Index

3x4, 6x2, 12x1, etc. *See* leads: arrangements (formats) in
 Auto Reports
 3x4 50 mm/s + 3R, 6x2 50 mm/s + 6R. *See* average cycles
 5 mm/mV, 10 mm/mV, etc. *See* gain settings
 50 or 60 Hz filter. *See* AC filter

A

"Abnormal ECG" label, 39
 "About" screen (device information), 29
 accessories, 5, 17, 87
 AC filter, 35, 36, 43, 85
 AC fuses, replacing, 80
 AC interference, 84, 85
 AC power. *See* powering the electrocardiograph
 acquisition screen, 51
 administrative tasks, 63–72
 AHA electrode labels (V1, V2, V3, etc.), 43, 47
 arrow keys, using, 11
 audio beeper, 27
 audit trail, 69, 72
 authorized-user list, 69, 70
 "Auto ECG Acquisition" screen, 51
 "Auto ECG Post-Print" screen, 52
 Auto ECGs
 recording, 49–55
 report settings, 35–41
 saved in test directory, 64–67
 automatic interpretation. *See* interpretation of ECGs
 automatic shutdown. *See* Power-Save
 Auto Report 2, enabling, 44
 Auto Save, 43, 55
 Auto Send, 44, 55
 average cycles, 38, 89

B

baseline centering, 43
 baseline filter
 default, choosing (on or off), 43
 enabling and disabling, 61

battery
 maintenance, 22, 76–79
 ordering, 5
 specifications, 89
 beeper, 27
 buttons. *See* function keys; keyboard; softkeys

C

C1, C2, C3, etc. *See* IEC electrode labels
 Cabrera lead-grouping scheme, 43, 89
 CardioPerfect workstation
 about, 2, 3
 connectivity kit, 5
 problems communicating with, 86
 sending tests to, 44, 52, 65
 carts, 4, 5, 15
 caution symbol defined, 13
 CD (product information), ordering replacement, 5
 cleaning procedures, 74
 clinical conditions. *See* history list
 Com ports, locations, 7
 confidentiality, 69
 configuration. *See* ECG settings; system settings
 confirming. *See* interpretation of ECGs, automatic: confirming
 connectivity kit for workstation, 5
 contact information, ii
 contraindications, 3
 copies, number to print, 39
 Customer Service, ii

D

database. *See* CardioPerfect workstation; scheduled patients
 list; test directory (saved tests)
 data. *See* patient data
 date and time, 27, 35, 36
 defibrillation precautions, 16, 17, 20
 demographics. *See* patient data
 "Device Administration" menu tree, 69
 device configuration settings, 27
 device ID, 27, 35, 36
 "Device Info" screen, 29

digital sampling rate, 89
dust cover, ordering, 5

E

ECG acquisition screen, 51
ECG post-print screen, 52
"ECG Preview" screen, 48
ECGs
 performing tests, 45–61
 settings, 10, 33–44
 See also Auto ECGs; Rhythm ECGs
editing
 history list, 31
 medication list, 30
 "miscellaneous" ECG settings, 43
 report formats, 35–42
 scheduled patient list, 68
 tests, 52, 54, 64, 66–67
 tracking (auditing) edits made, 72
 user list, 71
electrodes
 labeling schemes. *See* AHA; IEC
 ordering, 5
 placement locations, 47
 specifications, 90
 troubleshooting, 84
electromagnetic emissions. *See* EMC guidance and declarations
electronic medical records (EMR), 2
EMC guidance and declarations, 91–94
"Enter New Patient" screen, 50
equipotential stud, 6, 17
exporting tests. *See* sending tests
extended measurements, 37

F

features, 3
filters, types of, 90
 See also specific filter names
flow (task) diagrams
 Auto ECG testing (normal and stat), 49
 entering or searching for patient data, 57
 rhythm ECG testing, 56
formats. *See* Auto ECGs: report settings; Rhythm ECGs:
 report settings
frequency range, 35, 36, 89
function keys, 9
fuses, 77, 79, 80

G

gain settings
 changing, 61
 default, choosing, 43
 as printed on report, 35, 36

H

hardware set-up, 19–23
height-unit setting, 27
help, getting, 18
history list
 choosing from, 41
 editing, 10, 31

I

IEC electrode labels (C1, C2, C3, etc.), 43, 47
indications for use, 3
inspection procedures, 74
intended use, 2
Internet address, ii
interpretation of ECGs, automatic
 about, 2, 3, 4
 confirming, 39, 54, 66
 editing, 54, 66
 ordering, 5
 as printed on reports, 35, 36
 settings for Auto Reports, 39

K

keyboard, 8, 89
keys. *See* function keys; keyboard; softkeys

L

language setting, 27
"Lead Off" screen, 48
leads
 10 lead wires vs. 12 waveform leads, 95
 adjusting waveforms, 61
 arrangements in reports, 37–38, 42
 configurations (standard or Cabrera), 43, 89
 ordering, 5
 print timing (simultaneous or sequential), 43
 troubleshooting problems with, 84
 See also patient cable; electrodes
login, 69

M

main menu, 10
 mains. *See* AC
 maintenance tasks, 73–81
 Manual Save, 52
 Manual Send, 52
 measurements. *See* extended measurements
 medication list
 choosing from, 41
 editing, 10, 30
 memory card
 for communication with CardioPerfect workstation, 3
 problems writing to, 86
 sending tests to, 44, 52, 65
 slot location, 7
 specification, 90
 menus, navigating through, 11
 menu trees
 device administration, 69
 ECG setup, 34
 system setup, 26
 test directory, 64
 “miscellaneous” ECG settings, 43–44
 muscle filter, 43, 61
 muscle-tremor interference, 84, 85

N

navigating through the menus, 11

O

OK key, 9
 on/off, manual and automatic, 23
 options, 4
 ordering information, 5

P

pacemakers, use with, 17, 89
 paper. *See* thermal chart paper
 parts and accessories, 5, 17, 87
 passwords, 70, 71
 patient cable
 connecting, 20, 46–48
 description & illustration, 12
 ordering, 5
 precautions, 17
 See also electrodes; leads

patient data
 editing, 54
 enabling and disabling fields, 40
 entering, 50
 as printed on reports, 35, 36
 searching for, 57–60
 See also CardioPerfect workstation; scheduled patients list
 phone numbers, ii
 poster, ordering replacement, 5
 post-print screen, 52
 powering the electrocardiograph, 22–23, 89
 Power-Save, 23, 27
 preview screen, 48
 printer, internal (thermal), 75, 89
 print saved tests, 65
 “Print Settings” (printing your settings), 29
 print settings (report settings). *See* Auto ECGs: report settings;
 Rhythm ECGs: report settings
 privacy, 69
 pulmonary function device. *See* spirometry option

R

reason statements, 39
 recycling instructions, 81
 repairs, 88
 replacement parts and accessories, 5, 87
 reports. *See* Auto ECGs; Rhythm ECGs
 Rhythm ECGs
 recording, 56
 report settings, 42

S

safety, 15–17
 sampling rate, digital, 89
 saving tests. *See* Auto Save; Manual Save; test directory
 scheduled patients list, 10, 60, 68
 SD memory card. *See* memory card
 searching
 for patient data, 57–60
 for saved tests, 65
 security of data, 69
 sending (exporting) tests. *See* Auto Send; Manual Send; test
 directory
 sequential lead timing, 43
 serial number, displaying, 29
 service policy, 88
 set-up. *See* hardware set-up; ECG settings; system settings
 simultaneous lead timing, 43
 softkeys, 9
 specifications, 89–90
 speed settings. *See* thermal chart paper: speed settings

- spirometry option
 - about, 2, 3, 4
 - ordering, 5
 - port location, 7
 - tests saved in test directory, 64
- stat Auto ECGs, 49, 55
- Stop/Cancel key, 9
- storage capacity for ECGs, 89
- storage instructions, 81
- ST segment analysis, 51
- supplies, 5, 87
- symbols and markings, 13–14
- system settings, 10, 25–29

T

- Technical Support, ii
- test directory (saved tests), 10, 64–66
- testing the electrocardiograph, 75
- thermal chart paper
 - loading, 21
 - ordering, 5
 - specifications, 89
 - speed settings, 35, 36, 61
- time and date. *See* date and time
- timing, lead, 43
- troubleshooting, 83–86

U

- ultrasound machines, use with, 17
- “Unconfirmed” label, 35, 36, 39
 - See also* interpretation of ECGs, automatic: confirming
- upgrades, 4, 29
- URL, ii
- USB connection (Com port B), 7
 - See also* CardioPerfect workstation
- user authorization, 69, 70

V

- V1, V2, V3, etc. *See* AHA electrode labels
- versions, 29, 35, 36

W

- wall poster, ordering replacement, 5
- warning symbol defined, 13
- warranty, 87
- waveforms, ECG. *See* leads
- web site, ii
- weight-unit setting, 27

- Welch Allyn Technical Support, ii
- Welch cups, ordering, 5
- workstation. *See* CardioPerfect workstation

4341 State Street Road, PO Box 220, Skaneateles Falls, NY 13153-0220 USA
1 800 535 6663, + 1 315 685 4560 www.welchallyn.com

WelchAllyn®

Advancing Frontline Care™

Reorder Number (multi-language CD): 401151
Mat. Number (manual only): 701557, Ver: D