AED 10/AED 20 Pediatric Energy Reducer Brugervejledning

Dansk

Beskrivelse

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AED 10/AED 20 Pediatric Energy Reducer Instructions for Use

English

Description

- The Pediatric Energy Reducer is a reusable adapter which should be stored with the AED 10/AED 20 and used only while the unit is in AED mode (NOT manual mode).
- The energy reducer automatically lowers defibrillation energy to a level considered safe for infants and small children.

Indications for Use

- The energy reducer is indicated for use, with the AED 10/AED 20, on infants and children who are unconscious, not breathing and unresponsive. It is intended for use in either in-hospital or out-of-hospital arrests.
- The pediatric Energy Reducer should be used only on patients who are less than 8 years of age or are under 25 kg (55 lbs).
- It is intended to be used with non-attenuating adult or pediatric pads.
- The Pediatric Energy Reducer is intended to be used only with the AED 10/AED 20 defibrillator.
- It is intended for use by personnel who are authorized by a physical/medical director, and who have the following training and skills:
 - American Heart Association Heartsaver course, American Red Cross CPR/AED course or equivalent training.
 - Training in the use of the AED 10/AED 20.

Equipment Requirements and Instructions for Use

- Determine if the energy reducer should be used based on the patient’s age and weight. Use the energy reducer only if the patient is a child less than 8 years old or less than 25 kg (55 lbs). Do not delay treatment in making this determination.
- Plug the Pediatric Energy Reducer directly into the AED 10/AED 20 patient connector prior to connecting the electrode pads. The adapter can then be connected to either pediatric or adult size electrodes.
- Non-attenuating pediatric electrodes are recommended for infants and children up to 10 kg (22 lbs). However, if only non-attenuating adult pads are available, they can be used on smaller patients by placing the pads on the anterior (front) and posterior (back) sections of the patient’s torso.
- Follow the instructions provided with the pads, for proper placement of the electrodes.
- Operation of the AED 10/AED 20 remains the same, when using the energy reducer. Refer to the AED 10/AED 20 operating instructions for specific details.

General Precautions

- The Pediatric Energy Reducer is not indicated for use on children over 25 kg (55 lbs) or adults.
- The energy reducer must not be used with any manually operated defibrillator because it may create confusion about the appropriate energy to be selected.
- Only use the Pediatric Energy Reducer while the AED 10/AED 20 is in AED mode.
- Store the energy reducer with the AED 10/AED 20, at all times.
- Do not leave the energy reducer connected to the AED 10/AED 20 after use.
- Disconnect the energy reducer if the pads are transferred to a manual defibrillator.
- The Pediatric Energy Reducer should be tested for proper output, according to the same procedure and interval recommended in the AED 10/AED 20 operating instructions. When tested with any commercially available defibrillator analyzer, the energy reducer’s output should be one fourth of the full energy ± 15%.

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