














- Change patient mode _____  SETUP, MORE, CHANGE
- Change ECG1 lead _____  ECG, ECG1 LEAD
- Change RESP lead _____  ECG/RESP, MORE, (NEXT, CHANGE)
- Specify auto or manual NIBP _____  NIBP, AUTO/MANUAL


- Specify waveforms for display _____  SETUP, MORE, WAVE SELECT, (NEXT, ON/OFF)
- Change HR/PR tone volume _____  SETUP, MORE, (NEXT, CHANGE)
- View vital sign trends _____  SETUP, TRENDS, (NEXT TREND)
- Change time/date _____  SETUP, MORE, MORE (NEXT, UP or DOWN)

Alarms and Alerts

- Silence alarm or alert tone for 90 sec or until a new alarm or alert occurs _____  or SILENCE
- Inhibit alarm and alert tones for 4 min _____  SETUP, ALARMS, 4 MIN SUSPND
- Re-enable a silenced alarm or alert tone _____  or RESUME
- Change alarm tone volume _____  SETUP, MORE, (NEXT, CHANGE)
- Change alarm limits _____  SETUP, ALARMS, LIMITS



Propaq® CS Vital Signs Monitor

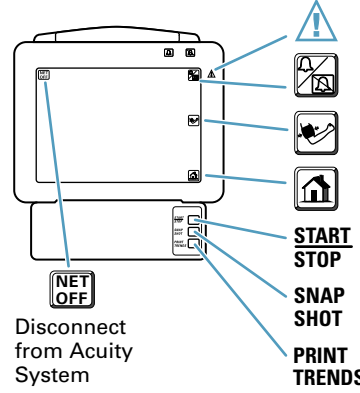





- Consult Directions for Use
- Silence/Re-enable a patient alarm or equipment alert
- Start/Stop an NIBP reading
- Main Menu
- Start/Stop printing
- Print previous eight seconds
- Print specified trends





©2005 - Printed in USA - P/N: 810-1103-02 Rev. A 6/05

Disconnect from Acuity System






800 289 2500 • 503 530 7500 • Beaverton, Oregon USA



- Change patient mode _____  SETUP, MORE, CHANGE
- Change ECG1 lead _____  ECG, ECG1 LEAD
- Change RESP lead _____  ECG/RESP, MORE, (NEXT, CHANGE)
- Specify auto or manual NIBP _____  NIBP, AUTO/MANUAL


- Specify waveforms for display _____  SETUP, MORE, WAVE SELECT, (NEXT, ON/OFF)
- Change HR/PR tone volume _____  SETUP, MORE, (NEXT, CHANGE)
- View vital sign trends _____  SETUP, TRENDS, (NEXT TREND)
- Change time/date _____  SETUP, MORE, MORE (NEXT, UP or DOWN)

Alarms and Alerts

- Silence alarm or alert tone for 90 sec or until a new alarm or alert occurs _____  or SILENCE
- Inhibit alarm and alert tones for 4 min _____  SETUP, ALARMS, 4 MIN SUSPND
- Re-enable a silenced alarm or alert tone _____  or RESUME
- Change alarm tone volume _____  SETUP, MORE, (NEXT, CHANGE)
- Change alarm limits _____  SETUP, ALARMS, LIMITS



Propaq® CS Vital Signs Monitor

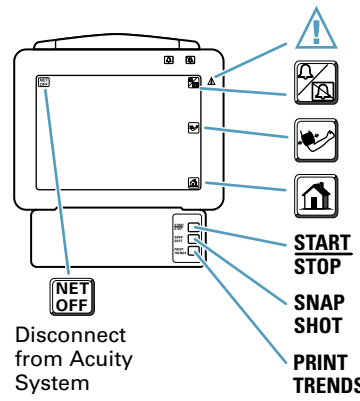

- Consult Directions for Use
- Silence/Re-enable a patient alarm or equipment alert
- Start/Stop an NIBP reading
- Main Menu
- Start/Stop printing
- Print previous eight seconds
- Print specified trends

©2005 - Printed in USA - P/N: 810-1103-02 Rev. A 6/05

Disconnect from Acuity System

800 289 2500 • 503 530 7500 • Beaverton, Oregon USA

