


Propaq® LT Vital Signs Monitor

Quick Reference

Neonate — Birth through 28 days, or 44 weeks gestational
 Pediatric — 29 days through 12 years
 Adult — 13 years and older

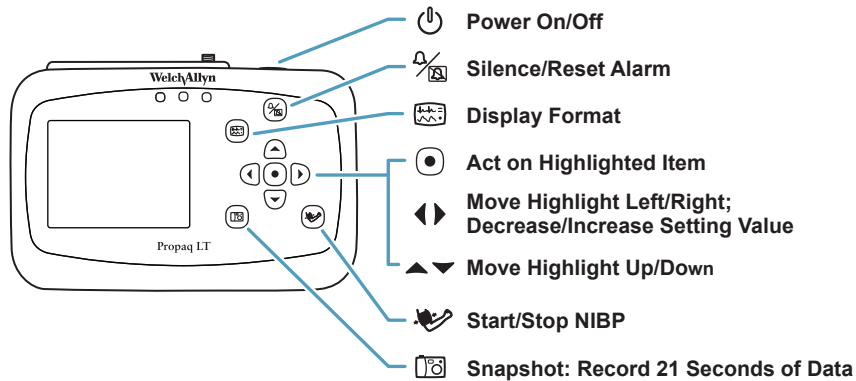

Change Alarm Limits (select) (select parameter) (change limits)
 (HR/PR, NIBP, SpO₂, Resp) _____

Change Alarm Tone (SpO₂) Setup Alarms - Alarm Tone (select)
 (Low, Medium, High) _____

Silence/Reset Alarm Tone
 Suspend Alarm Tones (HR/min) Setup Alarms - Suspend Audible Alarms
 (select)

Change (Vital Sign) Settings (select) (parameter)
 (HR/PR, NIBP, SpO₂, Resp) _____

Change Patient Mode Adult (select)
 (Adult, Pediatric, Neonate) _____

Change Lead (I, II, III, ...) II (select)

Set NIBP mode NIBP mmHg NIBP Mode (select)
 (Auto, Manual, Turbo) _____

Change Monitor Setup (NIBP) Setup (select) (set values)
 (Alarms, ECG, NIBP, ...) _____

Change time/date xx:xx:xx (select) (change)
 (Hour, Minute, Month, ...) _____

View Trends (NIBP) Trends

View Snapshots (HR/min) Snapshots