

# Welch Allyn®

## FlexiPort® Calibration Check Fitting

Directions for Use **Mat'l #714198 Ver. C**

# WelchAllyn®

Advancing Frontline Care™

Welch Allyn USA (Headquarters)  
Tel: 1-315-685-4560 or 1-800-535-6663  
Fax: 1-315-685-0054

Welch Allyn China Tel: (+86) 21 6327 9631  
Welch Allyn Singapore Tel: (+65) 291-0882  
Fax: (+86) 21 6327 9632 Fax: (+65) 291-5780

Welch Allyn Ireland  
(European Customer Service)  
Tel: (353) 46 90 67700  
Fax: (353) 46 90 67755

Welch Allyn U.K. Tel: 0-207-365-6780  
Welch Allyn Australia Tel: (+61) 2-9638-3000  
Fax: 0-207-365-9694 Fax: (+61) 2-9638-3500

Welch Allyn Italy  
Tel: (+39) 02-6968- 2425  
Fax: (+39) 02-6963-3697

Welch Allyn Germany Tel: (+49) 7477 9271 70  
Welch Allyn Japan Tel: (+81) 3-219-0071  
Fax: (+49) 7477 9271 90 Fax: (+81) 3-219-0074

Welch Allyn France  
Tel: (+33) 60 09 33 66 or (+33) 55 69 58 49  
Fax: (+33) 60 09 67 97

Welch Allyn Spain Tel: (+34) 917499357  
Welch Allyn South Africa Tel: (+27) 11 777 7509  
Fax: (+34) 917499361 Fax: (+27) 11 777 7510

Welch Allyn Canada  
Tel: 1-905-890-0004  
Fax: 1-905-890-0008

Welch Allyn Inc. • 4341 State Street Road • Skaneateles Fall, NY 13153 USA  
Tel : 800-535-6663 (USA only) or 315-685-4560 • Fax: 315-685-3361 • www.welchallyn.com

U.S. Patents 6,578,428; 6,036,718; and additional patents pending.

© 2009 Welch Allyn, Inc. All rights reserved. No one is permitted to reproduce or duplicate, in any form, this manual or any part thereof without permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product, that may result from failure to properly use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Welch Allyn® and FlexiPort® are trademarks of Welch Allyn, Inc.

### Warranty

Welch Allyn warrants the FlexiPort Test Fitting, when new, to be free of defects in material and workmanship and to perform in accordance with manufacturer's specifications for a period of one year from the date of purchase from Welch Allyn or its authorized distributors or agents. Welch Allyn will either repair or replace any components found to be defective or at variance from manufacturer's specifications within this time at no cost to the customer. It shall be the purchaser's responsibility to return the FlexiPort Test Fitting to Welch Allyn or an authorized distributor, agent, or service representative. This warranty does not include breakage or failure due to tampering, misuse, neglect, accidents, modification, or shipping. This warranty is also void if the instrument is not used in accordance with manufacturer's recommendations or if repaired by other than Welch Allyn or an authorized agent. Purchase date determines warranty requirements. No other express warranty is given.

### Intended Use

This test fitting is intended to enable connection between FlexiPort fittings on blood pressure devices and calibrated equipment during calibration check procedures.

### Contraindications

This test fitting should not be used during blood pressure readings on patients.

### Warnings

A warning statement in this manual identifies a condition or practice which, if not corrected or discontinued immediately could lead to injury, illness, or death.


### Maintenance

Clean as required.

### Directions For Use

To connect the test fitting with the Flexiport fitting, match up the base of the Flexiport fitting with the base of the test fitting. Gently push together until the fittings click, ensuring a successful connection.

**WARNING:** Ensure an airtight seal at all connection points prior to use.

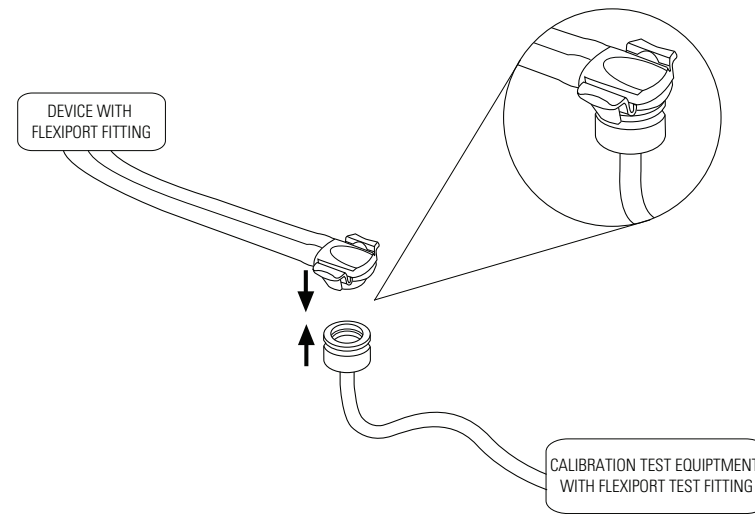

After a calibration test is complete, gently squeeze the side tabs of the port fitting and pull apart to disconnect the fittings.