


WelchAllyn®

767 Diagnostic System


3.5v 767 Wall Transformer
Third Handle Module Mounting Instructions

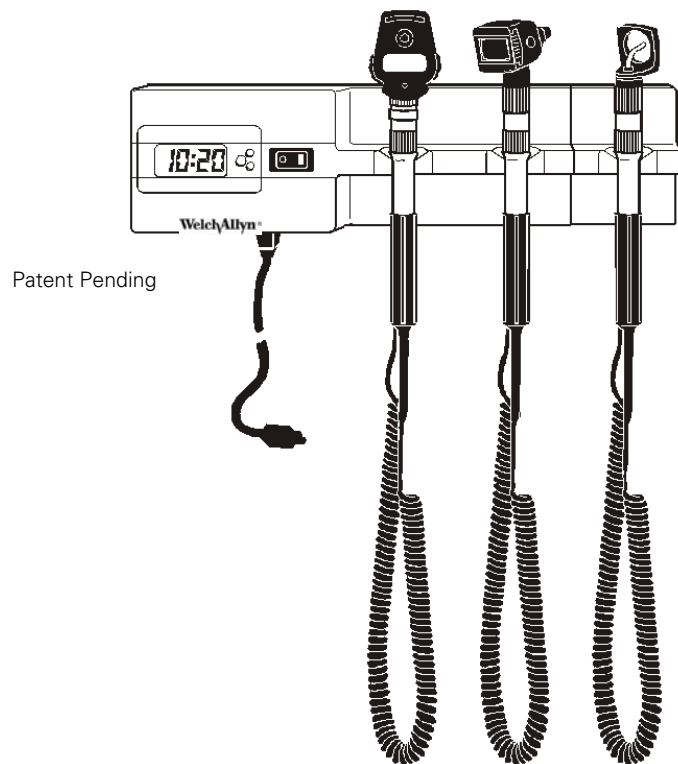


Listed to:
UL 60601-1
CSA C22.2 No. 601

WelchAllyn®


Thank you for purchasing the Welch Allyn 3.5v 767 Third handle Module. This manual provides instructions for mounting the 767 Third Handle Module to the 767 Wall Transformer. The 767 Third Handle carries a lifetime warranty against all manufacturing defects. **The coiled cords carry a special 10 year warranty against breakage during normal usage.**

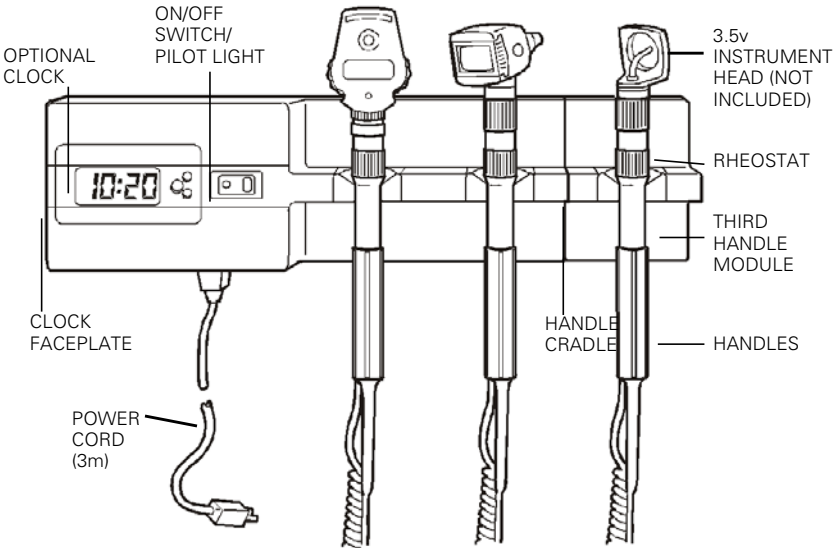


Step 3:

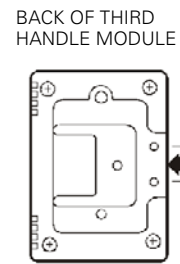


Step 6:

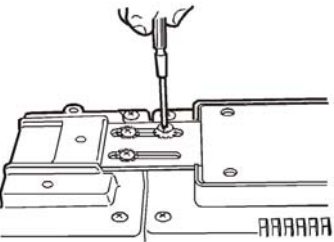


Step 5:

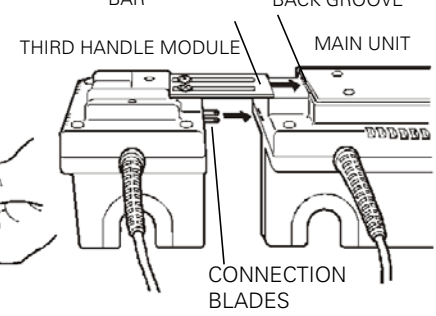

Warnings

 **CAUTION: DISCONNECT THE SYSTEM FROM THE MAINS SUPPLY BEFORE CONNECTING THE THIRD HANDLE (MODEL #76730).**

 **USE THIRD HANDLE MODULE (MODEL #76730) WITH WELCH ALLYN 3.5v 767 WALL TRANSFORMER ONLY.**

Avertissements

 **ATTENTION : DEBRANCHER LE SYSTEME DE L'ALIMENTATION SECTEUR AVANT DE RACCORDER LA TROISIEME POIGNEE (MODELE NO. 76730).**

 **N'UTILISER LE MODULE DE LA TROISIEME POIGNEE (MODELE NO. 6730) QU'AVEC LE TRANSFORMATEUR MURAL WELCH ALLYN 767 DE 3,5V.**

Mounting the Third Handle Module

A Phillips screwdriver is needed to connect the Third Handle Module.

Mounting the Third Handle Module:

1. Disconnect main unit power supply.
2. Remove main unit from the wall by lifting up and out (instructions found on page 2 of this manual).
3. Attach stabilization bar to back of Third Handle Module (as shown using two enclosed star washers and screws).
4. Line up the metal stabilization bar with the groove on the back of the main unit. Also, line up the connecting blades with the blade openings on the side of the main unit housing.
5. Slide Third Handle Module into the main unit until firmly in place.
6. Secure Third Handle Module in place (as shown using two enclosed star washers and screws).
7. Place unit on the wall following the instructions found on page 2 of this manual and reconnect main unit power supply.
8. Place instrument on third handle and lift from cradle. This will test for correct attachment.

WelchAllyn®

Welch Allyn, Inc.
4341 State Street Road
PO Box 220
Skaneateles Falls, NY 13153-0220
U.S.A.
Telephone: 800-535-6663 or
315-685-4100
Fax: 315-685-3361

Part No. 718438 Rev. A

Printed in U.S.A.