## Welch Allyn One Piece Blood Pressure Cuff

723448 Rev. A

### Warnings

**WARNING** Intravenous Systems (IV) - Do not connect cuffs with luer lock connectors to intravenous fluid systems or air may enter patient.

**WARNING** Size - Only use the cuff when the artery index marker falls within the printed range indicated on the cuff, otherwise erroneous readings may result.

**WARNING** Legibility - If artery index or range markings are not visible, do not use the cuff.

### Cleaning:

Before washing the cuff, remove the inflation bulb (if available) and tube fitting(s), close off tube(s) with a plug (available as accessory #5082-163), and place the hook and loop fasteners in the closed position.

#### Durable Cuffs

Clean with a damp cloth (70% alcohol or 0.5% bleach solution may be used) or wash in water (140°F, 60°C maximum) with soap or detergent.

#### Single Patient Cuffs

Clean with a damp cloth or wash in soap or detergent.

#### All One-Piece Cuffs

After washing, allow the cuff to air dry and re-assemble the tube fitting(s).

**Caution** DO NOT PRESS WITH AN IRON.

### Disinfection/Sterilization

#### Durable Cuffs Only

Use glutaraldehyde-type liquid disinfectants on the durable cuffs. Prolonged use of these disinfectants at full strength may cause discoloration of the white cuff markings. Do not use steam or heat to sterilize the cuff, bulb, or tubing. Use gas sterilization, if necessary.

### Warranty

Durable One-Piece Cuffs: Three-year warranty against original defects in material or workmanship. This warranty does not cover breakage or failure due to tampering, misuse, neglect, accidents, modification, or shipping. The warranty is void if the blood pressure cuff is not used in accordance with manufacturer's recommendations or if repaired or serviced by other than Welch Allyn or a Welch Allyn authorized representative.

Welch Allyn Inc. will repair or replace, free of charge, any Welch Allyn Single-Patient Blood Pressure Cuff proven to be defective through causes other than misuse, neglect, damage in shipment, or normal wear. Send all repairs prepaid, to:

Welch Allyn, Inc. Repair Department  
4341 State Street Road  
Skaneateles Falls, NY 13153, USA  
Telephone: 800-535-6663 (USA only) or 315-685-4560

4341 State Street Road  
Skaneateles Falls, NY 13153 USA  
Tel: 800-535-6663 (USA only) or 315-685-4560  
Fax: 315-685-3361

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