

Capturing Rectal Temperature

Welch Allyn SureTemp® Plus Electronic Thermometers Models 692/690




1. Ensure that the probe with the red ejection button and red probe well are installed.
2. Remove the probe from the probe well.
3. Observe the Rectal Mode indicator on the display (flashing lower body icon).



4. Load a Welch Allyn disposable probe cover. Apply lubricant if desired.
5. Separate the buttocks and gently insert the probe only 1.5 cm (5/8 inch), less for infants and children.

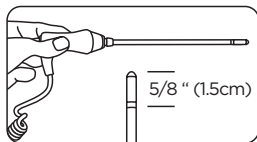


WARNING: Incorrect insertion can cause bowel perforation.



6. Hold the probe in place until the temperature has been captured.
7. Remove the probe and eject the probe cover.
8. Wash hands.
9. Use of Welch Allyn disposable probe covers is recommended. Use of any other probe cover may result in inaccurate temperature readings.

Monitor Mode

If you question a reading, recheck the temperature using the monitor mode on your thermometer. Take a temperature as desired.

- **692 model:** Withdraw the probe from the probe well and press the dedicated Monitor Mode  button to begin taking a reading.
- **690 model:** Continue to hold the probe in place after the temperature reading displays. Press the Mode Selection button  until the Monitor Mode  indicator appears on screen.
- Continue taking the temperature for three minutes.
- Long-term continuous monitoring is not recommended in any mode.

Clean and disinfect the SureTemp Plus thermometer on a routine basis according to each facility's policy whenever contamination occurs. For recommended cleaning and disinfection instructions, please refer to our Directions for Use manual.