



**Technical Service Bulletin**

**PATIENT MONITORS**

Welch Allyn, Inc.<sup>®</sup>  
 4341 State Street Road  
 Skaneateles Falls, New York  
 13153-0220 USA

Tel: 1-800-535-6663 Fax: 1-315-685-3361  
[www.WelchAllyn.com](http://www.WelchAllyn.com)

**Bulletin Type: AS NEEDED**

**Manufacturer: Welch Allyn**

**Date: 2015-03-15**

**SAP DIR #: 20013978**

**THIS DOCUMENT CONTAINS CONFIDENTIAL INFORMATION**

It can be distributed or made available to only those individuals, companies, and organizations which have current Welch Allyn Confidentiality Agreements, and have a need to know.

<b>Subject:</b>	TSB-SPOT LXI POWER-SUPPLY_BRACKET UPDATE	
<b>Distribution Scope:</b>	Welch Allyn Global Technical Support, Product, Field & Customer Service Teams. WA ASP's	
<b>Product(s) Referenced:</b>	All SPOT LXI 4500 Devices	
	All SPOT LXI 4500 power-supplies and power cords	
	80018582 / BRACKET KIT INSTRUCTIONS	
	<b>Part #</b>	<b>Description</b>
	4500-PS	SPOT LXI POWER SUPPLY WITH BRACKET KIT. Kit Includes part numbers: <ul style="list-style-type: none"> <li>• 721481 / 9V POWER SUPPLY</li> <li>• 410244 / BRACKET KIT_LARGE</li> </ul>
	761076-0	POWER CORD, DETACH.(DOM/JAPAN)
	761076-4	POWER CORD-DEATCHBLE (UK)
761076-2	POWER CORD-DETACH (EUROPEAN)	
714340	PLFM PWRCRD AUSTRALIA/NEW ZEALAND-ORN	

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.

<b>Summary:</b>	A new power-supply and bracket kit for Spot LXi has been released. This TSB communicates the new part numbers that will be used for replacing the customers' current power-supplies and power cords.
<b>Issue:</b>	The new power-supply, when used with the included bracket, attaches the power-supply to the bottom of the basket on the mobile stand. This configuration allows the power-cord to easily detach from the power-supply when the stand is moved.
<p><b>Action:</b></p> <p>When it is determined that an LXi power-supply is being replaced, confirm and note the part number of the power-supply that is being replaced.</p> <p>When the power-supply that is being replaced is part number 721481 / 9V POWER SUPPLY:</p> <ul style="list-style-type: none"> <li>Part number 4500-PS / SPOT LXI POWER SUPPLY WITH BRACKET KIT will be provided as the replacement.</li> </ul> <p>When the power-supply that is being replaced is not part number 721481 / 9V POWER SUPPLY:</p> <ul style="list-style-type: none"> <li>Part number 4500-PS / SPOT LXI POWER SUPPLY WITH BRACKET KIT <u>and</u> the country-appropriate power cord, listed in the table above, will be provided as the replacement.</li> </ul> <p><u>Note:</u> Refer to DIR 80018582 / BRACKET KIT INSTRUCTIONS for power-supply and bracket installation details.</p>	
<b>Reference to Standards:</b>	<ul style="list-style-type: none"> <li>21 CFR Part 820, ISO 13485, MPD SOP-0002</li> </ul>
<b>Quality Documents:</b>	<p><b>All service centers using SAP to record service transactions:</b> For each product serviced, record the service activity in SAP.</p> <p><b>All other service centers and Field Service:</b> For each product serviced, complete and file a service report and attach to the service DHR.</p>

**Notes:**

- Contact Welch Allyn regarding any questions or concerns from this or other issues.
- Drawings, illustrations, and part numbers in this document are for reference purposes only and subject to change.

**End of Bulletin**

<b>Revision History</b>					
<b>Version</b>	<b>Description</b>	<b>Change #</b>	<b>Init</b>	<b>Release Date</b>	<b>Appr</b>
A	Initial Release	D*	SMB	D*	D*
D* SEE SAP DIR FOR CHANGE NUMBER, APPROVER NAME AND DATE OF APPROVAL					