


<b>Document Description:</b> TSB- WAST - ModPG plugin 1.3.0.4 Release	<b>Document Number:</b> 20014100 <b>Version:</b> A
	Page 1 of 2
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

## Technical Service Bulletin

<b>Product:</b>	WELCH ALLYN SERVICE TOOL v1.8.0.0	<b>Date:</b>	2015-10-26
<b>Subject:</b>	TSB- WAST - ModPG plugin 1.3.0.4 Release		
<b>HW Version(s) Affected:</b>	N/A	<b>SW Version(s) Affected:</b>	1.8.0.0
<b>Serial Numbers Affected:</b>	N/A	<b>Lot or Date Code Affected:</b>	N/A
<b>Classification:</b>	Informational Only		
<b>Distribution:</b>	<input checked="" type="checkbox"/> Customer Care <input checked="" type="checkbox"/> Product Service <input checked="" type="checkbox"/> Field Service <input checked="" type="checkbox"/> ASPs <input checked="" type="checkbox"/> Distributors <input type="checkbox"/> Customers <input type="checkbox"/> Company Confidential		
<b>Training Required:</b>	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No		

**Summary:** The Welch Allyn Service Tool ModPG plugin 1.3.0.4 is being released with enhanced functionality to support the Connex Spot Monitor, CVSM, CIWS, and ModPG devices.

**Reference Documents**

SWF: WAST 1.8.0.0	50011612
WAST Installer v.1.8.0.0 SRN	60069496
WAST Unresolved Anomalies Report	60051519
SWF: WAST ModPG Plugin 1.3.0.4	50011787
SRN: WAST Mod PG Plug-in 1.3.0.4	60072692

Users should discontinue use of previous version of the Welch Allyn Service Tool and uninstall their current installed version of the Welch Allyn Service Tool, PartnerConnect (RSDS), and Welch Allyn USB driver from your PC using Add/Remove Programs if they are not running WAST 1.8.0.0 or greater version of software.

Download the current released version of the Welch Allyn Service Tool and the Installation guide from the Welch Allyn web site <http://www.welchallyn.com/en/service-support/service-center/service-tool.html>

The WAST 1.8.0.0 package contains the full version of the tool and it is not necessary to have an older version of the tool already installed. The ModPG plugin update will be loaded up in PartnerConnect and set to auto deploy to WAST 1.8.0.0 users only. Users that don't have WAST 1.8.0.0 will not receive this update until they upgrade WAST to the latest version.

The following table summarizes the changes between the previous ModPG Plugin release (v1.3.0.3) and the current release (v1.3.0.4).


Bugs addressed are listed below:

TFS ID#	Description
69994	Corrected mis-spelled word in service Logs - Celsius
70178	Service Logs after Power Calibration Primary/Safety results have missing actual values for user input voltage, current battery voltage (primary & safety) and deviation limit
69611	When Running NIBP accuracy Test and click done, NIBP EEPROM becomes corrupted

### **Software Compatibility**

The following table identifies the Hardware/Software systems used in testing the ModPG v 1.3.0.4 plugin for this release.

Mod PG Module Firmware	2.0.5
Tested Operating Systems:	Windows 7 Professional x64 Windows XP Professional SP3
Welch Allyn Driver Set	<ul style="list-style-type: none"> <li>a) WACP .NET Wrapper API (<b>WelchAllyn.IO.dll</b>) Version <b>2.11.4.25980</b></li> <li>b) WACP CDLL (<b>WACP.dll</b>) Version <b>2.10.3.0</b></li> <li>c) <b>FileTransfer.dll</b> Version <b>1.0.6.25981</b></li> <li>d) <b>RIFF.dll</b> Version <b>1.0.0.25980</b></li> <li>e) <b>Upgrade.dll</b> Version <b>2.3.1.25981</b></li> <li>f) <b>WAUSB</b> driver <b>1.9.11</b></li> </ul>

<b>Document Description:</b> TSB- WAST - ModPG plugin 1.3.0.4 Release	<b>Document Number:</b> 20014100 <b>Version:</b> A
	Page 3 of 3
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

<b>Version</b>	<b>Sec, Pg, Para Changed</b>	<b>Change Made</b>	<b>Date Version Created</b>	<b>Version Created By (initials)</b>
A	N/A	Initial Release	2015-05-11	KMG

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.	
MPD FCD-1482 Rev 7	Page 3 of 3