


<b>Document Description:</b> TSB- WAST – Braun 6000 plugin 1.2.0.0 Release	<b>Document Number:</b> 20014127 <b>Version:</b> A
	Page 1 of 2
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

## Technical Service Bulletin

<b>Product:</b> WELCH ALLYN SERVICE TOOL v1.8.0.0	<b>Date:</b> 2015-12-21
<b>Subject:</b> TSB- WAST Braun 6000 plugin v.1.2.0.0 Release	
<b>HW Version(s) Affected:</b> N/A	<b>SW Version(s) Affected:</b> 1.8.0.0
<b>Serial Numbers Affected:</b> N/A	<b>Lot or Date Code Affected:</b> N/A
<b>Classification:</b> Informational Only	
<b>Distribution:</b> <input checked="" type="checkbox"/> Customer Care <input checked="" type="checkbox"/> Product Service <input checked="" type="checkbox"/> Field Service <input checked="" type="checkbox"/> ASPs <input checked="" type="checkbox"/> Distributors <input type="checkbox"/> Customers <input type="checkbox"/> Company Confidential	
<b>Training Required:</b> <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No	

**Summary:** The Welch Allyn Service Tool WAST Braun 6000 plugin v.1.2.0.0 is being released with enhanced functionality to support the Connex Spot Monitor with Braun 6000, and Braun 6000 Standalone devices.

**Reference Documents**

SWF: WAST 1.8.0.0	50011612
WAST Installer v.1.8.0.0 SRN	60069496
WAST Unresolved Anomalies Report	60051519
SWF: WAST Braun 6000 plugin v.1.2.0.0	50011797
SRN: WAST Braun 6000 plugin v.1.2.0.0	60072997

Users should discontinue use of previous version of the Welch Allyn Service Tool and uninstall their current installed version of the Welch Allyn Service Tool, PartnerConnect (RSDS), and Welch Allyn USB driver from your PC using Add/Remove Programs if they are not running WAST 1.8.0.0 or greater version of software. Download the current released version of the Welch Allyn Service Tool and the Installation guide from the Welch Allyn web site <http://www.welchallyn.com/en/service-support/service-center/service-tool.html>

The WAST 1.8.0.0 package contains the full version of the tool and it is not necessary to have an older version of the tool already installed. The Braun plugin update will be loaded up in PartnerConnect and set to auto deploy to WAST 1.8.x.x users only and WAST will be versioned to 1.8.2.0. Users that don't have WAST 1.8.0.0 will not receive this update until they upgrade WAST to the latest version.

The following table summarizes the changes for the release of WAST Braun 6000 version 1.2.0.0

Issues addressed are listed below:

TFS ID#	Description
70913	Change label on Braun 6000 UI to match the DFU
70260	Enabling PerfecTemp in the Braun 6000 device information.

**Software/Hardware Compatibility**

The following table identifies the HW/SW systems used in testing the Braun 6000 Plugin:

Tested Operating System:	Windows 7 x86/x64 Windows 8 x64 Windows 8.1 x86
Braun 6000 Plug in Version	1.2.0.0
Braun 6000 Module Firmware	3.85, 3.98

Version	Sec, Pg, Para Changed	Change Made	Date Version Created	Version Created By (initials)
A	N/A	Initial Release	2015-12-21	KMG

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.