




Mobile Stand Mounting Plate Instructions

The following information will allow you to identify mobile stand mounting plates for replacement. These following instructions will guide you through removing the monitor and existing mounting plate from your mobile stand, installing the new mounting plate, adjusting the mobile stand swivel-head tension and installing your monitor onto the mobile stand.

Affected Mobile Stands:


Welch Allyn part number 008-0835-01
Manufactured July 8, 2004 through November 7, 2005
Manufacturer's Date Codes: 3Q04, 4Q04, 1Q05,
2Q05, 3Q05, 4Q05

Manufacturer Date Code
Example shown is
forth quarter 2005

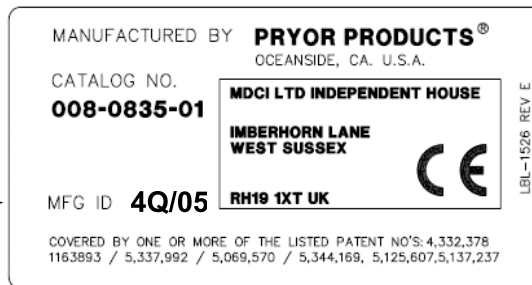


Note: This label is located under the base of the mobile stand.

Identifying Mounting Plates

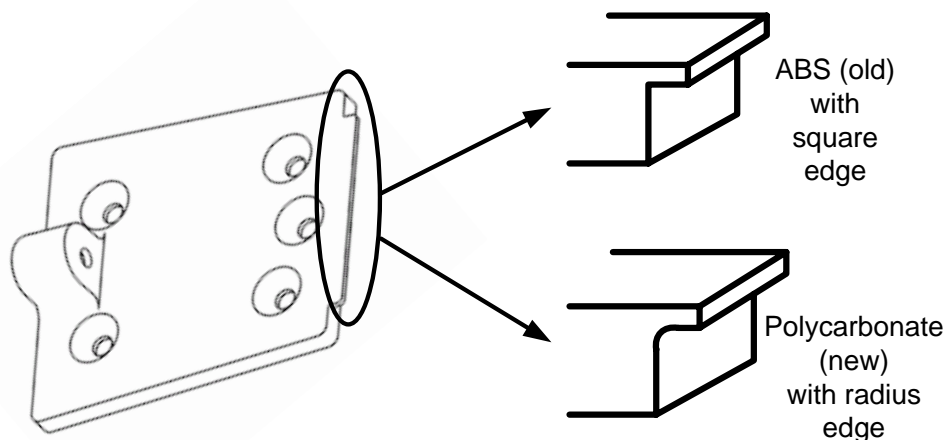

Mobile stand mounting plate kit includes

<u>Item</u>	<u>Description</u>	<u>Quantity</u>
1)	Retaining knob screw	1
2)	Retaining washer	1
3)	Mounting plate	1
4)	Security hex-head screw	1
5)	Security hex-head wrench	1
6)	Mobile Stand Mounting Plate Instructions *	1
7)	Instruction sheet *	1
*	(Not Shown)	

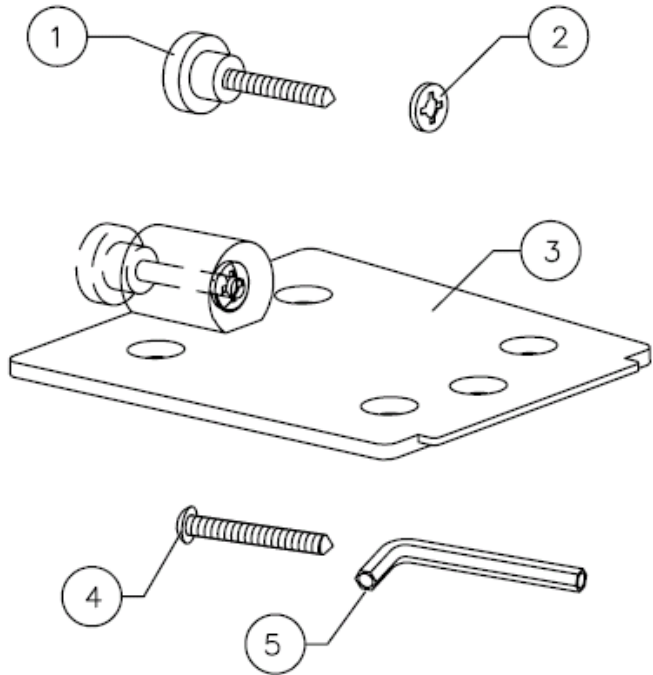


Required Tools

7/16 " nut driver or socket wrench,
Philips screw driver (#2 or medium).


Remove the monitor

While holding the monitor, loosen the retaining knob screw. Remove the monitor.

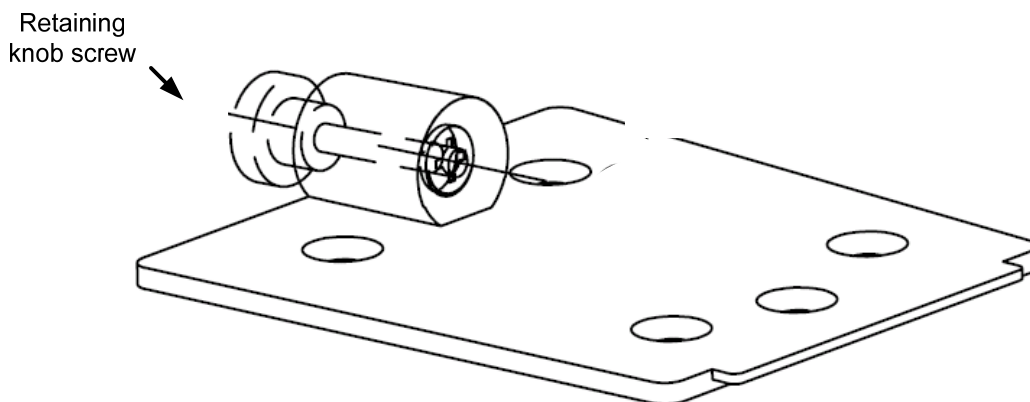


Remove the existing mounting plate:

1. Remove the 7/16 inch Esna Nuts (Nylon kept Nuts, 2 each) and the 7/16 inch washers (2 each) as shown below.

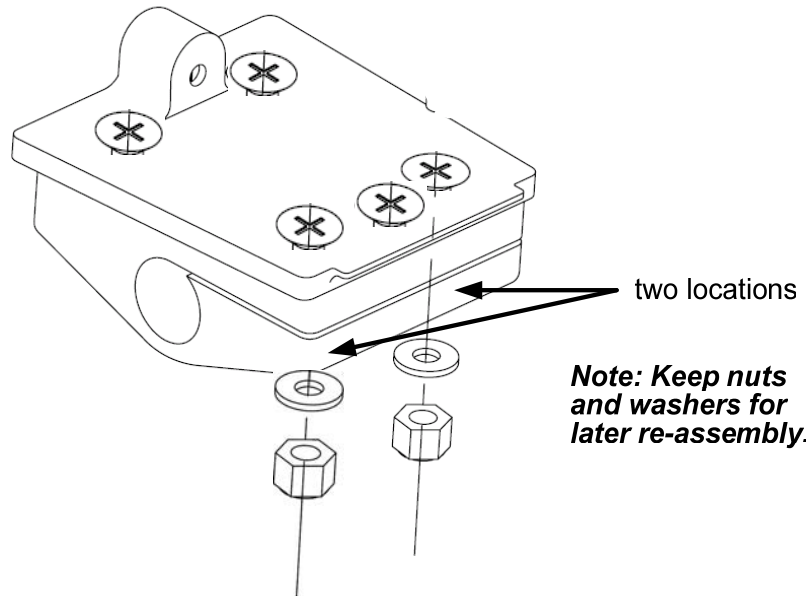


2. Remove the pan-head Phillips screws (5 each).

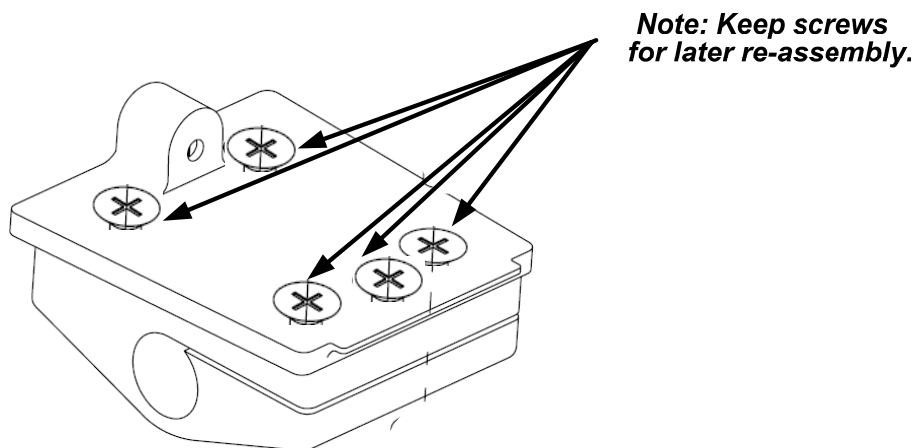


Install the new mounting plate

1. Install the new polycarbonate mounting plate from this service kit (020-0657-00) onto the top of the mobile stand swivel-head piece. Orient mounting plate as shown below. Align the mounting plate holes with the mobile stand swivel-head piece threaded inserts.
2. Install *short* pan-head Phillips screws into locations **A** (3 each, ¼ - 20 x ½ inch).

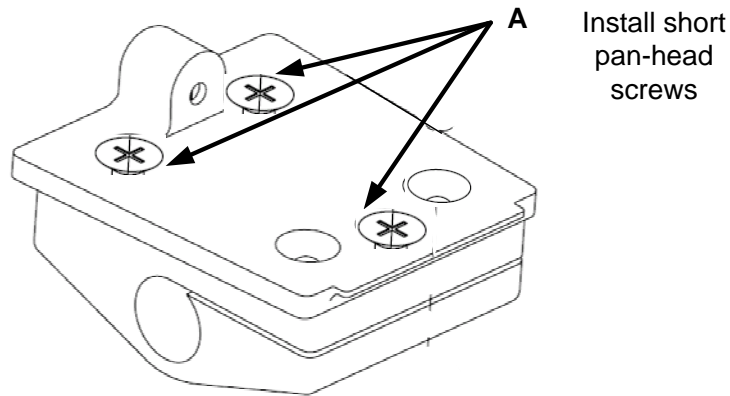


3. Install *long* pan-head Phillips screws into locations **B** (2 each, ¼ - 20 X 1 ½ inch).

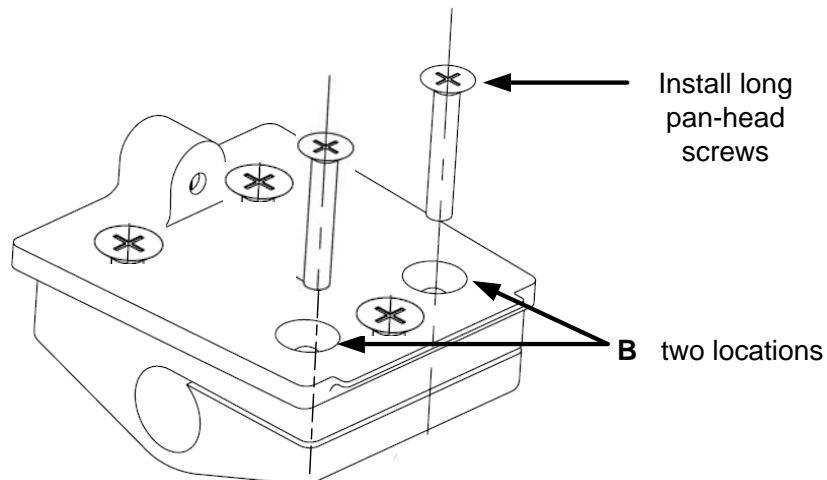


4. Securely tighten all screws at locations **A** and **B**.

5. Install 7/16 inch washers (2 each) and 7/16 inch Esna Nuts (2 each) at locations **C** shown below.

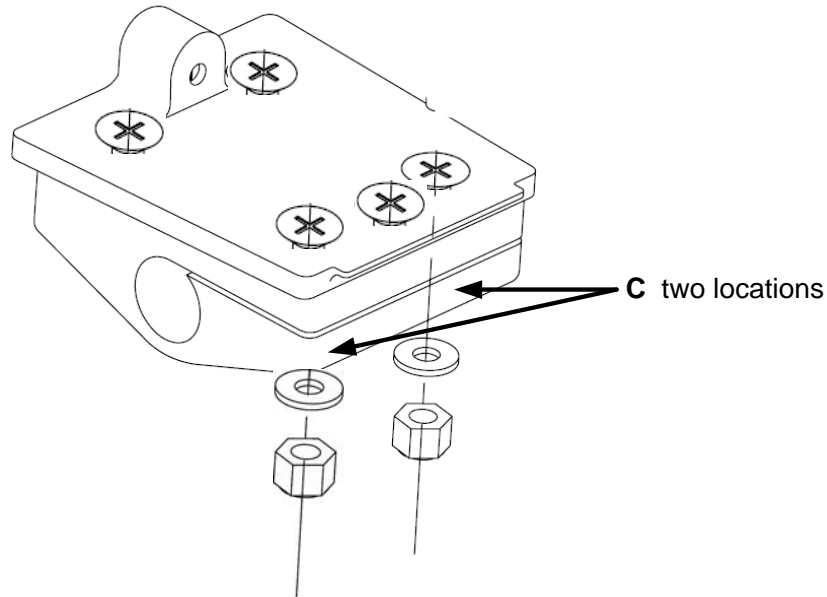


Adjust the mobile stand swivel-head tension: Tighten both of the Esna Nuts at location **C** equally until the desired tension is achieved. The original factory adjustment is 4 lbs \pm 1 lb as measured from the monitor handle being pulled with a tension-measurement device.

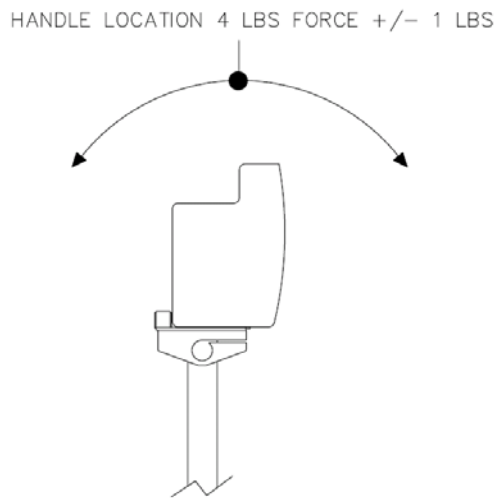

NOTE : *Never turn the pan head screws at location B to make a tension adjustment. Tension adjustment is set only with the Esna Nuts (Nylon kept Nuts) at location C.*

Install the monitor

When installing the monitor on the mounting plate, be sure the plate tongue is placed properly into the monitor mounting slot. Then tighten the retaining screw securely.

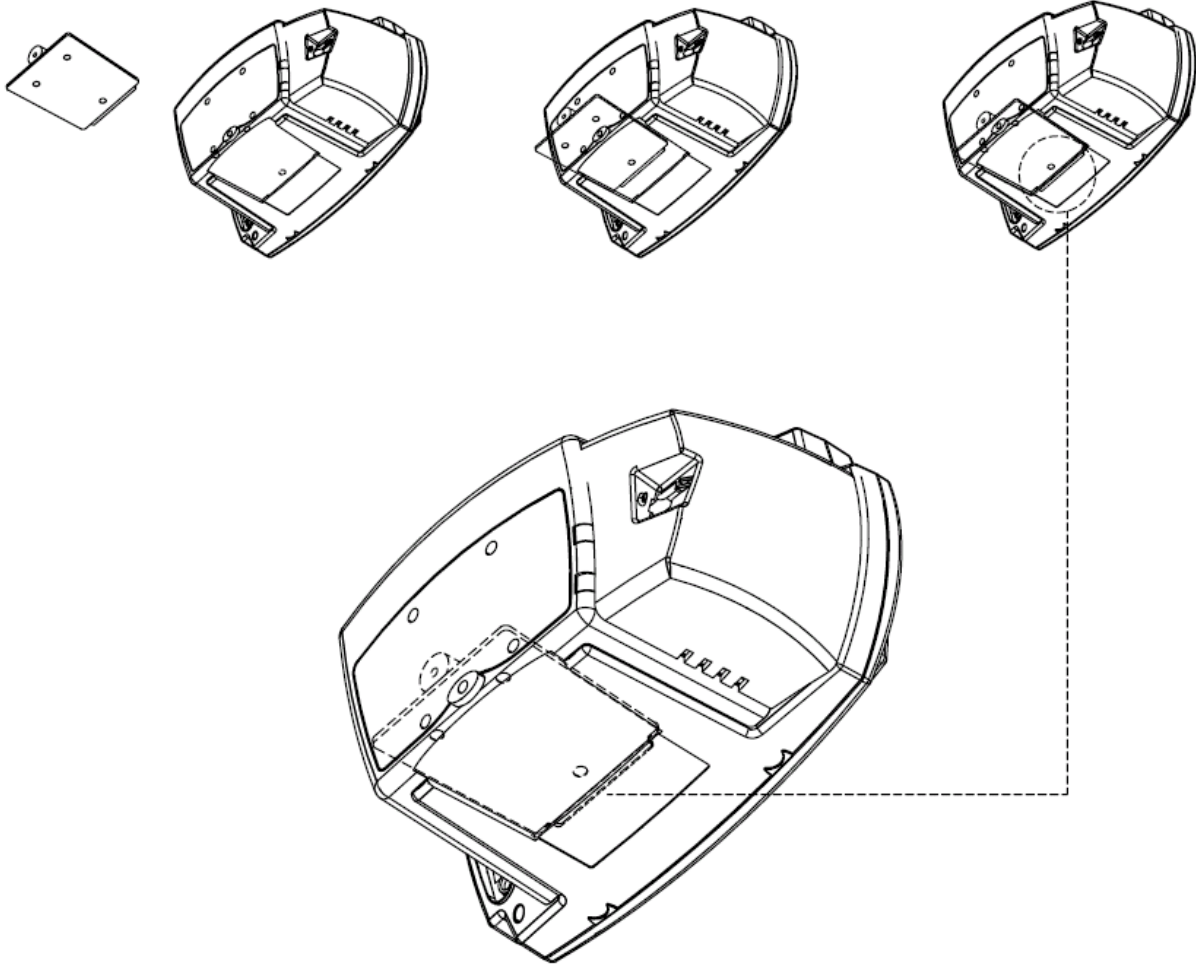

The security hex-head screw can be installed in place of the standard thumb knob screw if desired.



If you have questions call Welch Allyn technical service at USA: 1(800) 289-2501

Canada: 905 890-0008 Singapore: +65-62915780 Australia: 61-2-9638-3500
Europe, Middle East, Africa: +49 7477 9271 93 China: +8621-63279632
South America: 305-669-8951